SAFEHIP[®]
SELECT PROTECTION

ACTIVE

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User Manual

SAFEHIP® SELECT PROTECTION

SAFEHIP® provides clinical and economic solutions in the area of hip fracture prevention for individuals who have fallen or are at risk for falling.

SAFEHIP® ACTIVE

SAFEHIP® ACTIVE is easy to put on. We recommend that you follow the User Instructions listed to the right. The product can be worn both under and over your everyday clothing. If you have any doubt where the greater trochanter is, please consult a nurse or any other professional caregivers to guide you in fitting the product correctly. At regular intervals during daily activities, it is recommended you make sure that your SAFEHIP® ACTIVE is positioned correctly as shown in the User Instructions.

For Hard shields, it is important that the dome shape creates a bow over the greater trochanter. It is important that both shields are correctly centred over greater trochanter as shown in Diagram 1. Correct positioning of Soft shields can be ensured by inserting two fingers in the open protective space between the arms of the horseshoe and by making sure that the bony prominence of the greater trochanter can be felt and is positioned as shown in Diagram 2. The centre of the greater trochanter should be positioned centrally in the free protective space between the arms of the horseshoe. For Soft shields, it is very important that the open end of the horseshoe is facing downward towards the feet.

Washing Instructions

- Machine wash 40°C (104°F)
- Wash SAFEHIP® ACTIVE before using it for the first time.
- Close all Velcro® fastening areas before washing.
- Place SAFEHIP® ACTIVE in a mesh-washing pouch to protect other clothes that are washed simultaneously.
- Do not use bleach or washing powder with bleach as this will damage SAFEHIP®.
- Should it become absolutely necessary to disinfect SAFEHIP® ACTIVE, we recommend using Virkon®¹.

Important Information

SAFEHIP® ACTIVE has been developed as a hip protection device for the elderly users in daily life activities such as walking, jogging, household activities, and the like. If SAFEHIP® ACTIVE is used in sports activities where the energy in falls may be considerably greater than that of daily life activities, the protection offered by the product may be insufficient to reduce the risk of fracture. SAFEHIP® ACTIVE will not prevent all hip fractures and cannot be guaranteed to be 100% effective. If you have a fall while wearing SAFEHIP® ACTIVE, it is highly unlikely that you will damage the shields. If you do have a fall, simply run your fingers over the shields and along the outer edge of each shield and feel for any cracks. After washing and before wearing the product, the shields should always be checked. If any cracks or damages are found, the use of the product must be discontinued.

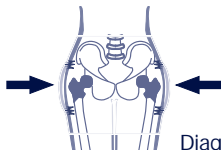
Diagram 1

Diagram 2

¹ Refer to user manual for Virkon® (<http://relyon.dupont.com/downloads/virkonifu.pdf>)

User Instructions

1. Position SAFEHIP® ACTIVE so the shields are positioned correctly on top of the greater trochanter and femoral head is protected.

(Note: The belt is placed in most cases under the waistline.)

2. Close Velcro® fastening and adjust for comfort and fit. The product should be adjusted so that it does not shift or turn around the body.

3. Do not tighten SAFEHIP® ACTIVE excessively so it impairs your comfort.

4. Please check that the SAFEHIP® ACTIVE and especially the shields are positioned as shown in Diagram 1 and Diagram 2.

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Kuhn und Bieri AG