# MED-COMFORT®

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**HONSUN (NANTONG) Co., Ltd.**  
No.8 Tongxing Road · Economic & Technological · Development Area  
226009 Nantong City · CHINA

**REF** 09850

# MED-COMFORT®

## Aneroid Sphygmomanometer

Read these instructions carefully before using the sphygmomanometer and save it for future reference.

### Intended use

Aneroid sphygmomanometers are intended exclusively for the measurement of blood pressure on healthy skin on the upper arm.

The blood pressure meter must be calibrated every two years.

### APPLICATION

The sphygmomanometer is suitable for use in hospital, at home, at work, at school and when travelling.

#### 1. Preparation

Relax and make yourself comfortable.

- It is very important that the person, whose blood pressure reading will be taken, is relaxed and sitting comfortably.
- For taking your blood pressure reading, rest your left arm at heart level; stretch your arm loosely and not too stiff.
- Rest your left arm on an even surface, e.g. on a table, with the palm up. Keep your arm still whilst measuring.

#### 2. Application of the Velcro cuff and the chest piece

- Place the cuff 2 to 3 cm above the elbow on your bare left upper arm; close the cuff with the Velcro. The cuff should be applied comfortably and not too tightly. There should be space for one or two fingers between the cuff and the arm. Never apply the cuff on top of clothes.
- Pull the free end of the cuff through the D-ring and close the cuff with the Velcro.

#### 3. Inflation of the cuff

- Close the air valve of the ball by turning the screw clockwise.
- Press the ball in steady intervals until either the needle or the measuring point is 30 mmHg higher than your normal systolic pressure value. If you do not know this value, only pump up to 200 mmHg.

#### 4. Systolic blood pressure – higher measured value

- Open the air valve slowly by turning the screw anti-clockwise and hold the chest piece of the stethoscope over the artery of the upper arm. A correct air outlet rate is essential for exact measurement. Therefore you should practice and master the recommended air outlet rate of 2 to 3 mmHg per second or a drop of one to two markings on the pressure reading per heartbeat. You should not keep the cuff inflated longer than necessary. When the cuff begins to deflate, you have to listen carefully using the stethoscope. As soon as you hear a weak rhythmic knocking or knocking sounds, make a note of the measured value. This is the measured value of the systolic blood pressure, the higher value. Listen carefully and make yourself familiar with the noise of your pulse. Check the procedure once more with your doctor.

#### 5. Diastolic blood pressure – lower measured value

- Deflate the air at a uniform deflation rate. The knocking sound stops as soon as you have reached the diastolic, the lower measured value.
- Deflate the cuff valve entirely. Remove the cuff from the arm and the stethoscope from your ears.

#### 6. Make a note of your measured values

- Repeat the measurement several times daily. Don't forget to make a note of your measured values and of the time of day immediately after the measurement.
- An appropriate time is in the morning after getting up or before dinner. Don't forget that your doctor is the only qualified person to analyse your blood pressure.

### CARE

With good care and maintenance your blood pressure set will be of long-term service to you.

#### Basic rules:

- Don't drop or shake the instrument.
- Never inflate the cuff higher than 300 mmHg.
- Never expose the cuff to intensive solar radiation!
- Never touch the cuff or parts of it with any sharp object as this could cause damage!
- Always deflate the cuff entirely before putting it away.
- You should by no means remove the manometer.
- Keep the instrument in the bag which comes with it, in order to keep all parts clean.
- Storage temperature: -20° C to +70° C with a relative humidity of 85 % (non-condensed!).
- Wipe the manometer and the ball with a damp cloth. It is not necessary to sterilize the parts as they are not in direct contact with the body of the patient.
- Wipe the Velcro and the hose with a damp cloth. The cuff may be washed with soap and cold water. Remember to rinse the cuff with clear water afterwards and to let it air-dry.