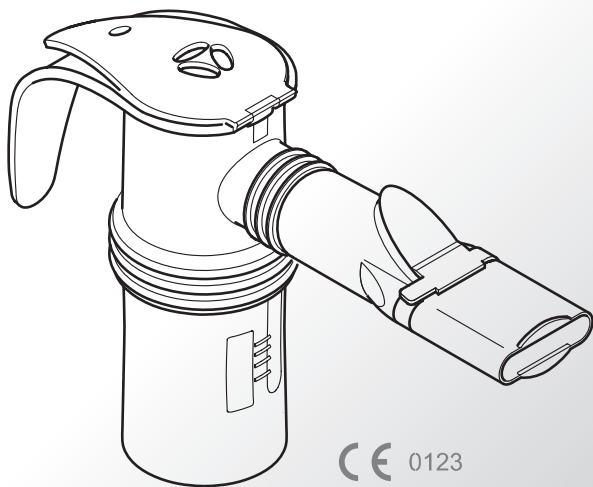
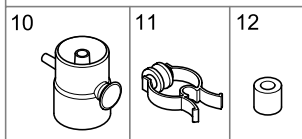
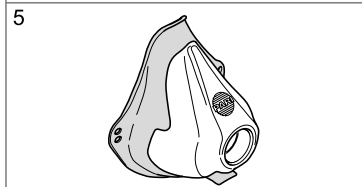
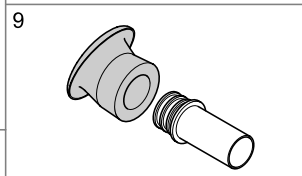
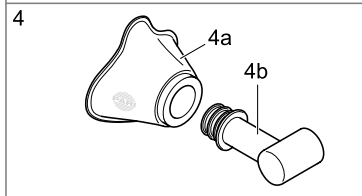
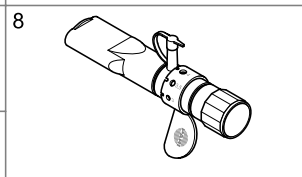
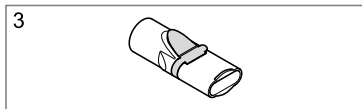
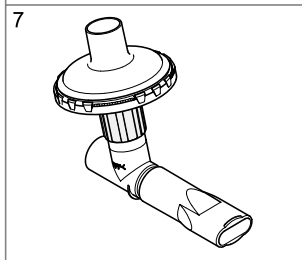
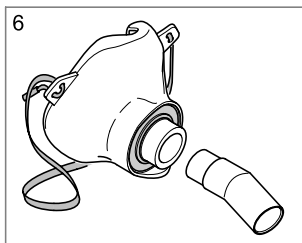
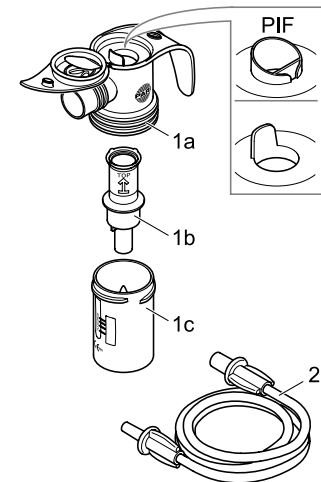
LC SPRINT® Familie

CE 0123

de	Gebrauchsanweisung	3
en	Instructions for use	21
da	Brugsanvisning	39
el	Οδηγίες χρήσεως	57
es	Instrucciones de uso	75
fi	Käyttöohje	93
fr	Notice d'utilisation	111
it	Istruzioni per l'uso	129
nl	Gebruiksaanwijzing	147
no	Bruksanvisning	165
pt	Manual de Instruções	183
sv	Bruksanvisning	201
tr	Kullanım kılavuzu	219

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INSTRUCTIONS FOR USE

Table of contents

1	IMPORTANT INFORMATION	22
2	PRODUCT DESCRIPTION	25
2.1	Components	25
2.2	Intended use.....	26
3	INHALATION	28
3.1	Assembling the nebuliser	28
3.2	Filling with medication	30
3.3	Performing the inhalation	31
4	CLEANING AND DISINFECTION	33
4.1	Preparation.....	33
4.2	Care of the connection tubing	33
4.3	Cleaning	33
4.4	Disinfection.....	34
4.5	Drying and storage	35
5	MISCELLANEOUS	36
5.1	Disposal.....	36
5.2	Contact.....	36
5.3	Technical data	36
6	REPLACEMENT SETS, SPARE PARTS AND ACCESSORIES	37

1 IMPORTANT INFORMATION

Please read these instructions for use through in their entirety. Do not discard so you can consult them at a later date. If you fail to comply with the instructions for use, injury or damage to the device cannot be ruled out.

These instructions for use are destined for the user at home. Separate instructions for use are available for technical personnel on the Internet under "Products" at www.pari.de/en/Products (on the relevant product page under "Products") or as a hard copy on request from PARI GmbH.

Safety-relevant warnings are categorised into hazard levels in these instructions for use:

- The signal word **WARNING** is used to indicate hazards which, without precautionary measures, might result in serious injury or even death.
- The signal word **CAUTION** is used to indicate hazards which, without precautionary measures, might result in minor to moderate injury or impair treatment.
- The signal word **NOTE** is used to indicate general precautionary measures which should be observed when using the device to avoid damaging it.

**WARNING**

When treating babies, children and anyone who requires assistance the following must be observed without fail:

- Babies, children and anyone who requires assistance must be supervised constantly by an adult during inhalation therapy. This is the only way to ensure safe and effective treatment. Such persons often underestimate the hazards involved (e.g. strangulation with the tubing), thus resulting in a **risk of injury**.
- The product contains small parts. Small parts might block the respiratory tract and pose a **choking hazard**. Please therefore make sure that they are kept out of the reach of children.
- The PARI babymask is used to treat babies and infants who are not yet able to inhale through a mouthpiece. The baby-mask should only be used in conjunction with the babybend. The baby or infant is able to exhale freely through the slots in the babybend, so avoiding the **risk of shortness of breath**.
- The nebulisers are only designed for patients who are able to breathe by themselves and are conscious. This is the only way to ensure effective treatment and avoid a **choking hazard**.

Patients with an surgical opening in the windpipe (tracheostoma): Patients who have undergone a tracheostomy are unable to breathe in with the mouthpiece and thus require the PARI TRACHEO SET with a special tracheal mask to inhale. When using the tracheal mask, make sure that the nebuliser cap is open during inhalation. This allows the user to exhale freely and avoids the **risk of shortness of breath**.

CAUTION

Replace all nebuliser components after a year at the latest as changes may occur in the material of the nebuliser over time. This might impair the effectiveness of the treatment. This is why we introduced the PARI Year Packs (see chapter 6).

For inhalation treatment only use a nebuliser and accessories which have been cleaned, disinfected and dried according to chapter 4. Contamination and residual moisture encourage the growth of bacteria, so increasing the risk of infection.

Please therefore observe the following instructions on hygiene:

- Before every use and cleaning wash your hands carefully.
 - Clean, disinfect and dry your nebuliser and accessories according to chapter 4. Make absolutely sure you also carry out cleaning, disinfection and drying before using the device for the first time.
 - For cleaning and disinfection always use fresh tap water with drinking water quality.
 - Make sure all components are dried properly whenever you have performed cleaning or disinfection.
 - Do not keep the nebuliser or accessories in a damp environment or together with damp objects.
-

2 PRODUCT DESCRIPTION

2.1 Components

Use the following table to check whether all components belonging to your nebuliser set are included (figure on front cover page). Should anything be missing, please notify the dealer from which you obtained the PARI nebuliser at once.

PARI nebuliser sets ^{a)} of PARI LC Sprint family		TurboBOY Year Pack S	JuniorBOY Year Pack S	BOY SX Year Pack	LC SPRINT	LC SPRINT Junior	LC SPRINT Junior Package	LC SPRINT STAR	BOY SX Package ^{b)}	LC SPRINT BABY
Nozzle attachment (1b)	blue	x		x	x				x	
	yellow		x			x	x			
	red			x				x	x	x
Mouthpiece (3)		x	x	x	x	x	x	x	x	
Babymask (4a) with baby-bend (4b)							x			x
LC interrupter (10)									x	
Filter insert (12) for PARI BOY compressors		x	x	x						
PIF Control System ^{c)}				x				x	x	x

- a) All kits basically include a nebuliser upper section (1a), a nebuliser lower section (1c) and a connection tube (2).
 b) Device
 c) PIF Control System: **Peak Inspiratory Flow**

2.2 Intended use

Used with a PARI compressor, the nebulisers of the PARI LC SPRINT family are designed for inhalation treatment of the lower and upper respiratory tract. Each nebuliser should only be used at home by a single patient for hygienic reasons.

With regular usage all parts of the nebuliser can be used for up to one year. Each application generally takes between 5 and 10 minutes, however not more than 20 minutes (depending on the dosage of medication and the compressor used).

Only approved medication should be used for inhalation treatment. This is generally determined by the doctor in charge. Make sure you take note of any restrictions given in the instructions for use of the relevant medication. The frequency of application depends on the clinical picture and the medication used.

The nebulisers are only designed for patients who are able to breathe by themselves and are conscious.

Only original PARI compressors and PARI accessories should be used. PARI GmbH cannot be held liable for damage caused by using PARI nebulisers with compressors or accessories from other manufacturers.

Please read and follow the instructions for use with your compressor and/or accessory.

The PARI LC SPRINT nebuliser is available in a range of models catering for different age groups and requirements.

- PARI LC SPRINT (nozzle attachment: blue)
Including mouthpiece for universal treatment of the respiratory tract in children from approx. 4 years and adults.
- PARI LC SPRINT Junior (nozzle attachment: yellow)
Specially designed to treat the respiratory tract in infants and children. Used in combination with a babymask to treat babies and infants up to approx. 4 years who are not yet able to inhale through a mouthpiece.
- PARI LC SPRINT STAR (nozzle attachment: red)
Including mouthpiece, specially designed to treat the deep respiratory tract in children up to approx. 4 years and adults, additionally equipped with inspiratory flow control (PIF Control System¹).
- PARI LC SPRINT BABY (nozzle attachment: red)
Specially designed for use with the babymask to treat the respiratory tract in babies (including premature babies) and infants up to approx. 4 years.

1) For further information about the PIF Control System see page 32.

3 INHALATION

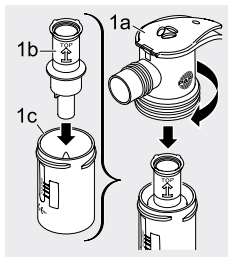
Inhalation with the mouthpiece is the most effective way of treating the lower respiratory tract as this allows the greatest possible quantity of medication to reach the lung. If it is not possible to use a mouthpiece, or if the patient prefers to inhale through a mask, PARI can provide the right accessories.

3.1 Assembling the nebuliser

CAUTION


Check the parts of your nebuliser before every application. Replace any broken, misshapen or seriously discoloured parts. Please also follow the instructions for assembly given below. Damaged components and/or an incorrectly assembled nebuliser may impair **functioning of the nebuliser** and thus **treatment as well**.

- Gently push the nozzle attachment (1b) as far as it will go onto the nozzle in the nebuliser lower section (1c) of the nebuliser. The arrow on the nozzle attachment must point upwards.
- Place the nebuliser upper section (1a) on the nebuliser lower section and turn it clockwise to lock the two parts together.

Use with babybend and babymask

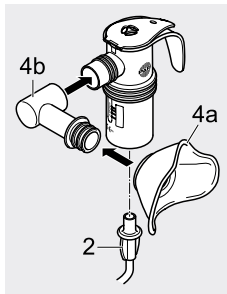
- Place the babymask (4a) on the babybend (4b).
- Use your thumb to press the babymask from the inside over the first lip of the babybend.
- Fit the babybend onto the nebuliser.
- Fit the connection tubing (2) onto the nebuliser and compressor.

Use with mouthpiece or child or adult mask

- Attach the mouthpiece (3) or the child or adult mask (5).
- Connect the nebuliser to the compressor using the connection tubing (2).

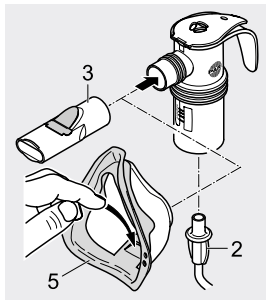i When using a mask, make sure that the expiratory valve plate is pressed out so you can exhale freely during inhalation.

3.2 Filling with medication

NOTE

Take care to ensure that the cap only moves in the direction allowed by the hinge. Otherwise the **nebuliser cap might snap off**.

- Release the closure on the nebuliser upper section (1a) by pressing your thumb against the underside of the cap.
- Pour the quantity of medication prescribed by your doctor into the nebuliser funnel.
- Ensure that the medication is filled no higher than the upper scale marking (max. level 8 ml). Overfilling may greatly impair the nebulisation of medication.
- If you plan to inhale several medications one after the other in one application, rinse out the nebuliser with tap water between inhalations. Remove excess water by shaking the nebuliser and then fill with medication as described.
- Close the cap of the nebuliser. Make sure that the cap snaps into place.

3.3 Performing the inhalation

Before the patient starts inhaling, ensure that all parts are firmly connected to each other.

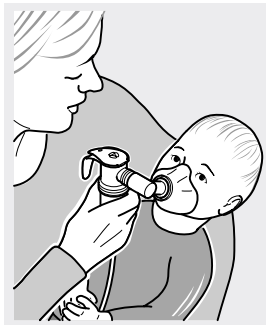
Perform inhalation until the medication is used up (signalled by a change in the sound of the nebuliser).

i Several drops of medication will remain in the nebuliser after inhalation.

Inhalation with babybend and babymask

- Swivel the bend so that it corresponds with the position of the baby or infant.
Always hold the nebuliser upright.
- Switch the compressor on.
- Press the mask gently but firmly over the mouth and nose.
- Perform treatment.

i For best inhalation, your child/baby should breathe in and out as slowly and deeply as possible.

- Switch the compressor off as soon as you have finished inhalation.

Inhalation with mouthpiece, child or adult mask

- Sit in an upright position and relax.
- Switch the compressor on.
- Take the mouthpiece between your teeth and close your lips around it or press the mask lightly over the mouth and nose.
- Breathe in through the mouthpiece or mask as slowly and deeply as possible and then out again in your own time.
- Switch the compressor off as soon as you have finished inhalation.

Information about using a nebuliser with the PIF Control System:

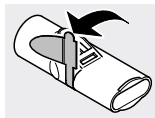
The PARI PIF Control System in the upper section of the nebuliser reduces the air supply, so creating increased resistance when inhaling. The objective here is to help you learn slow, controlled inhalation. If you notice increased resistance during treatment when inhaling, try to breathe in slowly that so such increased resistance is no longer felt. This will improve the uptake of medication in the lower respiratory tract.

4 CLEANING AND DISINFECTION

Clean the nebuliser and the accessories used after every application. Disinfect the nebuliser at least once a day.

4.1 Preparation

- Disconnect the tubing from the nebuliser.
- For this purpose dismantle the nebuliser into its individual parts.
- Carefully pull the expiratory valve out of the slot in the mouthpiece.
The valve should still be attached to the mouthpiece.


4.2 Care of the connection tubing

- Switch the compressor on.
- Leave the compressor running until any condensation in the tubing has been removed by the air flowing into the tube.

4.3 Cleaning

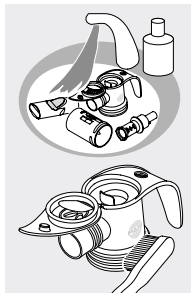- Thoroughly clean all components (except for the connection tubing) in warm tap water and dishwashing liquid for about 5 minutes. If necessary, use a clean brush to remove loose dirt (the brush should be solely reserved for this purpose).
- Then rinse all parts thoroughly under warm running water.
- You can remove excess water more quickly by shaking all parts.

4.4 Disinfection

After cleaning disinfect the dismantled nebuliser and accessories used (effective disinfection is only possible if the nebuliser has been cleaned).

i **Information when using masks with rubber bands:** The rubber band can only be cleaned as it might be damaged by the high temperatures used during disinfection.

In boiling water

- Place the individual parts of your nebuliser in boiling water for at least 15 minutes.

Use a clean pan and fresh drinking water.

i If your water is very hard, scale may be deposited on the nebuliser parts as a milky coating. This can only be prevented by using softened water.

NOTE

Plastic will melt if it comes into contact with the hot base of the pan. You should therefore make sure there is plenty of water in the pan. This way you will avoid **damaging the nebuliser parts**.

Using a standard disinfectant for baby bottles

Disinfection must last at least 15 minutes. As regards disinfection and the quantity of water necessary make sure you observe the instructions for use of the relevant disinfectant. Always check carefully that it is clean and fully functional.

 **CAUTION**

A damp environment may encourage the growth of bacteria. You should therefore remove the nebuliser and accessories from the pan or disinfectant as soon as disinfection has finished. Dry parts according to chapter 4.5. Drying completely reduces the **risk of infection**.

4.5 Drying and storage

- You can remove excess water more quickly by shaking all parts.
- Place the nebuliser parts on a dry, clean and absorbent surface and let them dry completely.
- When drying, between applications, and in particular during long breaks, wrap the nebuliser in a clean, lint-free cloth (e.g. a tea-towel) and keep in a dry, dust-free environment.
- During storage make sure the nebuliser and accessories are protected from continuous exposure to direct sunlight.

5 MISCELLANEOUS

5.1 Disposal

The PARI nebuliser may be disposed of with domestic waste unless the disposal regulations in the respective country provide otherwise.

5.2 Contact

For all product information and in the event of defects or problems with usage please contact our Service Center:

Tel.: +49-(0)8151-279 279 (German-speaking)
+49-(0)8151-279 220 (international)

5.3 Technical data

Nebuliser gases:	Air, oxygen
Minimum flow:	3.0 l/min.
Corresponding operating pressure:	0.5 bar/50 kPa
Maximum flow:	6.0 l/min.
Corresponding operating pressure:	2.0 bar/200 kPa
Minimum fill:	2 ml
Maximum fill:	8 ml

Aerosol data according to DIN EN 13544-1 is available on request from PARI GmbH or on the Internet under "Products" at www.pari.de/en/Products (on the relevant product page under "Technical data").

6 REPLACEMENT SETS, SPARE PARTS AND ACCESSORIES

The figures relating to the components can found on the front cover page.

Pos. no.	Description	Item no. ^{a)}
Replacement sets:		
	PARI JuniorBOY Year Pack S	023G1110
	PARI TurboBOY Year Pack S	023G1010
	PARI BOY SX Year Pack incl. 2 nozzle attachments	023G8510
Spare parts and accessories:		
1	PARI LC SPRINT	023G1000
	PARI LC SPRINT Junior	023G1100
	PARI LC SPRINT STAR	023G1250
	PARI LC SPRINT Junior Package (with BABY mask size 2)	023G1120
	PARI LC SPRINT BABY size 0 (for premature babies)	023G1400
	PARI LC SPRINT BABY size 1 (for age up to 1 year)	023G1401
	PARI LC SPRINT BABY size 2 (for age 1 to 3 years)	023G1402
	PARI LC SPRINT BABY size 3 (from approx. 4 years)	023G1403
2	Connection tubing for PARI compressors (1.2 m)	041B4591
3	Mouthpiece	022E3050

4	BABY mask size 0 with BABY bend (for premature babies)	041G0700
	BABY mask size 1 with BABY bend (for age up to 1 year)	041G0701
	BABY mask size 2 with BABY bend (for age 1-3 years)	041G0702
	BABY mask size 3 with BABY bend (from approx. 4 years)	041G0703
4b	BABY bend	041E0701
5	Adult mask soft	041G0740
	Child mask soft "Spiggy"	041G0741
6	PARI SMARTMASK (inc. 45 degree bend)	041G0730
7	PARI Filter/Valve Set	041G0500
8	PARI PEP S System	018G4000
9	PARI TRACHEO SET	025G0007
10	LC interrupter	022G1000
11	Nose clip	041E3500
-	Filter screw with filter	085B1130

a) Item numbers only valid in Germany.

We reserve the right to make technical changes.

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