Disposable

# Humidifier

**Rx Only**

2917 Weck Drive  
Research Triangle Park, NC 27709 USA  
USA: (866) 246-6990  
International: (919) 544-8000

© 2007, 2010, 2013 Teleflex Incorporated  
83188-12 R01

## ITALIANO

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### Disposable Humidifier

#### Directions for Use:

NOTE: These directions are general guidelines intended for use by qualified medical personnel.

1. Fill the humidifier jar to the maximum fill line with sterile, distilled or bottled water. Tightly secure the lid to the jar.
2. Connect the humidifier lid wing nut to the oxygen flowmeter outlet.
3. Attach oxygen tubing (not supplied) to the humidifier lid outlet stem. Be sure all connections are secure.
4. Turn the oxygen source to the prescribed flow.

**CAUTION:** An alarm whistle from the pressure relief valve will activate if the jar accumulates too much pressure. Inspect the system for excessive block-age or a kinked tube.

To test the pressure relief valve, occlude oxygen tubing at 3 LPM. Listen for an alarm condition.

**WARNING:** Oxygen supplies of 10 psi or less using low flow (less than 2 LPM) may affect the audible alarm.

**WARNING:** Re-processing of products intended for single use only may result in degraded performance or a loss of functionality. Re-use of single use only products may result in exposure to viral, bacterial, fungal, or prionic pathogens. Validated cleaning and sterilization methods and instructions for repro-cessing to original specifications are not available for these products. This product is not designed to be cleaned, disinfected, or re-sterilized.

Hudson RCI is a registered trademark of Teleflex Incorporated and its affiliates.

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