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EN Instructions for Use

Art.-No. 80000586 Natural peat hot water bottle

Art.-No. 80006534 Natural peat kidney warmer with belt

Art.-No. 80006533 Natural peat neck warmer

Please read the instructions for use carefully so that you continue to enjoy using this natural peat product for a long time!

1. Performance data:

1.1 Material

Filling: 100 % Natural peat

Fabric cover: 100 % cotton TEX, (Code 60)

PVC-film (Code 3)

Product packaging: corrugated cardboard (PAP, Code 20)

1.2 dimensions:

Natural peat hot water bottle

Dimensions : 20 x 15 cm

Natural peat kidney warmer with belt

Dimensions: 38 x 16 cm

Belt length: 28 cm per side

Natural peat neck warmer

Dimension: 38 x 30 cm with notches

2. Intended Use, Indication

After being heated or cooling off the natural peat product provides 3 hours of warmth or cooling.

2.1 The heat application is always recommended for pain, muscle tension, joints and spinal columns, for pains of a non-inflammatory origin, menstrual pain, in sub-acute and chronic arthritis pains or only for warming up.

2.2 The cold application is recommended for contusion, bruises, headaches (migraines), tooth aches, insect bites, bloody noses, muscle and joint inflammation, swelling.

3. Contraindication:

Heat application should not be used for pains of inflammatory origin and circulatory problems.

4. Warnings:

- Do not use a defective product. Any guarantee is voided by the use of a defective product!
- Sharp, pointed and hot objects can destroy the PVC-shell of the natural peat -product. Note that hot natural peat can escape and get on your skin!
- If stored in an environment that is too warm the natural peat can dry out through natural evaporation through the PVC-shell. This will make it no longer usable. Please observe all of the information about storage conditions below under Point 8.
- Flush eyes with clear water in the event of eye contact.

- The product is laboratory-tested. No side effects were identified on contact with the skin. However, side effects cannot be 100% predicted.
- If you have allergies to or your skin is sensitive to one of the materials used, consult a doctor before use. If sensitivities are registered during use, stop treatment immediately and consult a physician.

5. Preparation and Reprocessing

Remove all packaging including the cotton shell from the natural peat product before you heat it up or cool it off.

5.1 Heat Application:

The desired temperature for the application of the natural peat product comes to 50 – 60 °C. You should check the temperature of the natural peat product constantly if possible while heating it up.

NEVER LEAVE THE PRODUCT UNWATCHED WHILE IT IS BEING HEATED UP!

5.1.1 Heating up in the microwave: Turn the edges downward and lay the product on the turntable so that it does not touch the edges of the microwave. Warm for a maximum of 3 minutes at 600 W. If the product increases in volume, immediately stop warming it and check to see if the packaging was damaged. ⚠ If it is damaged, the product may no longer be heated and used! The product may only be heated again after it has cooled off completely. If more frequent warming is necessary, then heat it at 10 second intervals until the ideal temperature is reached (in total no more than 2 minutes, though).

5.1.2 Heating in a gas, convection or electric oven: Turn the edges down and lay the product on a porcelain plate such that it does not touch the side of the oven. Heat up for a maximum of ten minutes at 150°C. If the product increases in volume immediately stop heating and check to see if the packaging was damaged. ⚠ If damaged, the product may no longer be heated or used.

5.1.3 Heating in a Water Bath: Lay the product in a container with cold water. Heat up to around 80°C. After reaching the temperature turn the heating surface off and leave the pad in the water bath for another 5-10 minutes. Then take the product out and dry it off. ⚠ Don't let the water boil! This would irreversibly damage the product.

5.2 Cold Application

5.2.1 Refrigerator: Lay the natural peat product in the refrigerator for 2 hours. The product does not need to be checked while cooling.

NEVER PUT THE NATURAL PEAT PRODUCT IN A FREEZER!

6. Application:

Pull the cotton cover over the cooled or heated natural peat product and place it on the area for application according to the instructions.

7. Cleaning

The cotton cover can be washed at 60°C.

The PVC shell of the natural peat product is sensitive to strong detergents. Wash it off with a towel and warm water as necessary. Use the natural peat product with the clean cotton cover!

8. Storage

A product that has not been unpacked can be stored in its sales packaging at 5-25°C. Once it is unpacked, store the product in a cool, dark place (ideally between 8-15°C). In the case of longer-term storage between uses, the product should be stored in the refrigerator. This will reduce the risk that the natural peat will dry out (natural property of the PVC-packaging).

9. Disposal

The product can be placed in the household rubbish as residual waste.

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