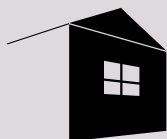
ALIV UP

DOMICILE - HOMECARE

ALIV UP - ALIV UP PLUS

**FAUTEUILS DE TRANSFERT
TRANSFER CHAIRS
TRANSPORTSTUHL**

ALIV UP PLUS

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EN - Instructions for installation and use

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HERDEGEN SAS,
6 rue Marc Seguin
77500 CHELLES – France

Copyright 2024 par HERDEGEN SAS– ALL RIGHTS RESERVED

In the interests of continuous improvement, HERDEGEN SAS reserves the right to change any detail or specification in this document without prior notice.

The information contained in this document is confidential. It may only be shared with a third party with the prior written consent of HERDEGEN SAS. This document may not be reproduced or communicated, in full or in part, in any way or by any means, whether technical or electronic, by photocopying, recording, or using any information or data extraction systems.

You can obtain a new copy of this document from HERDEGEN SAS by contacting your regional sales representative or directly contacting HERDEGEN SAS.

First edition printed in: 2024
Printed in France on certified paper

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FOREWORD

You have just purchased a HACARE product and we are pleased to have you as one of our clients. We also thank you for placing your trust in our company and our products.

Your trust assures us that the HACARE company continues to meet your specific requirements and needs. That's why we always strive to innovate, providing you with quality solutions and products that improve the comfort and quality of patient care while making the caregiver's daily tasks easier.

HACARE products have gained the confidence of many professionals and institutions due to their functional and aesthetic qualities, and are now considered to be a solid benchmark. Simplicity, reliability and safety are the hallmarks of our electrical healthcare cradles.

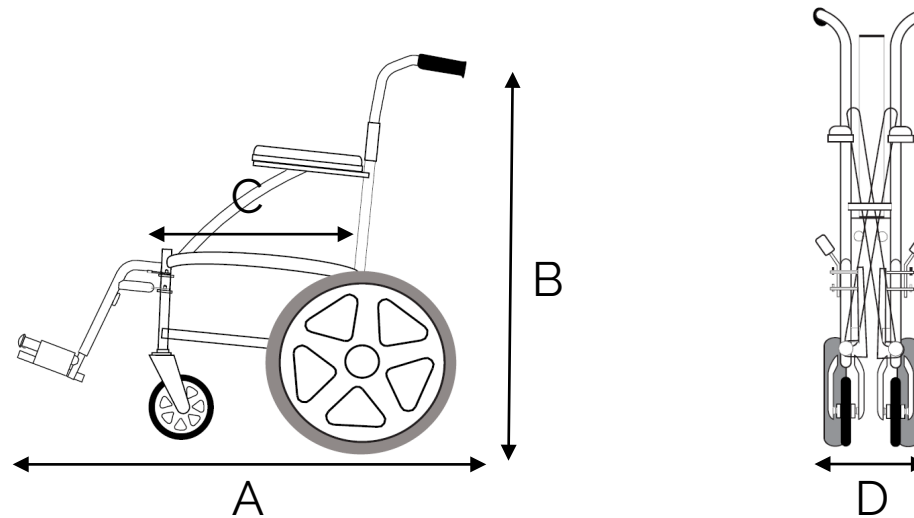
You've chosen HACARE, and we're committed to satisfying your needs.

However, you are responsible for the proper use of the cradles. That's why we invite you to familiarize yourself with the technical specifications, and the instructions for use and maintenance of the equipment by reading through this manual.

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EN - By reading this guide beforehand, you will be familiar with the features and recommendations that are essential for the optimal use and maintenance of your device. Do not use this device until you have read this manual for the first time. Always carry out a general inspection of your device before use. If you are unsure of any aspect of your device, contact your nearest dealer. As an authorised professional, he will be able to advise you and check your chair before and after use.

Any serious incident that has occurred in relation to the device shall be reported to the manufacturer and the competent authority of the Member State in which the user and/or the patient is established.

The warranty excludes damage arising from structural changes to our products, insufficient maintenance, defective or improper handling or storage or the use of non-original parts. Likewise, the warranty excludes parts or working parts subject to natural wear and tear.

1. INTENDED USE, INDICATIONS AND CONTRAINDICATIONS

Our ALIV'UP/ALIV'UP PLUS transfer chair is meant to accompany and move from one place to another people who don't have specific postural problems. This chair is not conceived for very young children or patients whose weight is inferior to 40kg or for other uses than the transport of a person. This chair needs the presence of a third person, responsible and physically able to move the chair. Other contraindications to be determined by the prescribing physician based on the patient's condition.

The ALIV'UP/ALIV'UP PLUS chair is a light chair, manoeuvrable and usable on a plain surface indoors and outdoors. Foldable, its footprint is reduced to be able to be transported and be stored easily.

Only healthcare professionals are entitled to prescribe and

recommend our ALIV'UP/ALIV'UP PLUS chair in order to use it in accordance with patient's needs.

The adaptation and the adjustments of the chair must be done by a certified professional.

In the event of prolonged use, a pressure sore cushion must be used on the chair to prevent the development of pressure sores.

Re-use

In case of a use by different persons, the ALIV'UP/ALIV'UP PLUS chair must be inspected in order to be, if appropriate, maintained and adjusted by a certified professional for the new user. The instruction manual, as well as the documents and original labels must be provided with the chair to the new user.

2. SAFETY INDICATION, ABNORMAL USES, DANGERS

Inspection before use

Before each use, the user or the companion must check that the wheels, their rubber tyre and their bearings are in good condition and that the brakes allow an efficient immobilization of the chair. Never use the chair if one of the clamping element is unscrewed, if a wheel or a part of the chair is suspected of presenting backlashes or if elements from the seat are torn or disjointed. Never use the chair if it is not in good condition.

Indoor use

When moving the chair, one must take into account, in addition to its dimensions, the objects and obstacles in order to avoid to hinder, or hurt the patient or the companion in their movements

Outdoor use

Used outdoors, attention must be paid on meteorological conditions as well as the condition of the floor in order to avoid adhesion risks, slipping and tilting that could prejudice the user and his or her companion. Never use the chair in an escalator.

Transport of the chair

It is not advised to lift the chair with the patient sitting on it. To transport the chair, fold up the chassis and lift it up using stable elements of the structure (armrests, handles, chassis). Don't use the footrests or the wheels to carry this operation.

When transported in a vehicle, it is advised to remove the footrests to reduce the chair's footprint to store it easily in a trunk. The chair must never be substituted to the vehicle's own seats. The patient must be transferred from the chair to the vehicle's seat while having taken care of locking the brakes beforehand.

Risks of pinching

When installing or storing the chair, attention must be paid on avoiding potential pinching linked to adjustment, footrests, disassembly of wheels, folding of the seat. Ask your certified distributor, who will explain how to manoeuvre your chair without danger.

Spare parts

The substitution of defective or altered parts must be made by a professional with the manufacturer's warranted spare parts. Replacement with non-compliant parts can be risky for the patient and can have the manufacturer's warranty cancelled.

Reception and description

Our ALIV'UP/ALIV'UP PLUS are controlled at different steps of their production then packed in a cardboard packaging. Upon receipt, please check that the packaging is not damaged. When unpacking the product, make sure that the instruction manual and the warranty booklet are present. These documents are to be delivered to the user imperatively for him or her to be aware of the use conditions and the warranty of the product. The chair is equipped with :- A seat with a backrest foldable half-height, a canva to sit on it, two armrests fixed laterally and a fastening strap.

- A foldable chassis with two lateral flanks and a cross system that allows to fold and unfold the chair.
- Two multidirectionnal front wheels
- Two back wheels that can be blocked with the brakes located on the sides and/or at the top of the backrest.
- Two adjustable in height and removable footrests fixed on each flank of the chair.

After having checked the conformity of the chair, the certified professional will adjust the footrests according to the user's height, as well as the strap that is used as a support belt. It is the professional's responsibility to inform and to make sure that the user understood well the safety or use instructions and warnings described in this instruction manual.

Environment : Recyclable, the cardboard packaging must be taken to a sorting center.

3. INSTALLATION, ADJUSTMENT

Folding and Unfolding (picture 1)

To fold the transfer chair : stand behind the transfer chair, then lift up

the two handles located on the two sides of the seat (1) and (2).

To unfold the transfer chair: stand behind the transfer chair, then move the two sides of the transfer chair (2) apart and finally press down on each side of the canvas seat (3) so that it is in contact with the 3 supports on each side of the transfer chair. Footrests can be folded on side or not during this operation. Straighten the backrest, locking the upper part with the two brackets (see next step «Folding the backrest»).

Folding the backrest (picture 2)

To fold the backrest, press on the lever (5) located on both sides of the frame and fold the backrest (6).

Installation of footrests

Both footrests are delivered with the paddles mounted. Both footrests can be folded up in the sitting position.

Fitting the footrest (picture 3)

1. Hold the footrests on the sides of the transfer chair frame.
2. Insert parts [A] and [B] of the footrest into brackets [C] and [D] of the frame.
3. Swivel the footrest inwards until it audibly locks into its final position.
4. Return the footrest paddles to the horizontal position.
5. Place the footbelt

Removal of the footrest (picture 4)

1. Push lever [E] to unlock the footrest.
2. Swivel the footrest to the outside of the transfer chair by 90°.
3. Pull the footrest upwards out of the tube to remove the footrest from brackets [C] and [D]

Footrest height adjustment

Caution : Risk of accident if the footrests are set too low.

1. Unscrew the screws [F] at the footrests with the hexagon key provided.
2. Pull the tube downwards and adjust it to the desired height.
3. Tighten the screws again. Make sure that both paddles have the same height adjustment.

Fitting the armrests

The armrests can be tilted backwards.

Replacing the armrests

The armrests are easy to replace, simply unscrew and remove the nut [I], then adjust the new armrest and screw it back on again

Using the brake

The brakes can be operated by means of the wheel lock lever or the parking brake.

Caution: The brakes are only designed to stop the transfer chair and not to brake the transfer chair in motion.

Checks and adjustments must be carried out regularly by an authorised dealer to ensure maximum safety for the user.

When the transfer chair is on a sloping surface, release the brakes one by one.

Parking brakes (picture 5)

CAUTION: Make sure that the transfer chair is on a flat, horizontal surface before releasing the brakes. Never release both brakes simultaneously.

1. To break, push the lever downwards
2. Release the brake by pulling lever up

Foot brake

1. To break, push on the lever downwards (1).
2. Release the brake by pulling the lever upwards (2).

Brake adjustment

The brake pressure on the tyre can be adjusted by unscrewing the fixing screws and sliding the brake on the tube..

Wheels

The rear wheels and the front wheels are delivered mounted. The wheels are fitted with puncture-proof PU tyres. Check that it is not damaged before each use.

4. ADVICES FOR AN APPROPRIATE USE

In case of doubts on its functioning, contact your distributor for advices and rehabilitation if necessary. A summary table at the end of this manual of instruction will sum up the essential information about the chair and its limits of use.

Support belt

This ajustement as well as the use of the belt must be defined by a certified professional. Even if the use of the belt is not necessary, it is

to be noted that it is important to adjust and loop it to avoid that it lays on the ground or that it gets stuck in the spokes of the wheels. For a safe use, it is advised to keep and loop the belt and pay attention on the fact that it is not too tight, in order to avoid discomfort.

Braking

The brakes can be activated by blocking the bracket located on the top.

They can be also activated by disengaging the lever (parking brakes) or by lifting it up. The tension of each brake can be tightened if necessary with the help of the screw located at the back of the chassis.

Locking of the backrest

The backrest is foldable in two for an optimum footprint or the transport of the chair (without the user) In order to undertake this operation, engage on both sides this clips while lifting vertically the superior part. Once locked, it shouldn't be able to fold up.

Footrest ajustements

The footrests are intended to fit perfectly the user's morphology. They are composed of two tubes. The inferior tubes slid and adjust to the desired height thanks to a control knob.

Stability in movement or stopped

Conceived to stay stable when moving as well as stopped, the stability of the chair depends nevertheless on keeping the gravity center of the patient inside a surface that includes the four fulcrums of the wheels

on the floor. The patient as well as the companion must pay attention to respect this rule, including when they want to reach an object. Only the arms can be outstretched beyond the chair. The buttock must always remain on the seat . Never lean on the footrests to get up.

Transfer

Equipped with fixed armrests, the transfer to go out of the chair must be done forwards while priory having blocked the brakes of the back wheels. The footrests must be removed or folded up. Once the feet are on the floor, the hands can lean on the armrests or grab the companion according to the level of mobility of the patient, to get up and reach the desired position. To sit on the chair, please lock the brakes beforehand, remove or fold up the footrests and not hinder the patient's movements. Lean on the armchairs to assist the movement.

In order to sit or go out of the chair, please check that it is placed on a plain and horizontal surface, without obstacles to avoid any risks of falling.

Opening and closing of the chair

To unfold the chair, separate the two flanks of the chair, then push from top to bottom on each side of the canva for it to be in contact with the 3 supports present on each side of the chair. Then, install the footrests on the axis provided for that purpose before locking them on each flank. Straighten out the backrest, while locking the top part thanks to two fixing com- ponents.

To fold up the chair, remove the footrests and lift the seat by pulling it up, and then unlock the backrest to fold up the top part.

5. MAINTENANCE, ADJUSTMENTS AND REGULAR CONTROL

As an essential element of security, brakes must be checked before each use in order to validate their efficiency to keep the chair still. The adjustment is done by screwing/unscrewing the two screws that allow the slide mechanism of the flange along the tube.

The seat and backrest's canvas, once the chair is unfolded, must be tended. Never use the chair if those are disjointed, torn, or are at risk of breaking. The armrests must stay clean and perfectly fixed to the frame. The screws must be correctly tightened in order to use the chair safely, without risks. They must allow the good rotation of the wheels.

The footrest's platforms must not present any cracks nor be broken. They must be able to articulate correctly on their rotation axis. Once installed, the blocking clips on each flank must keep the footrests together with the chassis. The footrests adjust in height thanks to the clamping screw.

Storage and use conditions Temperature

	Temperature		Humidity	
	mini	maxi	mini	maxi
Storage	5° C	+ 50° C	0%	80%
Use	0° C	+ 40° C	0%	80%

Cleaning and disinfection

For a safe use and longer longevity, the ALIV'UP/ALIV'UP PLUS chair must be regularly cleaned up. Remove all traces of dirt on the chassis, the brakes and the rotation axis with the help of a soft cloth, and non corrosive household product (liquid household cleaner, soapy water) before rinsing it. The seat, backrest and armrests as well as the handles situated at the back must be cleaned up too for a better comfort of use.

When the user of the chair changes, it is advised to disinfect more extensively the chair with disinfectant solutions like Surfanios®. Follow the instructions of the product.

6. ANNUAL MAINTENANCE, REPAIRS

For perfect security conditions and longevity of the chair, it is advised to realize to a certificated professional the maintenance of the chair at least once a year, or as soon as you are confronted to difficulties.

Warning : Only original parts of the ALIV'UP/ALIV'UP PLUS chair must be used to fix it. Otherwise the manufacturer's warranty will be lost.

Operations	Frequency	User	Certified distributor
Installation and commissioning	1st use		Certified distributor, installation commissioning, user and companion's training
Replacements of components			To be done by your certified distributor
Control of tyres	Before each use	Mandatory	In case of doubt, ask you certified distributor
Efficiency of the brakes			
Visual inspection, screwing and cleaning	Once a month	Strongly advised	
Maintenance	Annual	Strongly advised	To be done by your certified distributor
Repairs	When necessary		
Supply of spare sparts	When necessary		

Controls and repairs

Every component of the chair must be examined thoroughly in order to function properly in its intended use.

A careful inspection of the chassis must allow to control the absence of cracks, pull-outs, as well as alterations, deformations that could lead to breakage. Warning, the absence of painting can underline the emergence of a default. Special attention should be paid to welds and the cross system with respect to supported efforts.

The brake system must keep the efficiency expected. In case of doubts, it is advised to substitute parts found to be defective.

Attention must be paid on the wheels and the axis rotation, as well as the quality of their rubber tyres, absence of deformation or backlashes on the axes. The wheels must be able to rotate easily, without any particular noise and perpendicularly to the axis on a plain surface, and stop immediately using the brakes or when in contact with the floor. (without pushing it) All the clamping parts must be controlled and replaced if necessary. The screw thread must comply with the norm and not present any traces of wear and tear nor rust.

The footrest's platforms must be changed in case of cracks or malfunction. The tube to adjust the footrests in height must be able to slide without obstacles inside the top tube and must lock sturdily.

The seat, the backrest and the armrests must not present any imperfections that could lead to tearings or breaks. They must be systematically replaced in case of doubts. A complete test of the chair must be done before giving the chair back to the user. The operation must be written down on the maintenance and warranty booklet.

Storage

If you don't need to use your chair no more, proceed to a complete clean up and check if it is in good condition before storing it in a dry and clean room that respect the conditions of temperature and humidity abovementioned. To re-use your chair, it is advised to bring your ALIV'UP/ALIV'UP PLUS to a certified professional to be completely checked up and for the adjustments to be adequate.

End of life of the product

Don't throw your chair nor its accessories away when they are not usable anymore. Dispose of this product in accordance with your regional waste disposal regulations.

Warranty conditions

Our ALIV'UP/ALIV'UP PLUS chair benefit from a warranty against every manufacturing default. In case of repairs or substitution of some parts, warranty will be still be valid on Herdegen's conditionnal approval and as long as original spare parts are used. Warranty don't cover normal wear-and-tear of materials or parts. The manufacturer's warranty excludes every direct or indirect damages.

Our ALIV'UP/ALIV'UP PLUS chair has been validated by in-house testing and meets the requirements established by Ministry of Health and applied by CERAH, which is a certified organism. The warranty booklet provided with each chair must be kept and completed after each operation done on the product.

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