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Assembly instructions

1. Fit the stand tube on the centre of the base. The tube fits very tightly to avoid any play.
2. **Loosely** fit the clamp disk. The cams on the clamp disk must face inside.
3. Assemble the caster legs. **WARNING**: 2 casters of the same type must never be fitted side by side. The casters with the circle are electrically conductive and are used to discharge any electro-static charges.
4. Tighten the clamp disk (to approx. 15 Nm).
5. Stand the stand upright and push on the display head. Make sure it is firmly secured.
6. Adjust the height.
7. Fix the cable with cable clips. **Warning** - Risk of tripping and tilting. The bottom cable clip must be clipped on just above the centre of the base.

Instruction manual for Movable stand for display seca model 472

The seca 472 stand is a movable and height adjustable holding device for display and operating units for seca scales.

Safety instructions:

- Please observe the notes in the operating instructions.
- Only use the stand to hold seca display heads.
- Only use the assembly material supplied.
- When assembling the unit, make sure the screws and the 5 caster legs are firmly secured.
- **Warning - Risk of tripping and tilting.** Make sure that the cable of the display head is firmly secured in the cable holders and the bottom cable holder is clipped on far down, just above the centre of the base of the stand tube. Place the cable to the scale so that there is no risk of tripping. Do not attach any heavy objects to the stand. Secure the stand against accidentally rolling away by operating the locking levers on the casters.