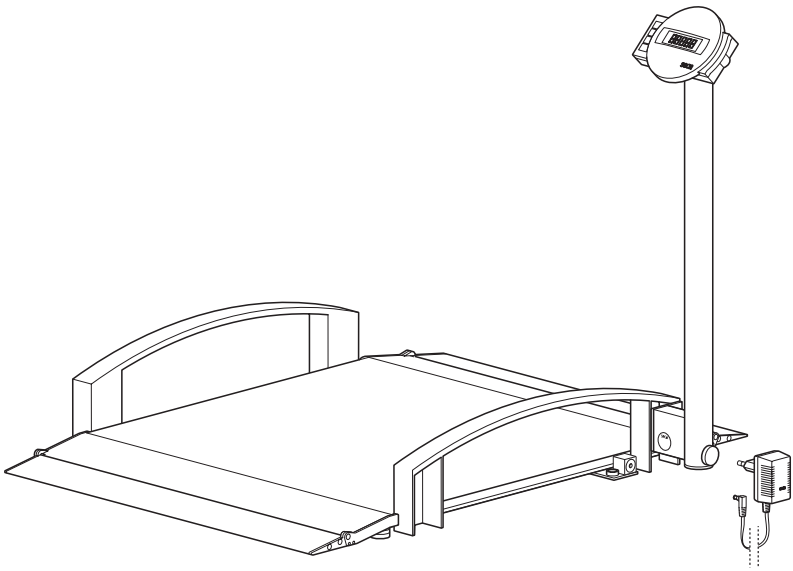
# seca 665

<b>D</b>	<b><i>Bedienungsanleitung und Garantieerklärung</i></b>	<b>3</b>
<b>GB</b>	<b><i>Instruction manual and guarantee</i></b>	<b>15</b>
<b>F</b>	<b><i>Mode d'emploi et garantie</i></b>	<b>27</b>
<b>I</b>	<b><i>Manuale di istruzioni e garanzia</i></b>	<b>39</b>
<b>E</b>	<b><i>Manual de instrucciones y garantía</i></b>	<b>51</b>
<b>DK</b>	<b><i>Betjeningsvejledning og garantibevis</i></b>	<b>63</b>
<b>S</b>	<b><i>Bruksanvisning och garanti</i></b>	<b>75</b>
<b>N</b>	<b><i>Bruksanvisning og garantierklæring</i></b>	<b>87</b>
<b>FIN</b>	<b><i>Käyttöohje ja takuu</i></b>	<b>99</b>
<b>NL</b>	<b><i>Bedieningshandleiding en garantieverklaring</i></b>	<b>111</b>
<b>P</b>	<b><i>Instruções de utilização e declaração de garantia</i></b>	<b>123</b>
<b>GR</b>	<b><i>Οδηγίες χειρισμού και εγγύηση</i></b>	<b>135</b>

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# Full certification

---

In seca products you are purchasing not only technology matured over a century, but also quality confirmed by authorities, the legal system and by institutes.

seca products already meet the provisional legal deadlines of tomorrow. With seca, you are buying the future.

**M**

The products in these operating instructions comply with German legislation governing medical products and therefore with Directive 93/42/EEC of the Council of the European Community which is embodied in national laws throughout Europe.

Scales carrying this symbol are approved for medical purposes in the European Community. seca scales carrying this symbol already meet the quality and technical requirements of the year 2003.

Scales carrying this symbol are calibrated in accordance with Precision Class III of EC Directive 90/384/EEC.

Products carrying this symbol meet the requirements

1. of the Directive for non-automatic scales in medicine, No. 90/384/EEC
2. for the law governing medical products, EC Directive No. 93/42/EEC
3. the Directive on electromagnetic compatibility

seca has been awarded the EN ISO 9001 certificate, which means that our company adheres to strict specifications for consistent high quality in research, production, sales and service. Furthermore, EN 46001 certifies seca's rigorous compliance with the most stringent European quality standards for medical products.

seca helps the environment.

We are concerned to save natural resources. We therefore strive to save packaging material where meaningful. What is left over can conveniently be disposed of locally via the German Dual System (recycling programme).

# 1. Congratulations!

---

In the **seca 665** electronic wheelchair scale, you have purchased a high-precision yet sturdy piece of equipment.

seca has been putting its experience at the service of health for over 150 years now, and as market leader in many countries of the world, is always setting new standards with its innovative developments for weighing and measuring.

In accordance with national regulations, the **seca 665** electronic wheelchair scale is used primarily in hospitals, doctors' surgeries and in-patient nursing facilities. The scale is approved to CE class III.

Weight display can be switched between kilogrammes (kg) and pounds (lbs). Weight is determined within a few seconds.

The pre-Tare function allows you to store the weights of three types of wheelchair. This weight is automatically deducted in the weighing operation.

The two fold-away ramps allow the wheelchair-user to get on and off in comfort.

The fold-away column allows the scale to be stored upright to save space.

The scale is of a very sturdy construction and will give you long and loyal service. It is easy to operate and the large display is easy to read.

The **seca 665** can be moved on castors and its low power consumption combined with its rechargeable batteries mean it will provide long, mobile service.

## 2. Safety

---

Before using the new scale, please take a little time to read the following safety instructions.

- Follow the instructions in the instructions for use.
- Keep the operating instructions and the declaration of conformity in a safe place.
- Do not drop the scale or subject it to violent shocks.
- Lay the scale carefully in its position for use.
- Secure the scale so that it does not fall over when stored in the upright position.
- When moving the scale in the upright position, ensure that it does not fall over.
- Be sure to use only the seca mains unit supplied. Before plugging the mains unit into a socket, check that the mains voltage indicated on the mains unit matches local mains voltage.
- Have scale serviced and re-calibrated on a regular basis (see chapter entitled "Maintenance" on page 25).
- Have repairs carried out only by authorised persons.

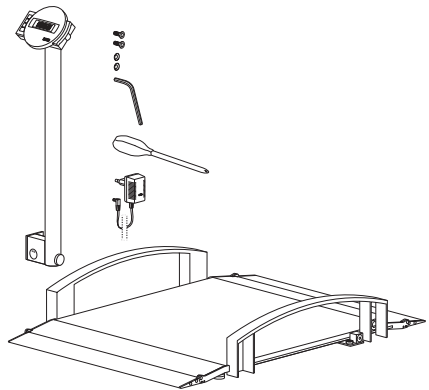
# 3. Before you start...

## Unpacking

- Remove the packaging and place the scale on a firm, level surface.

The scope of supply includes:


- Weighing platform
- Column with display head
- 2 mounting screws
- 2 serrated lock washers
- 1 Allen key
- 1 screwdriver
- 1 seca mains unit
- 1 set of operating instructions

## Fitting the column


The column and display head are fitted to the weighing platform.

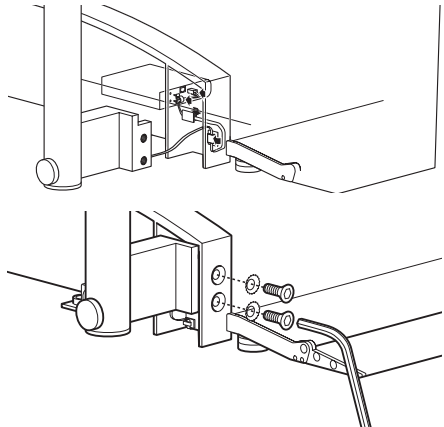- Guide the connecting cable through the opening.
- Place the column at the intended point at the end of the rail and tighten up the screws.
- Now connect the battery pack.

## Connecting battery pack

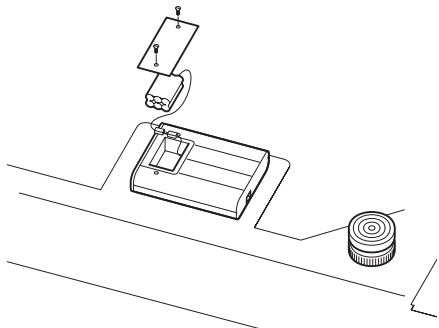- Carefully get the scale upright so that the base of the scale is accessible.
- Connect the connecting cable from the column to the electronics box.
- Undo the screws of the battery compartment and take off the cover.
- Connect the plug contacts to the battery pack inside.
- Close the cover and then turn the scale back up the right way.


## Power supply

The scale is powered by a mains unit and by rechargeable batteries. The scale works independently of the mains, which is very handy for mobile use.

The LC display and the evaluation electronics have a low power consumption – 2,000 weighing operations can be performed with a fully charged battery. Once battery voltage is no longer adequate, the battery symbol  first appears in the display to indicate that battery power is dropping. If the batteries are not charged, battery voltage will ultimately not be adequate for error-free measuring. In this case, **bAtE** appears in the display and weighing is no longer possible.

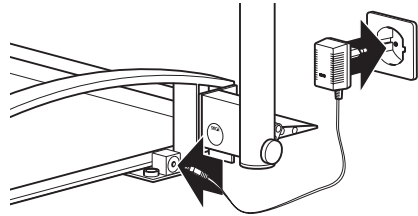
The rechargeable batteries are charged by the mains unit supplied. For this purpose, the mains unit needs to be connected to the mains for at least 24 hours.

## 4. Setting up and aligning the scale

---

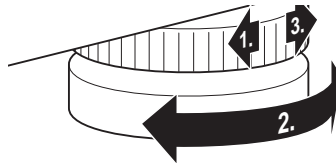- Place the scale on a level surface.
- Fold the column upwards. Secure the column by tightening up the knurled screw.
- Undo the locknuts using the four foot screws.
- Align the scale by undoing the foot screws. The air bubble in the spirit level must be right in the centre of the circle.
- Then re-tighten the locknuts.
- Fold down the ramps required.
- Ensure that only the feet of the scale are in contact with the floor. The scale may not be in contact at any other point.

spirit level

### Important!

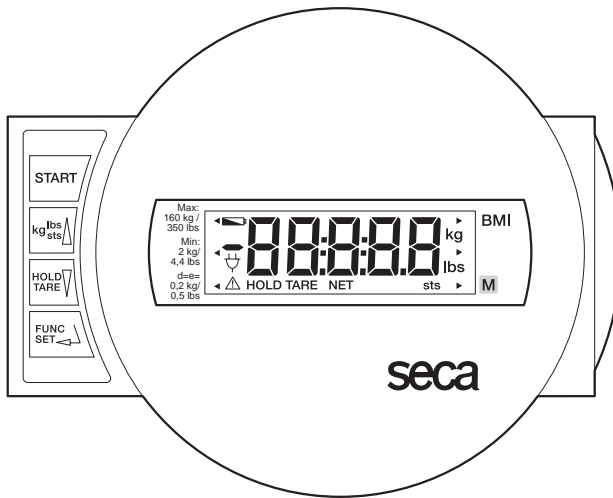
**The alignment of the base of the scale must be checked and corrected if necessary every time the location of the scale is changed.**

# 5. Operation

---

## Controls and displays

The controls and the display are on the display head on the column.

Switch scale on and off

press briefly → switch between kg and lbs  
in function menu → increase value, scroll

press briefly → activate hold function  
hold down → set display to zero, activate/deactivate tare function  
in function menu → decrease value, scroll

Activating the pre-Tare function and Enter key for input data

## Checking that the contents of the calibration counter are correct

This seca scale is calibrated. Calibration may be performed only by authorised centres. To ensure this, the scale is fitted with a calibration counter which records any change to data relevant to calibration. If you want to check whether the scale is properly calibrated, proceed as follows:

- If necessary, switch off the scale.
- Keep any key depressed and start the scale.

The current contents of the calibration counter are shown on the display for a few seconds.

- Compare the output contents of the calibration counter with the number shown on the calibration counter sticker.

For valid calibration, both numbers must match. If sticker and calibration counter do not match, then re-calibration is required. Contact your service agent or seca Customer Service.

START

2

**Both numbers must match.**

## Correct weighing

- Press the Start key.  
**SECA**, **888888** and **0.00** appear consecutively in the display. The scale is then automatically set to zero and ready to use
- If necessary, switch the weight display (see page 21).
- Move your chair onto the scale and keep still.
- Read the weighing result off the digital display.
- If a load greater than 300 kg or 660 lbs is placed on the scale, **STOP** appears in the display.
- To switch off the scale, press the Start key again.

START

STOP

START

## Switching the weight display

The weight display can be switched between kilogrammes (kg) and pounds (lbs).

- Switch on the scale using the green Start key. Weight is initially displayed in the last setting selected.

- To switch the weight display between kilogrammes and pounds, press the **kg/lbs/sts** key.

The **kg** or **lbs** display lights up.

## Automatic switch-off/economy mode

The scale automatically detects whether it is being operated on rechargeable batteries alone or whether it is connected to the mains.

- With rechargeable battery operation: automatic shut-down after 5 minutes.
- With mains operation: shut-down only via keypad.

## Taring weight display (TARE)

The Tare function is very useful if an additional weight, e.g. that of a wheelchair, is to be ignored for weighing purposes.

Proceed as follows:

- Press the green Start key.

- First place the additional weight (e.g. the wheelchair) on the scale and activate the Tare function by pressing and holding the **HOLD/TARE** key.

The display is reset to zero. **0.0** appears in the display. The note "NET" appears. You can now perform as many weighing operations as you like.

- Now move the patient and the wheelchair previously weighed onto the scale.

The scale determines the patient's weight. The Tare function can be re-activated at any time with an altered weight or without a weight.

- Put the new weight on the scale and depress and hold the **HOLD/TARE** key.
- The Tare weight is erased when the scale is turned off.

## Store weight value (HOLD)

You can also continue displaying the weight determined once the load has been removed. This makes it possible to look after a patient first, before having to note the weight.

- Press the **HOLD/TARE** key whilst the load is on the scale.

“HOLD” appears in the digital display. The value is stored until the scale switches off automatically.

The triangle with the exclamation mark indicates that this frozen display is an additional facility which is not subject to calibration.

- The HOLD function can be switched off by pressing the **HOLD** key again.

0.00 appears in the display if the scale has no load on it, otherwise the current weight. The “HOLD” display vanishes.

### Note:

**The HOLD and TARE functions can also be used simultaneously.**

## Weighing with pre-Tare

If the pre-Tare function is activated, a stored weight value is deducted from the currently measured weight. This function is useful, for example, if the weight of a wheelchair is not to be displayed.

Three different tare weights (wheelchair weights) can be stored.

- Press the Start key.  
**SECR**, **888888** and **0.00** appear consecutively in the display. After that, the scale is automatically set to zero and ready for operation.

- Press the **FUNC** key.  
The last function used appears in the display.

An arrow on the left-hand side of the display indicates which memory is set.

- Use the arrow keys to select the desired memory for storing the additional weight or where a weight may already be stored.
- Then press **FUNC**. In the display you will see the tare value from the memory.

- You can alter the value in 100 g increments using the arrow keys.
- Once the correct value is set, press the **FUNC** key again.  
The pre-Tare function is now activated.
- Move onto the scale in the wheelchair associated with that memory. Read the weighing result off the digital display.
- Press an arrow key if you would like to see the current weight value briefly.
- To return to normal weighing mode, press the **FUNC** key.
- To switch off the scale, press the Start key again.

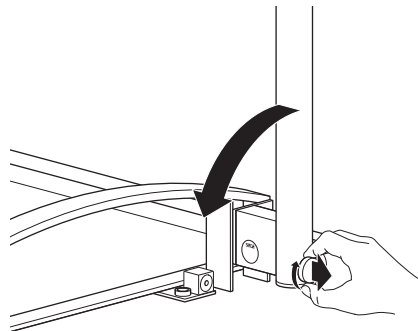
**Tip: note weight and the memory in which the weight is stored on your wheelchairs.**

## 6. Storing the scale

---

The scale can be stored upright to save space.

- Undo the knurled nut on the hinge of the column.
- Pull out the knurled screw and fold over the column.
- Re-engage the knurled screw.
- You can now prop the scale up on the column.
- The scale can now be moved.

The scale can be placed on the bottom rail for **short-term** storage. Secure the scale so that it does not fall over when stored in the upright position.

## 7. Cleaning

---

Clean the rubber surface and the housing as required using a domestic cleaning agent or commercially-available disinfectant. Follow the manufacturer's instructions.

Under no circumstances use abrasive or acid cleaners, white spirit, benzene or the like for cleaning. Such substances can damage the high-quality surfaces.

## 8. What do I do if...

---

### ... no weight display comes on?

- Is the scale switched on?
- Check the mains connection and the connection of the battery pack
- Is the plug contact on the scale correctly connected?
- Is the plug contact for connecting the display to the scale correctly connected?

### ... 0.0 does not appear before the weighing operation?

- Press the Start key again – there must not be any load on the scale – and only its feet should be in contact with the floor.

### ... one segment is illuminated constantly or not at all?

- The relevant segment has a fault. Inform Servicing.

### ... the display shows ?

- Battery voltage is dropping. Charge the rechargeable batteries in the next few days.

### ... bAtE appears in the display?

- The rechargeable battery is so severely discharged that weighing is no longer possible. Recharge the battery.

### ... 5tOP appears in the display?

- Maximum load has been exceeded.

### ... the display flashes?

- If you have just activated the Hold or Tare function, please wait until the scale has detected a steady weight value. The display will then stop flashing.
- If there has been no activation of a function beforehand, remove the load from the scale and wait until 0.0 is displayed, then weigh again.

### ... the display tENP appears?

- The ambient temperature of the scale is too high or too low. Place the scale in an ambient temperature between +10 °C and +40 °C. Wait about 15 minutes for the scale to adapt to the ambient temperature and then weigh again.

### ... the display E and a number appear?

- Switch off the scale with the Start key and start the scale again. The scale will then work normally again.  
If this is not the case, interrupt the power supply by briefly removing the rechargeable battery. If this measure is equally unsuccessful, inform the Service department.

## 9. Maintenance

---

Have recalibration performed in accordance with national regulations. Servicing and recalibration may be performed solely by authorised staff. Contact your service agent or seca Customer Service.

## 10. Technical data

---

### Dimensions: (width, length, height)

Scale in position for use, ramps folded out:	1102 x 1150 x 911 mm
Scale in position for use, ramps folded away:	1102 x 985 x 911 mm
Scale in upright position:	193 x 986 x 1102 mm

### Weight

approx. 30,3 kg

### Temperature range

+10 °C to +40 °C

### Calibration

approved for medical use, Class III

### Medical product as per Directive 93/42/EWG

Class I

### Colour

silver grey/black

### Maximum load

300 kg /660 lbs

### Minimum load

4 kg / 8,8 lbs

### Graduations

200 g / 0,5 lbs

### Accuracy at first calibration

0 to 100 kg:	± 100 g
100 kg to 300 kg:	± 200 g

### Power supply:

mains unit and built-in rechargeable battery pack

## 11. Disposal

---

### Batteries

Do not throw used batteries away in domestic waste. Dispose of batteries at collection points in the vicinity. When buying new batteries, select those low in harmful substances and containing no mercury (Hg), cadmium (Cd) or lead (Pb).

### Disposing of the scale

If the scale can no longer be used, the responsible waste disposal association will be glad to advise you on the measures necessary to dispose of it properly.

# 12. Warranty

---

A two-year warranty from date of delivery applies to defects attributable to poor materials or workmanship. All moveable parts - batteries, cables, mains units, rechargeable batteries etc. - are excluded. Defects which come under warranty will be made good for the customer at no charge on production of the receipt. No further claims can be entertained. The costs of transport in both directions will be borne by the customer should the equipment be located anywhere other than the customer's premises. In the event of transport damage, claims under warranty can be

honoured only if the complete original packaging was used for any transport and the scale secured and attached in that packaging just as it was when originally packed. All the packaging should therefore be retained.

A claim under warranty will not be honoured if the equipment is opened by persons not expressly authorised by seca to do so.

We would ask our customers abroad to contact their local sales agent in the event of a warranty matter.

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seca Vogel & Halke GmbH & Co.