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## EVAQ®-PRO

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# operation manual

## EVAQ®-PRO

Pressure ulcer therapy mattress with evacuation function

Please read the operating instructions carefully before using the system and heed the **warnings!**

<b>Product name</b>	<b>EVAQ®-PRO</b>
<b>scope of delivery</b>	Pressure ulcer therapy mattress with evacuation function, foam core and PU cover (removable with a zip), operating instructions
<b>Indications / areas of application</b>	<ul style="list-style-type: none"> <li>• Pressure ulcer prophylaxis</li> <li>• Pressure ulcer therapy up to grade 3 (Seiler / EPUAP)</li> <li>• Pain therapy</li> <li>• For patients with medium risk of pressure ulcers</li> </ul>
<b>Contraindications</b>	Not known
<b>Permitted patient weight min. - Max.</b>	0-140 kg
<b>material</b>	<ul style="list-style-type: none"> <li>• Core: PU foams</li> <li>• Cover: PU-coated PES carrier material, nylon</li> </ul>
<b>Dimensions</b>	Width: 80-100 cm Length: 190-220 cm
<b>Product weight</b>	11 kg (90/200 cm)
<b>Aid number (as defined in §139 SGB V)</b>	
<b>Medical device</b>	Class 1, MDR 2017/745  
<b>Technical safety / flame retardancy</b>	Flame retardancy test according to DIN EN 597 I + II, BS 7175 sec 3 crib 5, BS 7177
<b>Testing for toxicological harmlessness / biocompatibility</b>	DIN EN ISO 10993-5, DIN EN ISO 10993-1, DIN EN ISO 10993-10
<b>Warnings</b>	<ul style="list-style-type: none"> <li>• Before the first application, the system must be prepared for the first use (see "Preparing the system for use").</li> <li>• Suitable for adjustable lying surfaces</li> <li>• Side rails at least 22 cm above the top of the mattress</li> </ul>
<b>guarantee</b>	3 years on the functional properties of the foam core (pro-rata guarantee)
<b>maintenance</b>	No
<b>Cleaning instructions (mattress cover)</b>	<p>The cover can be disinfected by wiping with commercially available disinfectants made from isopropanol (e.g. Bacillol) or with other suitable DGHM-listed products (e.g. ECOLAB cleaning cloth Incidin OxyWipe S XL). Chemothermal cleaning of the cover is possible. The cover can be reprocessed by autoclaving according to the guidelines of the RKI.</p> 
<b>Cleaning instructions (core)</b>	<p>The foam core without cover can be cleaned with household means. This can be achieved by soaking the contaminated area or spraying the entire foam core with a disinfectant (eg Bacillol or ECOLAB foam spray Incidin OxyFoam S consisting of hydrogen peroxide). Then the foam core must be air-dried. The foam core can be reprocessed by autoclaving according to the guidelines of the RKI.</p>
<b>Prepare the system for use</b>	<ol style="list-style-type: none"> <li>1. Remove the mattress from the box immediately upon receipt.</li> <li>2. Put the mattress in the bed.</li> <li>3. Note the print on the top of the mattress "THIS SIDE UP".</li> <li>4. The straps and handles are only intended for evacuation.</li> <li>5. Please watch the training video at <a href="http://www.funke-medical.de/de/downloads/#filme">www.funke-medical.de/de/downloads/#filme</a>.</li> </ol>

RKI = Robert Koch Institute

DHGM = German Society for Hygiene and Microbiology

MPG = Medical Devices Act

Autoclaving = fractional vacuum process at 105 degrees C and 55 minutes

 **Manufacturer in the sense of the MPG and MDR 2017/745**

Funke Medical AG • Ährenfeld 10 • 46348 Raesfeld • Tel. : + 49-2865-266 93 50 • Fax: + 49-2865-8133

info@funke-medical.de • www.funke-medical.de

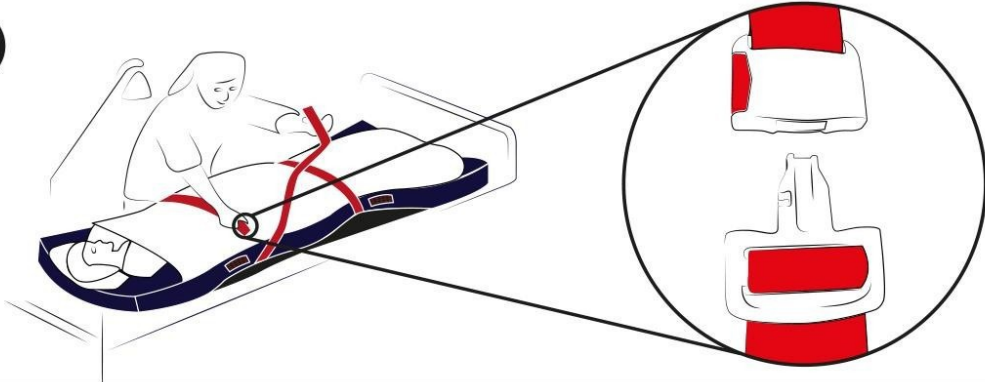
Download: 02.07.2020 12:30


1

2

3

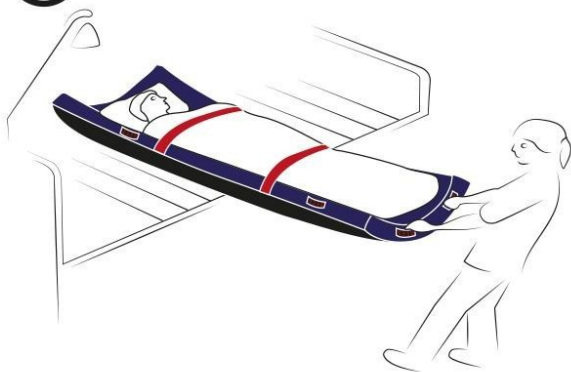4

5

Funke Medical AG  
Ährenfeld 10  
46348 Raesfeld  
Tel.: +49-2865-266 93 50  
Fax: +49-2865-81 33  
info@funke-medical.de  
www.funke-medical.de  
-GERMANY-

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Funke Medical AG • Ährenfeld 10 • 46348 Raesfeld • Tel.: +49-2865-266 93 50 • Fax: +49-2865-8133