CE 2797
Lieferant/distributor:

Hersteller/manufacturer:

tic Medizintechnik GmbH & Co. KG

Endelner Feld 9 · D - 46286 Dorsten
Fon: +49 (0) 23 69 . 20 85 - 0
Fax: +49 (0) 23 69 . 20 85 - 10
eMail: info@ticmed.de
Web: www.ticmed.de
Shop: www.tic-direkt.de

05/2019/R5TIC

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Contents

Technical Data	2
Warning and safety information	2
Pictograms	3
General Information	4
Definition of terms	5
Purpose	5
Mode of operation PRIMUS _{med}	5
Liability and Warranty	6
Contraindications	6
Device Description PRIMUS _{med}	7
Configuration of Device	7
Labelling	7
Scope of Delivery	8
Application of PRIMUS _{med}	9
Preparation of the vacuum therapy	9
Application of the erection ring to the plastic cylinder	9
Performing the vacuum therapy	9
Removal of the erection ring after application	10
Cleaning of the vacuum erection aid system	11
Advice and Hints	11
Vacuum erection versus natural erection	11
Try it out instead of going straight for it	12
Reduction ring	12
Erection ring	12
Disposal	14
Environmentally friendly disposal	14
erection rings (Germany)	14

Technical Data

Model	PRIMUS _{med}
Type	Vacuum Erection Aid System
MPA class	Ila
Dimensions	63 mm x 63 mm x 330 mm
Weight	Including battery approx. 420 g
Power supply	3 x 1,5 V AA (Mignon) Alkaline battery
Classification	B 
IP Protection class	IP 30, protected against access with e.g. tools >= 2,5mm Ø
Operating conditions	Ambient temperature: +5°C to +40°C; relative humidity from 15% to 90%, non-condensing, but without necessary water vapor partial pressure of more than 50hPa; air pressure from 700hPa to 1060hPa
Transport and storage conditions	Temperature range: -25°C to +5°C; relative humidity up to 90% with +5°C to 35°C (without condensation); >35°C to 70°C with water vapor pressure up to 50hPa
Expected durability (system)	15 years
Operating period/quantity of applications in use (supplied batteries)	30 applications lasting 20 minutes

Time for cooling down or warming up due to inconvenient storage temperature until operable at 20°C ambient temperature

from -25°C to +5°C warming up	2 hours
from +70°C to +40°C cooling down	2 hours

Warning and safety information

Allergic reactions to touchable materials are possible (for material data please refer to the listing of scope of delivery in the device description on page 8 of this manual)

Unauthorized changes and damages of the system cannot secure the success of the therapy. Also, unauthorized changes of the system can lead to injuries or damage of property.

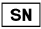
In case of the following circumstances, please contact the producer or the supplier:

You have the feeling that the performance of the system changed although you did not make any changes on the system and the batteries are completely charged.

Pictograms

E

What do the symbols mean?

	Please read the instruction manual carefully before using the system for the first time and follow the included instructions.
	Describes a possibly dangerous situation. If this is not avoided, injuries can result.
	Indicates a possibly harmful situation. If this is not avoided, damage to property can result.
	The user information provides important advice for easy or intended application.
	Proof of conformity to the Medical Devices Council Directive 93/42/EEC of 14th June 1993. It is a Class I medical device or a Class IIa medical device if a Notified Body (4-digit numeric code) is indicated.
	Proof of conformity to the Medical Devices Council Directive 93/42/EEC of 14th June 1993. It is
	Name and address of the manufacturer
	Year of manufacture
	This product is marked with the recycling symbol according to the EEC Directive 2002/96/EC. It is not permitted to dispose of it with the household/domestic waste. The responsible contact for the disposal is the point of sales/manufacturer.
	Applied part type B that in normal use necessarily comes into physical contact with the patient
	Serial number of the device
	Batch code
	Item number of the manufacturer
	EXP

General Information

Dear customer,

With the PRIMUS_{med} vacuum erection aid system you have chosen a particularly user-friendly product for the treatment of erectile dysfunction or functional erection disorders. The user manual will help you to optimally use the vacuum erection aid system. PRIMUS_{med} offers you the following benefits:

- the alternative or supplement to treatment with drugs
- simple, safe and with few side effects
- the application can be repeated as often as required
- ergonomic design of the device

Please read the instruction manual carefully before using the system. Strict compliance is essential for the intended use, the correct implementation of the device and the user safety dependent on it.

The PRIMUS_{med} vacuum erection aid system is a single-patient-use device!

This manual is part of the device and must be stored together with it, to allow people responsible for the application of the device access it at any time.

Definition of terms

Erectile dysfunction (impotentia coeundi) - commonly known as functional erectile disorder, erection problems, impotence - is a sexual disorder where a man over a certain period of time does not succeed in the majority of attempts to achieve or sustain a suitable erection (swelling and hardening of the penis), sufficient for a satisfying sex life. Short-term erectile problems are not considered to be erectile dysfunction.

Purpose

The PRIMUS_{med} vacuum erection aid system is intended for the treatment of erectile dysfunction resulting from:

- deficiencies in the arterial supply in the area of man's sexual organ
- disorders of the venous / cavernous vascular system (venous leak) when erection rings alone are not sufficiently effective
- damage to the peripheral nervous system resulting in the disorder

Mode of operation PRIMUS_{med}

The electric motor in the pump head reduces the atmospheric pressure in the plastic cylinder and thereby supports the natural flow of blood into the cavernous body of the penis - an erection results. To avoid a premature return blood flow and thereby to maintain the erection, in most cases an erection ring is used, which is moved from the plastic cylinder to the penis.

General Information

The vacuum therapy has become more and more accepted as the most acknowledged treatment method for erectile dysfunction. In addition to the unproblematic handling, the very high success rate speaks above all for this as an additional form of therapy, without side-effects.

E

Liability and Warranty

The PRIMUS_{med} vacuum erection aid system is warranted for a period of 24 months provided it is handled properly, this excludes the erection rings and lubricant. During this time, the device will be repaired or replaced by the manufacturer/supplier free of charge.

Incorrect use, non-compliance with the instructions manual and unauthorised changes to the system exclude the manufacturer's/supplier's liability for any resulting damage of property and personal injury and will void the warranty.

PLEASE NOTE: Only accessories authorised by the manufacturer may be used!

The manufacturer reserves the right to make technical changes that serve to improve the device, the functionality and the accessories.

The manufacturer expressly rules out regular safety checks.

Contraindications

The following persons may use the PRIMUS_{med} system only after consultation with their doctor:

- patients with limited or no pain perception, for instance in paraplegia or poly-neuropathy
- patients suffering from blood clotting disorders or those taking anti-coagulants (anti-clotting drugs).

General Information

The PRIMUS_{med} system must not be used:

- by patients suffering from priapism
- in blood diseases with formation of thrombi in the capillary system
- when suffering from all forms of sexually transmitted diseases
- after excessive consumption of alcohol or under the influence of drugs, as this will reduce the pain perception greatly

E

Device Description PRIMUS_{med}

Configuration of Device

(see Fig.: 1)

a	Pump head
b	Push button to create the vacuum
c	Push button to release the vacuum
d	Plastic cylinder
e	Reduction ring large
f	Reduction ring small
g	Slip-on cone (for the erection ring)

Labelling

<p>PRIMUSmed REF ti1091</p>  <p>SN 1 0 9 1 5 0 0 0 0 1</p> <p>LOT A-0000 PZN 9124643</p>     2019 <p>Medizinprodukt  2797</p>	 <p>SN 1091500001</p> <p>2019  tic Medizintechnik GmbH & Co. KG Endelner Feld 9 · D-46286 Dorsten</p> <p>PRIMUSmed    2797</p>
---	---

Label packaging

Label pump head

Device Description PRIMUS_{med}

Scope of Delivery

Art.-No.	Description	Material	PZN
ti1091	PRIMUS _{med} system: - Pump head - Plastic cylinder - Slip-on cone	ABS ASB ABS	9124643
ti1092	PRIMUS _{med} erection ring small, d = 10 mm	Elastomer	8855310
ti1093	PRIMUS _{med} erection ring medium, d = 12.5 mm	Elastomer	8855422
ti1094	PRIMUS _{med} erection ring large, d = 15 mm	Elastomer	8855439
ti1096	PRIMUS _{med} reduction ring small	Silicone	8855273
ti1097	PRIMUS _{med} reduction ring large	Silicone	8855304
ti1098	Lubricant 150 ml for PRIMUS _{med} system	see tube	9124666
-	PRIMUS _{med} user manual	-	-
-	PRIMUS _{med} application video	-	-
-	PRIMUS _{med} wash bag	Nylon	-
ti6063	1.5 V AA alkaline batteries (set of 3)	-	-

All items marked with a PZN can, in Germany, also be bought in a pharmacy. Please indicate the PZN in the pharmacy.

In our online shop www.tic-direkt.de you will find besides the listed articles and other medical devices, e.g. amongst others also an extra-large erection ring (d = 20 mm).

Preparation of the vacuum therapy

1. Assemble the vacuum erection device system as shown in Fig. 3.
2. Strong pubic hair in the application area should be removed before treatment. Alternatively, you can apply some of the included lubricant onto the root of the penis.
3. Ensure the battery compartment of the pump head is fitted with three 1.5 V AA alkaline batteries. Fig. 10 shows the correct arrangement of the poles (+/-).

Application of the erection ring to the plastic cylinder

1. Place some lubricant onto the slip-on cone.
2. Stretch the erection ring you have selected by pulling on the straps provided.
3. Slip the erection ring by about 2 cm onto the plastic cylinder, using the slip-on cone (Fig. 4).
4. Remove the slip-on cone and place a reduction ring into the plastic cylinder (Fig. 5).
5. Place lubricant onto the inner and outer rim of the reduction ring (Fig. 6).

Performing the vacuum therapy

1. Insert your penis into the plastic cylinder.
2. Press the vacuum erection device PRIMUS_{system} med against your body (Fig. 7).
3. Operate the vacuum pump as follows:
Push the button to create the vacuum for about 4 - 5 seconds. Then wait for about 10 seconds until the erection has established. Should the resultant erection not be sufficient, push the button again for about 2 - 3 seconds to recreate a vacuum. If the erection after a further pause of 10 seconds is still not satisfactory, repeat this process

Application of PRIMUS_{med}

(recreate vacuum for 3 seconds / pause 10 seconds) until you have reached an appropriate erection (Fig. 8).

This very careful and progressive handling is only required for the first few applications, for as long as you need to assess how long you have to operate the vacuum pump to achieve a satisfactory erection.

E

 CAUTION	If you encounter pain during or after use of the device, despite having used it correctly, stop the treatment and discuss this with your treating physician!
 CAUTION	Should the vacuum in the plastic cylinder prove to be too high and thus cause pain, release the vacuum by pressing the push button!

4. When you have a satisfactory erection, transfer the erection ring from the plastic cylinder to the root of the penis. You should ensure that the tabs of the erection ring perpendicular to the penis.
5. Press the button to release the vacuum and remove the vacuum erection aid system (Fig. 9).

Removal of the erection ring after application

1. Stretch the erection ring by pulling on the straps provided.
2. Wait a few seconds, until the erection is lost.
3. Pull the erection ring with a uniform movement off the penis.

Application of PRIMUS_{med}

Cleaning of the vacuum erection aid system

CAUTION

The pump head is excluded from the cleaning process described below, and should only be cleaned with a damp cloth. The penetration of water into the pump head will render this inoperative!

E

All used components of the PRIMUS_{med} system (with the exception of the pump head) should be cleaned after use in warm pH-neutral soapy water. The soap residues must be completely removed under running water.

Advice and Hints

Vacuum erection versus natural erection

1. Normally, the vacuum erection is slightly larger than the natural erection.
2. The erection ring maintains an erection past the ejaculation, so that sexual intercourse may be extended if applicable.
3. Since the erection can be maintained by the erection ring only outside of the body, during sexual intercourse a kind of "joint" effect can occur, i.e. the penis bends at the root of the penis. Most men however do not see this as a problem.

Try it out instead of going straight for it

1. The first trials using the PRIMUS_{med} system should be carried out without the intention of subsequent sexual intercourse.
2. Take your time when choosing the erection ring most suitable for you.
3. The best results are achieved with a relatively low vacuum. Take time to find the appropriate strength for you. Take care that the erection builds up slowly.

Reduction ring

1. Ensure that the vacuum remains stable during the pumping process. No air leaks should occur between your body and the reducer if possible. Your penis should become erect within 10 to 90 seconds. The reducer closes at the beginning of the application with the surrounding pubic area, with increasing erection of the penis with its shaft.
2. The small reducer ring should only be used additionally, if you notice during use that the diameter of the large reduction ring is too big for you.

Erection ring

One of the most important conditions for the proper functioning of the PRIMUS_{med} system is the precise location of the erection ring. This is made from a very elastic silicone compound and should sit tightly around the penis. This is necessary for the accumulation of blood in the penis. However, one has to ensure that the erection ring is under no circumstances tighter than necessary.

Advice and Hints

Therefore, please test the largest erection ring first. Only when you have achieved a good erection with the PRIMUS_{med} system, but this - despite erection ring - subsides within five minutes or less, should you try the next smaller erection ring.

 CAUTION	Do not leave the erection ring in place for longer than 30 minutes! If the erection ring remains on the penis over a longer period of time, this can cause a harmful interruption of blood supply and there is the risk of tissue damage!
 CAUTION	Do not run the risk of falling asleep with the erection ring in place and do not use it after consumption of alcohol! Only wear the erection ring during sexual intercourse and only a few minutes before and after.

If you wear the erection ring, ejaculation may not be as strong as usual. To reduce the pressure on the penis, the erection ring can be shifted slightly forward or removed just before ejaculation. The erection is maintained even after ejaculation, until the erection ring is removed.

	To ensure the erection ring retains its elasticity over a longer period of time, you should clean it after use, wrap it without air in the plastic cover and seal it securely. Then store the erection ring according to storage conditions on the label of the erection ring.
--	--

Error and troubleshooting

The PRIMUSmed is a certified and approved medical device. Therefore, please report all serious incidents occurring in connection with the product to the manufacturer and the competent state authority (in Germany, the competent authority is the Federal Institute for Drugs and Medical Devices (BfArM)).

If the solutions proposed do not solve the problem, please contact customer service or your dealer..

– **The unit does not turn on when you press the power button.**

- Make sure that the batteries are inserted correctly and in contact.
- If necessary, replace batteries.

Disposal

Environmentally friendly disposal

The national legal regulations in force in your country for the environmentally sound disposal of accessories, batteries and packaging must be observed..

This product must be disposed of separately from household waste in accordance with applicable legislation. Hand it over at an official collection point.

Separate collection and recycling will conserve natural resources and ensure that all health and environmental protection is respected when recycling the product.

erection rings (Germany)

In Germany, all erection rings listed in the delivery can currently be disposed of with normal household waste after the end of therapy.

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