
**seca 375**  
**seca 376**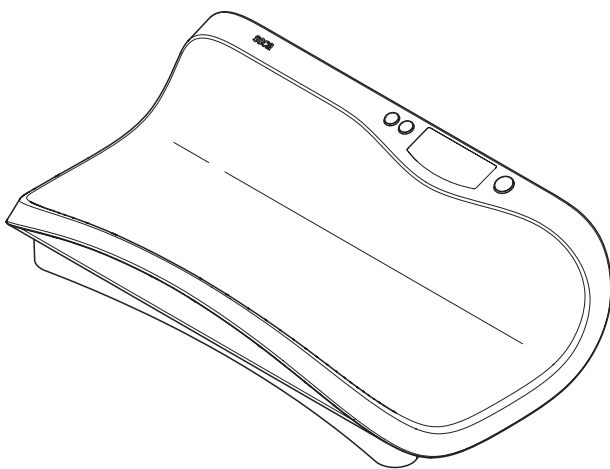

<b>D</b>	<b><i>Bedienungsanleitung und Garantieerklärung</i></b>	<b>3</b>
<b>GB</b>	<b><i>Instruction manual and guarantee</i></b>	<b>13</b>
<b>F</b>	<b><i>Mode d'emploi et garantie</i></b>	<b>23</b>
<b>I</b>	<b><i>Manuale di istruzioni e garanzia</i></b>	<b>33</b>
<b>E</b>	<b><i>Manual de instrucciones y garantía</i></b>	<b>43</b>
<b>DK</b>	<b><i>Betjeningsvejledning og garantibevis</i></b>	<b>53</b>
<b>S</b>	<b><i>Bruksanvisning och garanti</i></b>	<b>63</b>
<b>N</b>	<b><i>Bruksanvisning og garantierklæring</i></b>	<b>73</b>
<b>FIN</b>	<b><i>Käyttöohje ja takuu</i></b>	<b>83</b>
<b>NL</b>	<b><i>Bedieningshandleiding en garantieverklaring</i></b>	<b>93</b>
<b>P</b>	<b><i>Instruções de utilização e declaração de garantia</i></b>	<b>103</b>
<b>GR</b>	<b><i>Οδηγίες χειρισμού και εγγύηση</i></b>	<b>113</b>

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---

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# Full certification

---

In seca products you are purchasing not only technology matured over a century, but also quality confirmed by authorities, the legal system and by institutes.

seca products already meet the quality and technology requirements of tomorrow. With-seca, you are buying the future.

**M**

The products in these operating instructions comply with German legislation governing medical products and therefore with Directive 93/42/EEC of the Council of the European Community which is embodied in national laws throughout Europe.

Scales carrying this symbol are approved for medical purposes in the European Community. seca scales with this symbol meet the stringent quality and technical requirements placed on scales which can be calibrated.

Scales carrying this symbol are calibrated in accordance with Precision Class III of EC Directive 90/384/EEC.

Products carrying this symbol comply with the following directives and standards:

1. Directive 90/384/EEC governing non-automatic weighing instruments
2. Directive 93/42/EEC governing medical devices
3. DIN EN 45501 governing metrological aspects of non-automatic scales

seca's professionalism has also been officially recognized. The certificate from TÜV Product Service, the body responsible for medical devices, confirms that as a manufacturer of medical devices, seca rigorously complies with stringent legal requirements. seca's quality assurance system covers the development, production, sales and service of medical scales and measuring systems and ergonometry.

seca helps the environment.

We are concerned to save natural resources. We therefore strive to save packaging material where meaningful. What is left over can conveniently be disposed of locally via the German Dual System (recycling programme).

# 1. Congratulations!

---

In the **seca 375/376** electronic baby scale you have acquired a highly-accurate yet simultaneously sturdy piece of equipment. seca has been putting its experience at the service of health for over 150 years now, and as market leader in many countries of the world, is always setting new standards with its innovative developments for weighing and measuring.

The **seca 375/376** electronic baby scale is designed for use in a doctor's surgery or hospital. It serves both to establish birth weight and to check weight gain during the growth process.

These scales are a Class I medical device calibrated to meet Precision Class III (model 376) or IIII (model 375).

## 2. Safety

---

- Follow the instructions in the instructions for use.
- Keep the instructions for use and the declaration of conformity contained therein in a safe place.
- Never leave a baby lying on the scale unsupervised.
- Ensure that the scale is standing firmly on a smooth, level surface.
- Do not drop the scale or subject it to violent shocks.
- When using the scale with a mains unit, ensure that the supply cable is routed in such a way as to exclude any type of tripping hazard.
- Use only the type of battery stated (see "Battery and mains (option)" on page 15).
- Have scale serviced and re-calibrated on a regular basis (see "Servicing/recalibration" on page 20).
- Have repairs carried out only by authorised persons.

## 3. Before you start...

---

### Power supply

The scales are battery powered or draw power from an AC adaptor (optional). The LC display has low power consumption – 4000 weighing operations are possible on one battery. If there is insufficient battery voltage, first  appears in the display and after more power is consumed **bAtt** is shown.


## Battery and mains (option)

You should change the battery if  or **bAtE** appears in the display. You need a PP3, 9V E block battery.


- Turn the rear panel of the scales towards the front.
- Press the tab on the battery compartment and remove the cover.
- Remove the empty battery and insert a new battery in the battery compartment.

### Check that polarity is correct!

- Close the cover.

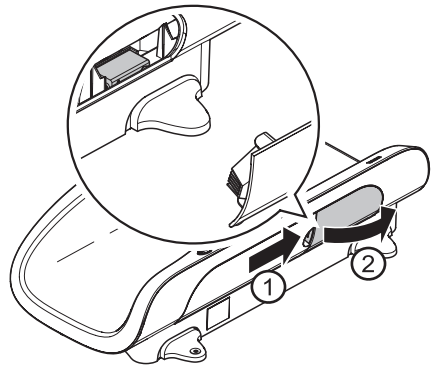Connection for mains unit: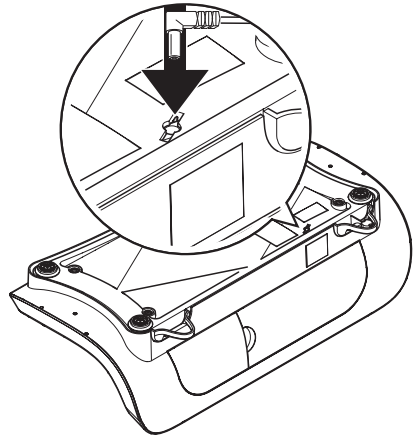


## 4. Setting up the scale

---

The scale is fully assembled on delivery.

- Remove the packaging and place the scale on a firm, level surface.

- Align the scale by undoing the foot screws. The air bubble in the spirit level must be right in the centre of the circle.
- Ensure that only the feet of the scale are in contact with the floor. The scale may not be in contact at any other point.

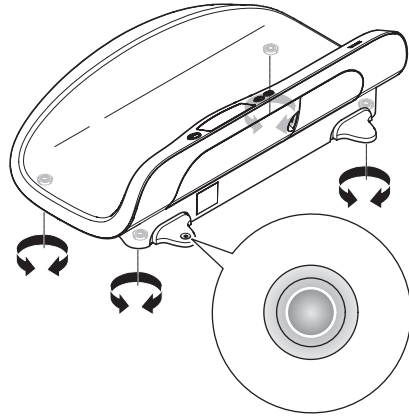## 5. Operation

---

### Correct weighing

- Press the green **START** key.  
**SECA**, **88888** and **0.000** appear consecutively in the display. The scale is then automatically set to zero and ready for use.
- If required, switch the weighing range (see page 16).
- Place the baby on the scale.
- Read the weighing result off the digital display.
- To switch off the scale, press the Start key again.

If the scale is operated with a mains unit, the ⚡ symbol appears in the display.

seca 375:

seca 376:

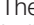
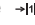
### Toggling between weighing ranges (only with Model 376)


The **seca 375/376** has two weighing ranges which can be selected. Weighing range 1 has a reduced maximum capacity but greater display resolution, whilst weighing range 2 allows you to use the maximum load capacity of the scale. Depending on the application in question, you decide whether you would prefer to use the fine display resolution or the high load range.

- Switch on the scale at the green Start key. The scale initially operates in the weighing range last selected.

- To change the weighing range, press the key .
- The  or  symbols in the display indicate which weighing range is currently active. This weighing range remains activated even after the scales are switched off.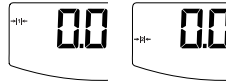

## Automatic switch-off/economy mode

To save power, when run on a battery the scales switch off after a short period without a load present on the scales. In mains operation, the scales remain switched on permanently.

## Taring weight display (NET)

The tare function is useful if an additional weight (such as cloths or a pad on the weighing surface) are to be ignored in the weighing result. Proceed as follows:

- Place the additional weight on the scale and press the button **NET**.  
The note "NET" appears.
- Wait until the display stops flashing and shows **0.000**.
- You can now perform as many weighing operations as you like with this additional weight.
- TARA is deactivated by pressing the button **NET** once again, or by switching the scales off.

seca 375:

seca 376:

seca 375:

seca 376:

### Important!

The maximum additional weight is 7.5 kg (weighing range 1, model 376), 10 kg (model 375) and 15 kg (weighing range 2, model 376). The maximum weight which can be displayed is reduced by the weight of the objects already placed on the scale.

## Storing the weight value (HOLD)

- Press the **HOLD** key whilst the load is on the scale.

The display flashes until a stable weight has been measured. The display is then frozen until the next weighing operation takes place.

- “HOLD” and  appears in the digital display. The triangle with the exclamation mark indicates that this frozen display is an additional facility which is not subject to calibration.

- The HOLD function can be switched off by pressing the **HOLD** key again.

**0.000** appears in the display if the scale has no load on it, otherwise the current weight. The “HOLD” display vanishes.

### Note:

The HOLD and TARE functions can also be used simultaneously.

## Checking that the contents of the calibration counter are correct

This seca scale is calibrated. Calibration may be performed only by authorised centres. To ensure this, the scale is fitted with a calibration counter which records any change to data relevant to calibration. If you want to check whether the scale is properly calibrated, proceed as follows:

- If necessary, switch off the scale.
- Keep any key depressed and start the scale.
- The current contents of the calibration counter are shown on the display for a few seconds.
- Compare the output contents of the calibration counter with the number shown on the calibration counter sticker.

For valid calibration, both numbers must match. If sticker and calibration counter do not match, then re-calibration is required. Contact your service agent or seca Customer Service.

If a supplementary calibration is necessary, then the supplementary calibration mark illustrated opposite is used to record the calibration counter position instead of the seca calibration mark shown above. This mark is secured with an additional seal by the person authorised to perform the supplementary calibration. The supplementary calibration mark is available from seca Customer Service under the number 14-05-01-886.

**Both numbers must match.**

## 6. Cleaning

---

Clean the tray and the housing as required with a domestic cleaning agent or commercially-available disinfectant. Follow the manufacturer's instructions.

## 7. What do I do if...

---

### ... no weight is displayed with a load on the scale?

- Is the scale switched on?
- Check the battery.

### ... 0.0000 does not appear before weighing?

- Press the Start key again - there must not be any load on the scale.

### ... one segment is illuminated permanently or not at all?

- That point has a fault. Inform your service agent.

### ... the display shows ?

- Battery voltage is dropping. You should replace the battery in the next few days.

### ... bAtE appears in the display?

- The battery is empty. Put in a new battery.

### ... 5tOP appears in the display?

- Maximum load has been exceeded.

### ... the display tErP appears?

- The ambient temperature of the scale is too high or too low. Place the scale in an ambient temperature between +10 °C and +40 °C. Wait about 15 minutes for the scale to adapt to the ambient temperature and then weigh again.

### ... Er:0 1 !: appears in the display?

- The scale is overloaded or the load is unevenly distributed. Take the load off the scale or distribute weight more evenly and restart scale.

### ... Er:0 i2: appears in the display?

- The scale was switched on with too great a load on it. Remove load and restart.

## 8. Servicing/recalibration

---

Have the scales recalibrated by authorised persons in accordance with national legal requirements. The year of initial calibration can be found behind the CE symbol on the rating plate above the number of the appointed office, 0108 (Hamburg Calibration Office). Recalibration is required in all cases if one or more calibration seals have been damaged or calibration counter contents does not match the number on the valid calibration counter sticker (see "Checking that the contents of the calibration counter are correct" on page 19). We recommend having your scales serviced by your local service agent before having it recalibrated. Seca Customer Service will be pleased to give you further assistance.

# 9. Technical data

---

## Dimensions

length:	358 mm
width:	620 mm
height:	190 mm

## Weight

ca. 3.5 kg

## Temperature range

+10 °C to +40 °C

## Height of figures

23 mm

## Power supply

Battery / (option mains unit)

## Power consumption

approx. 25 mA

## Medical product

### as per Directive 93/42/EEC

Class I

### *Model 375:*

#### Approval

Class III medical approval

#### Maximum load

15 kg

#### Minimum load

0.1 kg

#### Graduations

up to 10 kg 10 g

10 to 15 kg 20 g

#### Tare range

up to 10 kg

#### Accuracy at first calibration

up to 2 kg ± 10 g

2 to 10 kg ± 15 g

10 to 15 kg ± 30 g

### *Model 376:*

#### Approval

Class III medical approval

#### Maximum load

Weighing range 1 7.5 kg

Weighing range 2 15 kg

#### Minimum load

Weighing range 1 0.1 kg

Weighing range 2 0.2 kg

#### Graduations

Weighing range 1 5 g

Weighing range 2 10 g

#### Tare range

Weighing range 1 up to 7.5 kg

Weighing range 2 up to 15 kg

#### Accuracy at first calibration

Weighing range 1, up to 2.5 kg ± 2.5 g

Weighing range 1, 2.5 to 7.5 kg ± 5 g

Weighing range 2, up to 5 kg ± 5 g

Weighing range 2, 5 to 15 kg ± 10 g

# 10. Accessories

---

## Plug-in mains unit

Use only seca original plug-in mains units with 9 V or controlled 12 V output voltage. Commercially-available mains units may supply a higher voltage than stated and thus lead to damage to the scale.

Plug-type AC adaptor, Euro: 230V~ / 50Hz / 12V= / 130mA, order no. 68-32-10-252

Plug-in AC adaptor, reversible, Euro: 115V~ / 230V~ / 50-60Hz / 12V= / 350mA, order no. 68-32-10-243

# 11. Disposal

---

## Batteries

Do not throw used batteries away in domestic waste. Dispose of batteries at collection points in the vicinity. When buying new batteries, select those low in harmful substances and containing no mercury (Hg), cadmium (Cd) or lead (Pb).

## Disposing of the scale

If the scale can no longer be used, the responsible waste disposal association will be glad to advise you on the measures necessary to dispose of it properly.

# 12. Warranty

---

A two-year warranty from date of delivery applies to defects attributable to poor materials or workmanship. All moveable parts - batteries, cables, mains units, rechargeable batteries etc. - are excluded. Defects which come under warranty will be made good for the customer at no charge on production of the receipt. No further claims can be entertained. The costs of transport in both directions will be borne by the customer should the equipment be located anywhere other than the customer's premises. In the event of transport damage, claims under warranty can be

honoured only if the complete original packaging was used for any transport and the scale secured and attached in that packaging just as it was when originally packed. All the packaging should therefore be retained.

A claim under warranty will not be honoured if the equipment is opened by persons not expressly authorised by seca to do so.

We would ask our customers abroad to contact their local sales agent in the event of a warranty matter.

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