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### Mobilex gummiramper/-kiler

Varenr. 306040-306602

**MOBILEX A/S**  
Grønlandsvej 5  
DK-8660 Skanderborg  
Danmark  
www.mobilex.dk

## Indledning

Kære kunde

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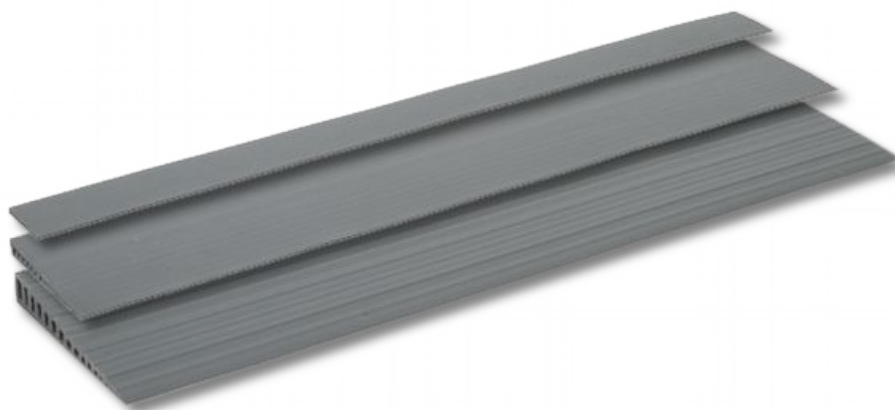
## General information

Dear customer

This user manual contains a description of the medical device and important guidances to ensure a correct and safe usage of the product. It is important prior to use to read this manual carefully. It is especially important to read the safety requirements and follow these. We continuously develop our products and we reserve the right without further notice to change the specifications and functions of products.

## Intended use of the device

The product is a medical device designed to enable elderly or disabled people with or without the assistance of a helper to overcome obstacles, such as thresholds. The ramps are designed to compensate for height differences from 4 mm to 60 mm and are modular in design. The ramps can be used both indoors and outdoors. Other uses than those described are not permitted.

By questions please take contact to your dealer or directly to Mobilex A/S.

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Phone: +45 87 93 22 20  
Telefax: +45 87 93 17 77  
E-Mail: info@mobilex.dk

## Assembly and storage

The rubber ramps are supplied either with or without adhesive film. Equipped with adhesive film, they are suitable for a smooth surface such as tiles or other smooth floors and can therefore be stuck firmly to the subfloor. For carpets or sensitive wooden floors, ramps without adhesive foil must be used. Mobilex also offers a cleaning liquid to remove adhesive residues (item nr. 306905). Mobilex accepts no liability for floors damaged by the adhesive. The rubber frames are available in 17 different heights (from 4 mm to 60 mm) and 3 different widths (700 mm, 900 mm and 1000 mm). If the rubber ramps are to be further adjusted in width, this can easily be done with a normal hobby knife or a saw blade (item nr. 306904). The ramps can therefore be easily adapted to all door widths. The ramps are made of TPE and have a non-slip surface. The rubber ramps are easy to store in a room or car and are quick and easy to use.

## Maximum load

The Mobilex rubber ramps are tested and approved up to a maximum load of 300 kg.

## Material and cleaning

The Mobilex rubber ramps are made of TPE and contain no toxic substances (PVC-free). The rubber ramps can easily be cleaned with normal detergents that do not contain solvents.

## Expected life and disposal of the product

Mobilex rubber ramps have a service life of at least 10 years under normal use. If the product can no longer be used, it must be taken to the local waste disposal site.

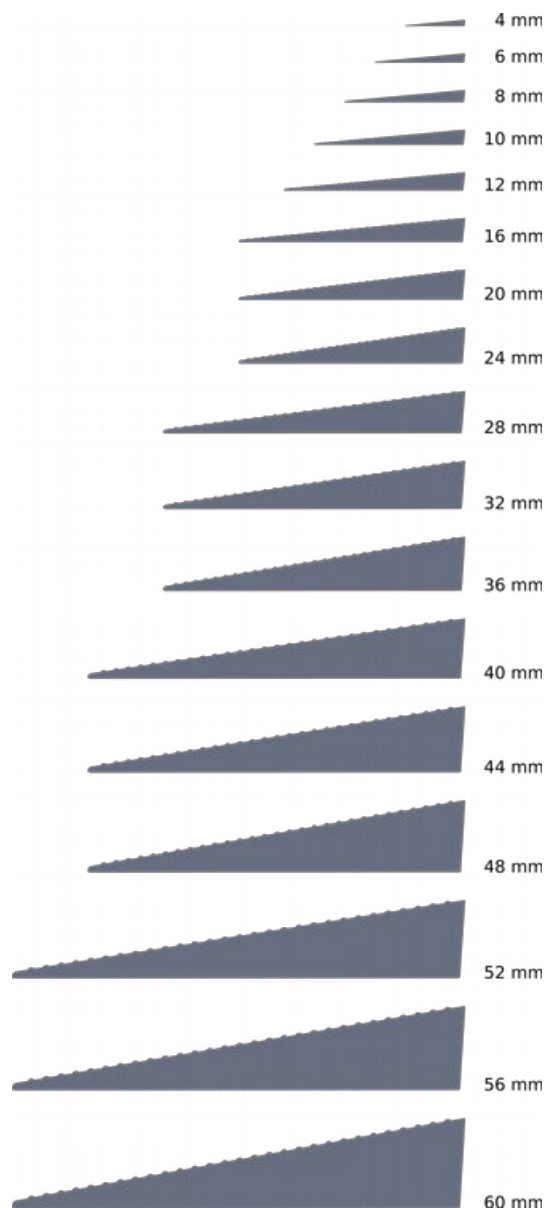
## Warranty

Mobilex A/S offers a 2 year warranty on damage caused by production or material defects. The warranty expires if the product is not repaired, maintained or used as intended. Parts subject to normal wear and tear are not covered by the warranty. Product modifications without our written consent invalidate the CE marking and our product liability. In the event of product damage covered by this warranty, please notify your dealer or Mobilex A/S directly. The warranty does not cover transport costs, damages during repair or personal injury. The warranty does not cover injury or other damage resulting from non-compliance with the instructions for use.

## Synoptic table


Height	Depth	Width	Weight
4 mm	40 mm	900 mm	130 g
4 mm	40 mm	1000 mm	150 g
6 mm	60 mm	900 mm	310 g
6 mm	60 mm	1000 mm	350 g
8 mm	80 mm	900 mm	490 g
8 mm	80 mm	1000 mm	540 g
10 mm	100 mm	900 mm	770 g
10 mm	100 mm	1000 mm	860 g
12 mm	120 mm	900 mm	1070 g
12 mm	120 mm	1000 mm	1190 g
16 mm	150 mm	900 mm	1230 g
16 mm	150 mm	1000 mm	1370 g
20 mm	150 mm	900 mm	1850 g
20 mm	150 mm	1000 mm	2050 g
24 mm	150 mm	900 mm	1950 g
24 mm	150 mm	1000 mm	2170 g
28 mm	200 mm	900 mm	2180 g
32 mm	200 mm	900 mm	2960 g
36 mm	200 mm	900 mm	3990 g
40 mm	250 mm	900 mm	3550 g
44 mm	250 mm	900 mm	4250 g
48 mm	250 mm	900 mm	5190 g
52 mm	300 mm	900 mm	5800 g
56 mm	300 mm	900 mm	6500 g
60 mm	300 mm	900 mm	7700 g

**The Mobilex rubber ramps with adhesive tape are also available in 700 mm length.**

### Cutting the rubber ramps to size

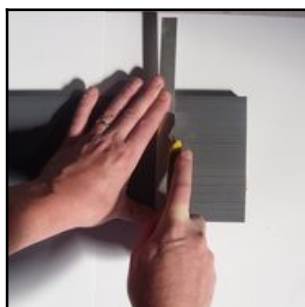With a sharp hobby knife or the Mobilex saw blade (item nr. 306904) you can easily adjust the width of the rubber frames. For higher ramps, the cut must be made several times. Bend the ramp back slightly to facilitate the next cut.

*Cutting low ramps to size*


*Cutting higher ramps to size*

*Ramp on site*

## Possible combinations

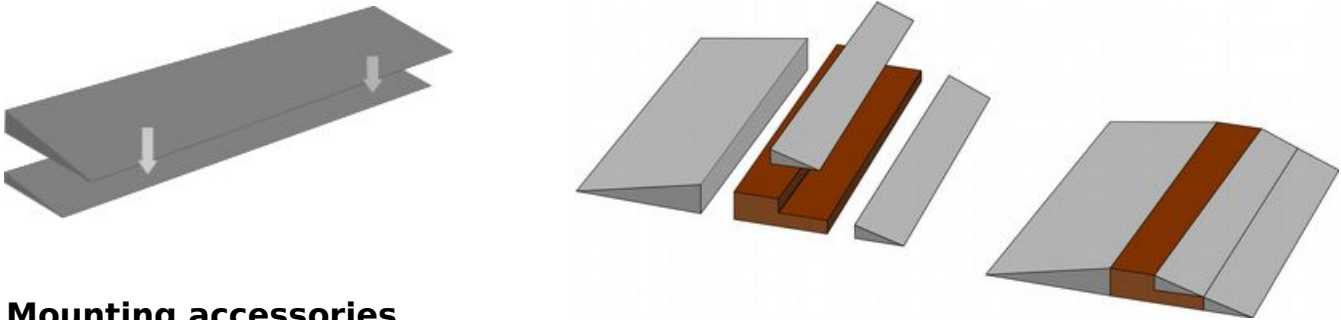Measure the inside, outside and height of the threshold as accurately as possible and select the appropriate ramps. The ramps can also be laid on top of each other in modular fashion.

## Mounting accessories

The mounting kit for rubber ramps (item nr. 306910) consists of two mounting brackets which are fixed in the door threshold and two pins which are inserted laterally into the ramp. This allows the ramp to be attached to the threshold.

## Double-sided adhesive tape

Mobilex rubber ramps are supplied as standard with double-sided adhesive tape on the underside. The double-sided adhesive tape is also available as an accessory for replacement or retrofitting. (item nr. 306911 or 306912).

## Solvent for adhesive removal

To remove the double-sided adhesive tapes from the underside of the rubber ramps, a special solvent is available as an accessory. (item nr. 306905).

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The company Mobilex A/S, Grønlandsvej 5, 8660 Skanderborg, Denmark declares that the

**Mobilex rubber ramps** (item nr. 306040 - 306602)

are designed in accordance with the technical documentation and made of which conforms to the following basic requirements:

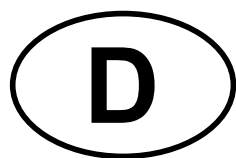
**EU Council Directive 93/42/EEC for medical equipment**

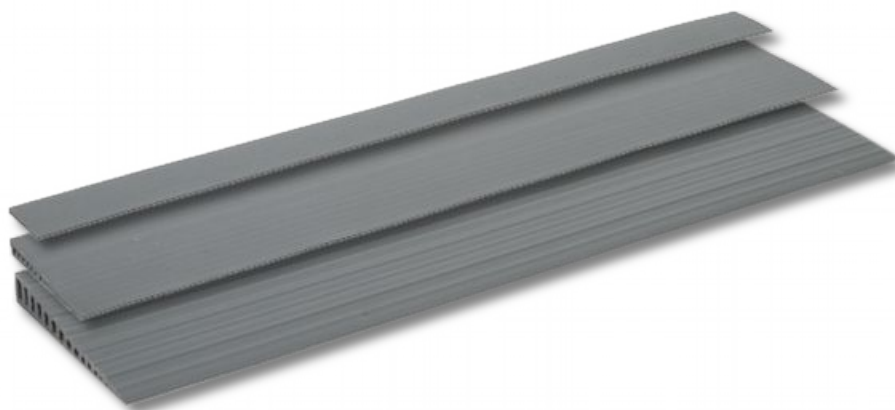
Lars Nygaard  
Managing director  
Mobilex A/S

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La ditta Mobilex A/S con sede in Grønlandsvej 5, 8660 Skanderborg, Danimarca dichiara che le