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www.rossmax.com

Warranty Registration (Must be completed within 10 days of purchase)

Customer Name: _____
Street Address: _____
City: _____
State: _____
ZIP code: _____
Telephone: _____
E-mail address: _____

Gender: Male Female **Age:** _____

Product Information

Date of purchase: _____

Store where purchased: _____

Price Paid (excl. Tax): _____

This instrument is covered by a 5 year guarantee from the date of purchase, batteries and accessories are not included. The guarantee is valid only on presentation of the guarantee card completed by the dealer confirming date of purchase or the receipt. Opening or altering the instrument invalidates the guarantee. The guarantee does not cover damage, accidents or non-compliance with the instruction manual. Please contact Rossmax service.

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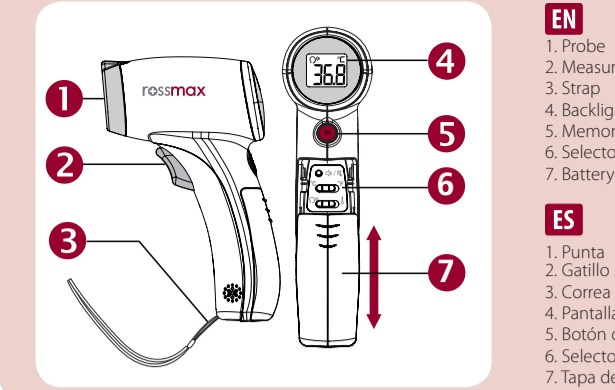
EN

- Probe
- Measuring Trigger
- Strap
- Backlight LCD Display
- Memory Button
- Selector Zone
- Battery Cover

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EN English Introduction

Rossmax non-contact temple thermometer using an infrared detector to detect body temperature from the temple for adult and child in the hospital, home or public place. This thermometer could also detect ambient temperature.

The quality of the infrared thermometer has been verified and conforms to the provisions of the EC council directive 93/42/EEC (Medical Device Directive) Annex I essential requirements and applied harmonized standards. Comply with ASTM I1965-98.

* This thermometer converts the temple temperature to display its "oral equivalent" (according to the result of the clinical evaluation)

⚠️>Please read the instruction manual carefully before using the product and be sure to keep this manual.

Function

The thermometer has been designed for practical use. It's not meant to replace a visit to the doctor. Please also remember to compare the measurement result to your regular body temperature.

The object mode shows the actual, unadjusted surface temperatures, which is different from the body temperature. It can help you to measure any object other than body surface, for example the baby's milk.

If thermometer detects a body temperature >= 37.5°C(99.5°F) there will be four beeps sound with flashing reading to warn the user for potential fever.

How to measure temple temperature

1. Vertically aim the thermometer at the temple with a distance of less than 10 cm. If the temple is covered with hair, sweat or dirt, remove the obstacle from the temple to improve the measuring accuracy.

2. Press the "Measuring Trigger" button to take temperature. Measurement is complete when you hear a long beep sound.

3. Read the recorded temperature from the LCD display.

Switching between Fahrenheit and Celsius

No matter the thermometer unit is powered on or off, for changing between the °C and °F please open the battery cover and set the slide switch locating on the top of the battery to the corresponding measuring scale before the next measurement.

After setting a new measuring scale, the measuring site display icon will also be switched accordingly.

How to measure object temperature

No matter the thermometer unit is powered on or off, for changing the measuring target between the temple and object, please open the battery cover and set the slide switch locating on the top of the battery to the corresponding measuring mode or target before the next measurement. After setting a new measuring target, the measuring site display icon will also be switched accordingly.

Once aiming the measured target, by pressing the measuring trigger and then releasing the trigger for getting one time measurement.

While via continuously pressing and holding the measuring trigger can get a string of the consecutive, updated measuring reading until releasing the trigger.

Note:

1. Applications include temperature measurements for Water, Milk, Cloth, Skin or other object.

2. This mode shows the actual, unadjusted object temperatures, which is different from the body temperature.

Enabling or disabling the beep sound

- Press the measuring trigger to turn on the device.
- Press the "buzzer" button to turn on and the display shows "bi-On".
- Press the "buzzer" button to turn off and the display shows "bi-Off".

Function memory

There are total 9 set memories for measurement records. If the reading of the thermometers is within the normal temperature range of 34 °C to 42.2 °C (93.2 °F to 108°F), when the measurement is done, the measurement data is saved into memory.

* Press "Memory" button to repeatedly to recall each temperature memory.

Care and cleaning

- Storage temperature range: It should be stored at room temperature between -25~+55°C, (RH<=85%)
- Keep the unit dry and away from any liquids and direct sunlight.
- The probe should not be submerged into liquids.
- Please check the device if damaged once it falls. If you can't make sure of it, please send the complete device to the nearest retailer for re-calibration.
- Holding the thermometer too long may cause a higher ambient temperature reading of the probe. This could make the body temperature measurement lower than usual. Please try not to touch the thermometer probe part for avoiding the incorrect reading.

Battery Replacement

- Open the battery cover: use thumb to push battery cover out.
- Hold the device and flip the battery out.
- Insert the new battery down with the correct electrical polarity.
- Place back the battery cover.
- Keep the battery away from children.

Troubleshooting:

Error Message	Problem	Solution
EE	The system is not functioning properly.	Unload the battery, wait for 1 minute and re-power it. If the message reappears, contact the retailer for service.
EI	Exceeding operating temperature range	Allow the thermometer to rest in a room for at least 30 minutes at room temperature; for temple mode:16~40°C (60.8~104°F); for object mode:10~40°C (50~104°F).
Hi	(1) In temple mode: Temperature taken is higher than +42.2°C (108°F) (2) In object mode: Temperature taken is higher than 99.9°C (199.9°F)	Please select the target within specifications. If a malfunction still exists, please contact the nearest retailer.
Lo	(1) In temple mode: Temperature taken is lower than +34°C (93.2°F) (2) In object mode: Temperature taken is lower than 0°C (32°F)	Please select the target within specifications. If a malfunction still exists, please contact the nearest retailer.
BI-ON	Device cannot be powered on to the ready stage.	Change with a new battery.

Specification

Temperature measurement range	Temple mode: 34–42.2°C (93.2–108°F) Object mode: 0–99.9°C (32–199.9°F)
Accuracy	For temple mode: +/-0.3°C (0.5°F) during 34–42.2°C (93.2–108°F). For object mode: +/- 1°C or +/- 2°F when measuring the object temperature between 0–99.9°C (or 32–199.9°F)
Operating condition	Temple mode:16~40°C (60.8~104°F),RH<=85%;700~1060 hPa Object mode:10~40°C (50~104°F), RH<=85%;700~1060 hPa
Storage and transportation condition	Temperature:-25~55°C (-13~131°F), RH<=85%;700~1060 hPa
Memory	9 set
Display resolution	0.2/0.32 X 1
Weight	141.5g (without battery)
Size	10.4cm(L) x 5.2cm(W) x 1.5cm(H)
Auto shutdown	30 sec
Battery life	3000 consecutive measurements or 1 year with 1-2 measurements per day including stand-by mode.
Safety classification	II Type BF equipment
IP Classification	IP22: Protection against water and dust
*Dispose of device and batteries according to local regulations	

Caution

- Only use the device once you have read and understood these instructions for use. Keep the instruction manual for use.
- The thermometer and patient needs to stay in the room in which the measurement is taken for at least 30 minutes before use.
- It is recommended that you always take the temperature in the same location, since temperature readings may vary according to the locations.
- It is recommended that three temperatures are taken and the highest one taken as the reading - if new born infants
- Children with a compromised immune system and for whom the presence or absence of fever is critical.

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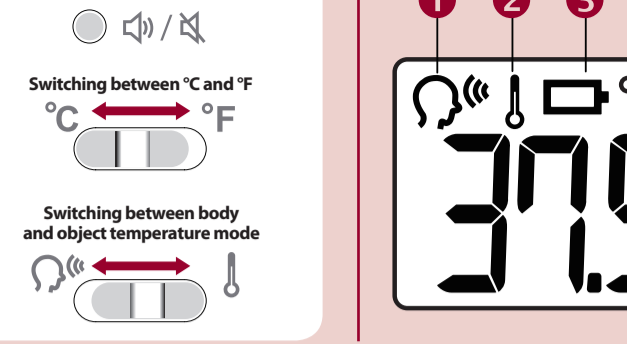
*Dispose of device and batteries according to local regulations

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