Toilettenstuhl RFM® fahrbar

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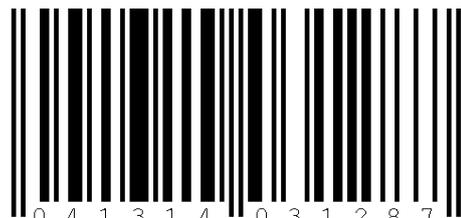
Hi. Mi. Nr. 18.46.02.0061
PZN (Pharmazentralnummer): **09466266**

Mobile commode chair RFM®

⚠ Carefully read the complete instruction manual before mounting, adjusting or using the mobile commode chair RFM.

Intended purpose

The mobile commode chair RFM supports people who are restricted in their movement and mobility and have trouble reaching the toilet. Indoor use only.

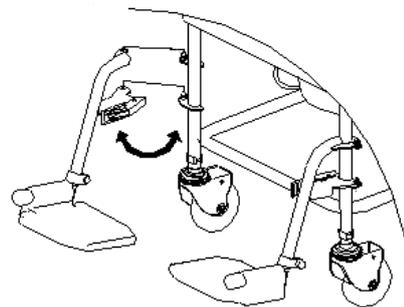
Indication

The commode chair can be used by people with


- Minor disorders of balance
- Malfunctions of the lower extremities
- State of paralysis
- Poor trunk control
- Restricted mobility of the hip and knee joints

Assembly instructions

1. Remove the lower frame, seat, backrest and seat kit, pegs and rotary handles from the box.
2. Remove the bucket, lid and splash guard from the box.
3. Place the seat on the lower frame and make sure that the narrow part of the seat recess pointing is in the same direction as the clips.
4. Assemble the backrest and seat assembly kit on the lower frame and make sure that they are tightly screwed. There is a push bar on the back rest, used to slide the patient with the chair across the toilet. For this purpose, the seat is folded up and the splash guard is placed on the pan frame. The commode chair is now opened and pushed across the toilet.
5. Tighten the left and right twist handle and fix it.
6. Mount the left and right foot supports on the base frame clips and snap them. The foot supports can be slewed outwards or removed and are adjustable in height so that they can be adapted to the length of the patient's leg.
7. Fasten the padded armrests on the side of the frame. These can be pivoted when needed or to facilitate the transport.

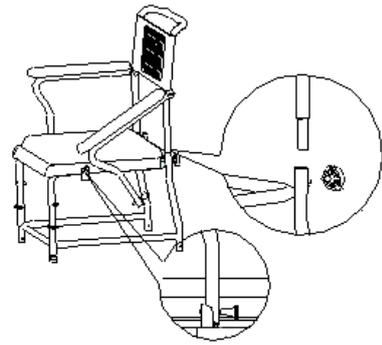8. At the rear end of the seat assembly kit there are two notches for fastening the padded seat. It is pressed down and catches in the notches.
9. Under the seat there is a guide rail, which allows the commode container to be pushed or pulled out from behind underneath the chair. It has a lid and a handle to ensure safe transport. Remove the bedpan and the lid at the same time in order to empty the bucket.


10. The wheels are inserted from below into the frame and can be easily removed again. All wheels are pivotable.

Contraindication

The use of commode chair is not indicated for persons with

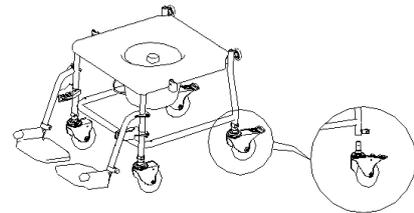- Disorders of perception
- Serious disorders of balance
- People with amputated limbs

Safety instructions

1. Check the fitting and strength of all screws, nuts and rotary knobs weekly to ensure that the use of the commode chair is safe.
2. To ensure that your commode chair provides the best service, always use original spare parts, which you can get from your dealer.
3. Make sure that the patient has placed his arms and legs on the armrests and foot support to avoid injury when pushing!
4. Caution: Make sure that the patient has not placed his arms on the armrests before you fold them away!
5. Press both levers on the fixable castors down before the commode chair is used, in order to prevent any unintended movement of the commode chair.
6. Do not use a faulty commode chair!
7. Maximum load: 120 kg

Using the brakes

Close to the two rear wheels there are parking brakes, which are locked into place by pressing down the lever. By pressing down a smaller lever, the brakes are released again. The directional locks are also located on the rear wheels, which are activated by locking the brakes to avoid the slipping of the commode chair.

Cleaning

You should clean and disinfect the commode chair, which is intended for reuse, at regular intervals. This is compulsory if the chair is used and re-used by another patient! Use a commercially available mild detergent without scouring additives. The removable seat can be rinsed with running water and then disinfected. The bucket should also be disinfected and cleaned in this way. The entire chair can be disinfected with the commercially available detergents in accordance with the manufacturer's instructions, e.g. RFM Sept. Do not use abrasive or corrosive cleaning agents or high-pressure and steam cleaners!

Maintenance

The mobile commode chair RFM is maintenance-free. All padded surfaces and the bucket are made of easy-care PE or PVC, which is water-repellent. The frame is made of powder-coated aluminum and therefore not susceptible to corrosion.

Malfunction: 

If you notice any damage or signs of wear and tear, as well as loose mounted parts, contact your dealer.

The commode chair is suitable for use at home / in the apartment and should remain dry at 5-45 °C (40-60% r. F.).

Scope of delivery

Commode chair preassembled
Commode bucket with a lid
Seat and cover
Instruction manual

Technical specification / dimensions and weight

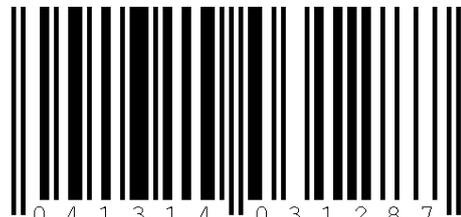
Weight: 13 kg
Seat height: 53 cm
Total height: 95 cm
Height of the backrest: 19 cm
Seat width: 45 cm
Seat depth: 44 cm
Seat angle: 90° with the cover closed
Height of the armrests: 30 cm
Width between armrests: 45 cm
Width of backrest: 44 cm
Overall width: 56 cm
Padding of the backrest: 15 mm
Padding of the armrests: 30mm
Diameter of the wheels: 12 cm
Maximum load: 120 kg

Guarantee

The warranty period is 24 months and shall apply to any product defects that can verifiably be attributed to manufacturing or material defect.

Daimlerstrasse 12 a
D-25337 Elmshorn
Phone no.:
Fax: →(0)4121 4733 23
www.rehaforum.com
info@rehaforum.com
Rev. 05/2016

4 0 4 1 3 1 4 0 3 1 2 8 7
Hi. Mi. No. 18.46.02.0061
PZN (central pharma number): **09466266**