## Duschstuhl RFM, faltbar

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DE

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### Manual

UK 

Please read the complete instruction manual carefully before installing, adjusting or using the RFM folding shower chair for the first time. Keep the instructions for use for future reference.

#### Intended use

The RFM folding shower chair is designed to support people with reduced mobility and enables them to shower while seated. It facilitates regular body hygiene and drying after showering.

The shower chair can be placed in the shower tray for showering or in a bathtub with shower facilities, provided the bathtub is wide enough.

#### Indication

Sitting in the shower is suitable for people who are unable to stand during body cleansing due to: Walking difficulties, signs of paralysis, slight loss of balance, impaired vision, loss of strength due to illness or cardiovascular problems, amputation of the lower limbs and torso and back pain.

#### Contraindication

The use of the shower chair is not recommended for persons with severe balance disorders, persons with amputations of the upper limbs, severe perceptual disorders or inability to sit due to illness.

#### Mounting/Instruction

1. The shower chair RFM, foldable, is delivered pre-assembled in a cardboard box. Check the contents and packaging for intactness and completeness.

2. Remove the chair from the carton and unfold the side parts outwards before pressing the seat surface onto the clamping device of the frame parts. The seat must snap into place.

3. Push the backrest onto the upper ends of the frame, pushing in the push buttons. These snap into place on the back of the backrest.

4. The legs are adjustable in height. To adjust, pull the legs out of the frame and fix them with the clips.

5. There are openings in the frame as well as in the pull-out legs through which the metal pin of the clip must pass to hold the legs at the desired height. The curved lock automatically wraps around the frame parts.

### **Safety Instructions** ⚠

1. Before each use, make sure that the RFM folding shower chair has a firm, straight position.
2. Check the strength of the backrest before use!
3. Make sure that all four legs are fixed at the same height. Check the position of the clips.
4. Caution: There is a risk of jamming when the seat surface is folded down!
5. Damaged or worn parts may only be replaced by a specialist dealer!
6. Do not use a defective shower chair!

The RFM foldable shower chair is designed for a maximum load of 113 kg and is only intended for use in closed rooms!

### **Cleaning, disinfection and reuse**

You should clean and regularly disinfect the RFM folding shower chair, which is intended for reuse, after each use. This is absolutely necessary for use and re-use by another patient/person! Rinse the shower chair with warm water and use mild soapy water for cleaning. Dry the shower chair with a soft cloth and ensure good ventilation in the bathroom.

Disinfection can be carried out with a commercially available agent according to the manufacturer's instructions, e.g. Descosept AF. The materials of the shower chair, polyethylene and anodised aluminium, are insensitive to the use of commercially available disinfectants and cleaning agents.

Do not use abrasive or corrosive cleaning agents or high-pressure or steam cleaners!

### **Maintenance**

The RFM folding shower chair is maintenance-free. All surfaces are made of easy-care polyethylene. The frame parts are made of anodised aluminium and are therefore not susceptible to corrosion.

Malfunction: 

If you notice any damage or signs of wear or loose parts, contact your dealer.

### **Scope of delivery**

Shower chair RFM foldable, pre-assembled incl. backrest

Four clips for the chair legs

Instruction manual

### **Technical Data/Dimensions and Weight**

Weight: approx. 5 kg

Height (incl. backrest): 84 cm (adjustable up to 96 cm)

Width: 50 cm

Depth: 46 cm

Seat width: 40 cm

Seat depth: 41 cm

Seat height: 42 cm (adjustable up to 58 cm)

Width (outside armrests): 56 cm

Max. Load capacity: 113 kg

### **Warranty**

The warranty period is 24 months and covers all product defects that can be proven to be due to manufacturing or material defects.

### **Useful life**

We, the company Rehaforum Medical GmbH, Daimlerstrasse 12a, 25337 Elmshorn as manufacturer of the product shower chair RFM foldable, art. no. 09674705 declare that the expected service life of the above mentioned product is up to 5 years if the use is in accordance with the information in the user manual. The product can be used beyond this period if it is in a safe condition.

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