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EN- Please read these safety instructions carefully before use, and keep them for future question.

1.Safety indications

- ♥ ALWAYS consult with your physician or supplier to determine proper adjustment and use of the device.
- ♥ Don't store outside and don't let under the sun (Beware : don't let the product in a car in case of high outside temperature !!).
- ♥ Don't use in extreme temperature (above 38°C or 100°F or below 0°C or 32°F).
- ♥ Check carefully all parts of this item (screws, tubes, tips, grips)..
- ♥ Check regularly if all screws are tightened.
- ♥ Use only as a commode chair and in proper way : don't stand up on the chair.
- ♥ ALWAYS observe the weight limit on the labelling of your product.
- ♥ No modification must be made on this device. Use only original parts to replace the broken or defect one and let this be done by the technical staff.
- ♥ After unfolding or assembling the chair, make sure that it is securely locked in OPEN position and level to the ground before using. Some chairs are already mounted.
- ♥ DO NOT hang anything on the Chair.
- ♥ ALWAYS test to see that the chair and attachments are properly and securely locked in place before using.
- ♥ In case you didn't use your chair for a long time, let it checked by the technical staff, before using.
- ♥ Do not store outdoor, the chair must be used at home.
- ♥ Do not install or remove the pan while the patient is seating on the chair! High risk of sex injuries.
- ♥ Be careful when removing the armrests to avoid trapping your fingers.
- ♥ Check the chair before using, in case of doubt ask your physician or provider.
- ♥ Use only the chair on a flat and solid ground.
- ♥ It is forbidden to seat on the armrests.
- ♥ Do not lean out of the chair to avoid losing balance.

- ♥ Any serious incident that has occurred in relation to the device shall be reported to the manufacturer and the competent authority of the Member State in which the user and/or the patient is established.

2. Indication and Contraindication

Commode chairs can be used at home or in nursing homes to assist a person who has difficulty getting to the bathroom by bringing the toilet into the bedroom. Standard, foldable or on castors, they shall be use only indoor.

Indications: Used in the home when the patient is too debilitated to reach the bathroom.

Contraindications: Don't use the commode chair in case of:
- Cognitive disorder - Loss of balance - Siting impairment

3. Installation

Only specialists can adjust the product and teach you how to use it. Ask the technical staff how to use this product and make sure that you understood clearly their explanation. In case of doubt, don't hesitate to ask again.

Assembly:

1. Remove commode from packaging and check for craks and damages.
 2. Unfold commode's frame [A] and stand upright with sides perpendicular to the backframe see drawing [2].
 3. Lift up front tube [B] see drawing [2] and attach it by depressing push buttons and sliding brace into side frame socket [C] located on each side of the chair. Beware risk of injury! Make sure that push button "snap" correctly through holes and that the chair is stable.
 4. Attach backrest tube [D] by depressing push buttons and sliding brace into side frame socket located on each side at the back of the chair. Beware risk of injury! Make sure that push button "snap" correctly through holes and that the backrest is connected to the frame.
 5. Install commode bucket and anti splash guard [E] into bracket and attach lid [F] onto back of frame.
 6. To unfold, depress push button on back support and remove. Depress push button on front frame and remove the front tube.
- To remove the pan, lift up the seatplate and remove the pan from the top. Close the pan with the lid before transporting it.

Leg adjustment see drawing [H]:

Depress the push button in the frame and adjust the leg extension to the required height adjustment hole. Make sure that the push button is fully engaged through the extension leg hole. Check the stability before use. Adjust all legs at the same level.

Rental:

This product can be rented to different patients. A thorough maintenance, cleaning and disinfection must be performed before the product is given to a new patient in accordance with the following steps:

- 1- Visual inspection: ensure that the product has not been damaged and check for any defect.
 - 2- Functional inspection: ensure that the product functions properly and repair it if needed.
 - 3- Cleaning and disinfection: this product can be cleaned and disinfected with mild, chlorine-free detergents such as Survanios (Anios laboratories). Do NOT use any abrasive cleaners or high-pressure cleaners that could damage the product.
- Use only appropriate tools to perform the maintenance, cleaning and disinfection.
The user's manual is an integral part of the product and must be handed over to every new user.

3. Cleaning

The chair has to be cleaned regularly and after each use with a mild cleaning agent and a soft cloth.

4. Materials

The product is made up of steel tubing covered with epoxy painting. Upolsteries are in plastic. The chair is delivered with a pan.

5. Disposal

Dispose of this product in accordance with your regional waste disposal regulations.

6. Technical specifications

This product has a 2 years warranty, from the date of purchase, against any manufacturing defect. This product is biocompatible. Test : EN-ISO 9999 : 091203, EN 12182, EN 1041, ISO 10993-ff. Duration of use: 2 years.

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