## Bettgalgen RFM

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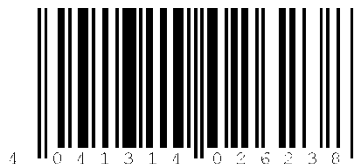
## RFM Bed Trapeze Bar

User manual

Carefully read the complete user manual before using and mounting the RFM Bed Trapeze Bar for the first time. Keep the user manual safely so that it is available for later use.

### **Intended purpose**

The RFM Bed Trapeze Bar can be used in indoor rooms in domestic as well as in care or clinic situations. It serves for assistance in rising from the lying to the sitting position and for pulling up or holding tight on the trapeze handle in order to change the lying or sitting position in bed. The use of a bedpan is made easier through the raising of the buttocks by using the trapeze handle to lift the body. For this, the user / patient must use his or her own body strength to hold firmly onto the trapeze handle.

### **Indication**

The RFM Bed Trapeze Bar is suitable for people who are unable to sit up in bed unaided or to change their position because of paralysis, amputations of the lower extremities, strong torso or back pain or loss of strength due to heart or circulatory diseases.

### Contraindication

The use of the Bed Trapeze Bar is not suitable for people with severe cognitive disorders, back injuries, hemiplegia or inability to sit due to sickness and patients with apoplexy, who are prohibited from changing a lying or sitting position.

### Assembly instructions

1. The RFM Bed Trapeze Bar is delivered in a box. Check if the contents and the packaging are complete and intact.
2. Remove the single parts (base of the Bed Trapeze Bar, mounting rod, elbow and handle) from the box.
3. Place the base of the Bed Trapeze Bar on the floor. Remove the clear film. Note: the surface of the base has a size of approximately 86 x 76 cm.
4. Insert the mounting bar into the opening of the base and make sure that the longer, thick part of the bar is at the upper end.  
**An arrow marker**, which is present on the rod, indicates this.
5. Tighten the screw in the base of the Bed Trapeze Bar, in order to fix the mounting rod.
6. Insert the elbow in the opening of the mounting rod and make sure that the arm of the elbow protrudes above the base of the Trapeze Bar.

7. Attach the handle to the end of the elbow by closing, tightening and pulling the loop through the metal eyelet.

8. Depending on the bed height and arm length, the handle height can be adjusted between approx. 1.33 m and 1.24 m when measured from the floor. To do so, from the front or the back, push the grey belt through the plastic guides of the belt divider. This automatically fixes the belt in its guides due to the hanging position of the handle.

### Safety regulations

1. Ensure that the trapeze bar is standing at the head end of the bed and that the trapeze handle is hanging parallel to the head end of the bed.
2. The handle should be at the height of the upper chest / height of the chin.
3. If you are holding the handle, do not release it suddenly. If you swing it, it may strike your head or the head of the nursing staff.
4. Check the height and the fixed connection of the handle regularly. Ensure that the Bed Trapeze Bar is only moved in the dismantled state to prevent damage and injuries!
5. **The maximum load of the Trapeze Bar is 100 kg!**
6. Ensure that the Bed Trapeze Bar is fixed to the bottom and the screw of the base is firmly tightened. The Bed Trapeze Bar is only suitable for indoor use!
7. In order to switch to a wheelchair or commode chair, please ask the nursing staff to assist you from in order to avoid accidents!
8. Do not leave children unattended in the vicinity of the Bed Trapeze Bar - it can cause personal injury or risk of falling!
9. Do not use a defective RFM Bed Trapeze Bar!
10. Do not hang any objects on the RFM Bed Trapeze Bar!

### **Cleaning and Disinfection**

You should clean and disinfect the Bed Trapeze Bar, which is intended for reuse, at regular intervals. This is compulsory if the Trapeze Bar is used and re-used by another person or patient! Wipe the frame and the handle with a commercially available, mild soap solution and dry all the elements with a towel or tea towel.

The RFM Bed Trapeze Bar can be disinfected with a commercially available disinfectant. It is advisable to disinfect the handle daily.

The painted frame and plastic handle are not sensitive to corrosion.

Do not use abrasive or corrosive cleaning agents or high-pressure and steam cleaners!

### **Maintenance**

The RFM Bed Trapeze Bar is maintenance-free.

Periodically check the tightness of the base and the strap of the handle.

If you notice wear or changes / corrosion on the material or signs of fraying on the handle, contact your dealer. Replace defective parts, if possible. Do not use a defective RFM Bed Trapeze Bar!

### **Scope of Supply**

RFM Trapeze bar disassembled, consisting of an installation pole, a base plate, angle piece, trapeze handle and user instructions

### **Technical specification / dimensions and weight**

Weight: 13 kg

Max. load capacity: 100 kg

Height: 176 cm, jib length approx. 75 cm

Dimensions of the stand area of the base: 83 x 76 cm

Side dimensions of the handle: approx. 25 cm

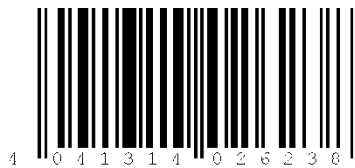
Material: painted steel

### **Guarantee**

The warranty period is 24 months and shall apply to any product defects that can be verifiably attributed to manufacturing or material defect.

Rehaforum Medical GmbH  
Daimlerstrasse 12a  
25337 Elmshorn  
Phone: +49 4121 473312  
www.rehaforum.com

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