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POSIMED®

POSIMED®

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POSIMED®

Positioning pillow

Please read the operating instructions carefully before using the system and heed the **warnings!**

Product name	POSIMED®
scope of delivery	Storage cushion with PU cover, operating instructions
Indications / areas of application	<ul style="list-style-type: none"> • Pressure ulcer prophylaxis • Pressure ulcer therapy • Pain therapy
Contraindications	Not known
Permitted patient weight min. - Max.	Suitable for all patient weights.
material	<ul style="list-style-type: none"> • Filling: polypropylene microbeads • Cover: PU-coated PES carrier material
Dimensions	See label on the product
Product weight	0.2 - 1 kg
Aid number (as defined in §139 SGB V)	
Medical device	Class 1, MDR 2017/745 
Technical safety / flame retardancy	Flame retardancy test according to DIN EN 597 I + II, BS 7175 sec 3 crib 5, BS 7177
Testing for toxicological harmlessness / biocompatibility	DIN EN ISO 10993-5, DIN EN ISO 10993-1, DIN EN ISO 10993-10
Warnings	Before the first application, the system must be prepared for the first use (see "Preparing the system for use").
guarantee	5 years on the functional properties (pro-rata guarantee)
maintenance	No
Cleaning instructions (pillowcase)	<p>The cover can be disinfected by wiping with commercially available disinfectants made from isopropanol (e.g. Bacillo) or with other suitable DGHM-listed products (e.g. ECOLAB cleaning cloth Incidin OxyWipe S XL). The cover can be reprocessed by autoclaving according to the guidelines of the RKI.</p> 
Cleaning instructions (core)	The entire pillow can be reprocessed by autoclaving according to the guidelines of the RKI.
Prepare the system for use	<ol style="list-style-type: none"> 1. Remove the pillow from the box. 2. The system is now ready for use.

RKI = Robert Koch Institute

DHGM = German Society for Hygiene and Microbiology

MPG = Medical Devices Act

Autoclaving = fractional vacuum process at 105 degrees C and 55 minutes

 **Manufacturer in the sense of the MPG and MDR 2017/745**

Funke Medical AG • Ährenfeld 10 • 46348 Raesfeld • Tel. : + 49-2865-266 93 50 • Fax: + 49-2865-8133

info@funke-medical.de • www.funke-medical.de

Download: 02.07.2020 12:30

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Funke Medical AG • Ährenfeld 10 • 46348 Raesfeld • Tel.: +49-2865-266 93 50 • Fax: +49-2865-8133