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POSIMED®

POSIMED®

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POSIMED®

Positioning pillow

Please read the operating instructions carefully before using the system and heed the **warnings!**

Product name	POSIMED®
scope of delivery	Storage cushion with PU cover, operating instructions
Indications / areas of application	<ul style="list-style-type: none">• Pressure ulcer prophylaxis• Pressure ulcer therapy• Pain therapy
Contraindications	Not known
Permitted patient weight min. - Max.	Suitable for all patient weights.
material	<ul style="list-style-type: none">• Filling: polypropylene microbeads• Cover: PU-coated PES carrier material
Dimensions	See label on the product
Product weight	0.2 - 1 kg
Aid number (as defined in §139 SGB V)	
Medical device	Class 1, MDR 2017/745 
Technical safety / flame retardancy	Flame retardancy test according to DIN EN 597 I + II, BS 7175 sec 3 crib 5, BS 7177
Testing for toxicological harmlessness / biocompatibility	DIN EN ISO 10993-5, DIN EN ISO 10993-1, DIN EN ISO 10993-10
Warnings	Before the first application, the system must be prepared for the first use (see "Preparing the system for use").
guarantee	5 years on the functional properties (pro-rata guarantee)
maintenance	No
Cleaning instructions (pillowcase)	The cover can be disinfected by wiping with commercially available disinfectants made from isopropanol (e.g. Bacillo) or with other suitable DGHM-listed products (e.g. ECOLAB cleaning cloth Incidin OxyWipe S XL). The cover can be reprocessed by autoclaving according to the guidelines of the RKI. 
Cleaning instructions (core)	The entire pillow can be reprocessed by autoclaving according to the guidelines of the RKI.
Prepare the system for use	<ol style="list-style-type: none">1. Remove the pillow from the box.2. The system is now ready for use.

RKI = Robert Koch Institute

DHGM = German Society for Hygiene and Microbiology

MPG = Medical Devices Act

Autoclaving = fractional vacuum process at 105 degrees C and 55 minutes

 **Manufacturer in the sense of the MPG and MDR 2017/745**

Funke Medical AG • Ährenfeld 10 • 46348 Raesfeld • Tel. : + 49-2865-266 93 50 • Fax: + 49-2865-8133

info@funke-medical.de • www.funke-medical.de

Download: 02.07.2020 12:30

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Funke Medical AG • Ährenfeld 10 • 46348 Raesfeld • Tel.: +49-2865-266 93 50 • Fax: +49-2865-8133