
seca 704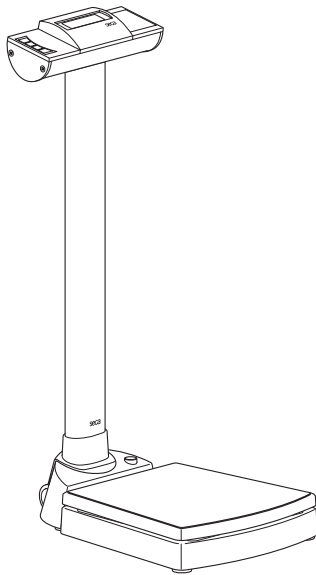

D	<i>Bedienungsanleitung und Garantieerklärung</i>	3
GB	<i>Instruction manual and guarantee</i>	15
F	<i>Mode d'emploi et garantie</i>	27
I	<i>Manuale di istruzioni e garanzia</i>	39
E	<i>Manual de instrucciones y garantía</i>	51
DK	<i>Betjeningsvejledning og garantibevis</i>	63
S	<i>Bruksanvisning och garanti</i>	75
N	<i>Bruksanvisning og garantierklæring</i>	87
FIN	<i>Käyttöohje ja takuu</i>	99
NL	<i>Bedieningshandleiding en garantieverklaring</i>	111
P	<i>Instruções de utilização e declaração de garantia</i>	123
GR	<i>Οδηγίες χειρισμού και εγγύηση</i>	135

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Full certification

In seca products you are purchasing not only technology matured over a century, but also quality confirmed by authorities, the legal system and by institutes.

seca products already comply with future statutory provisions. With seca, you are buying the future.

M

The products in these operating instructions comply with German legislation governing medical products and therefore with Directive 93/42/EEC of the Council of the European Community which is embodied in national laws throughout Europe.

Scales carrying this symbol are approved for medical purposes in the European Community. seca scales with this symbol meet the stringent quality and technical requirements placed on scales which can be calibrated.

Scales carrying this symbol are calibrated in accordance with Precision Class III of EC Directive 90/384/EEC.

Products carrying this symbol comply with the following directives and standards:

4. Directive 90/384/EEC governing non-automatic weighing instruments
5. Directive 93/42/EEC governing medical devices
6. DIN EN 45501 governing metrological aspects of non-automatic scales

seca's professionalism has also been officially recognized. The certificate from TÜV Product Service, the body responsible for medical devices, confirms that as a manufacturer of medical devices, seca rigorously complies with stringent legal requirements. seca's quality assurance system covers the development, production, sales and service of medical scales and measuring systems and ergometry.

seca helps the environment.

We are concerned to save natural resources. We therefore strive to save packaging material where meaningful. What is left over can conveniently be disposed of locally via the German Dual System (recycling programme).

1. Congratulations!

By purchasing the **seca 704** electronic personal scale, you have acquired a highly-accurate and sturdy piece of equipment.

seca has been putting its experience at the service of health for over 150 years now, and as market leader in many countries of the world, is always setting new standards with its innovative developments for weighing and measuring.

In accordance with national regulations, the **seca 704** electronic personal scale is used primarily in hospitals, doctors' surgeries and in-patient nursing facilities.

The scales are a Class I medical product, and have been calibrated in accordance with the precision class III.

In addition to the conventional determination of weight, the **seca 704** also has a function for determining body mass index. For this purpose, height is entered on the keypad and the body mass index associated with the weight value is output.

The **seca 704** can be moved on castors, and its low power consumption means that it can be used on a mobile basis with batteries for long periods.

2. Safety

Before using the new scale, please take a little time to read the following safety instructions.


- Follow the safety instructions in the instruction manual.
- Keep the operating manual and the declaration of conformity contained therein in a safe place.
- Do not drop the scale or subject it to violent shocks.
- Have scale serviced and re-calibrated on a regular basis (see "Servicing/recalibration" on page 24).
- Have repairs carried out exclusively by authorised persons.
- Use only the type of battery stated (see "Changing the batteries" on page 24).
- When using the scale with a mains unit, ensure that the supply cable is routed in such a way as to exclude any type of tripping hazard.

3. Before you start...

Unpacking

The scope of supply includes:

- Weighing platform
- Column with display head
- Column socket
- 1 WAF 5 Allen key
- 2 Screws
- Battery holder containing 6 mignon-type AA 1.5 V batteries
- 1 set of operating instructions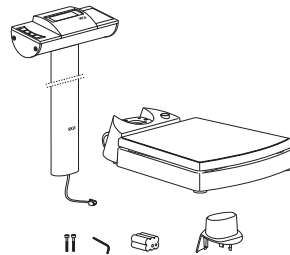


Fitting the column

The column with the LCD display can be fitted in two positions, allowing you to determine in which direction the display is legible - facing either towards or away from the platform.

- Push the column socket onto the column.
- Place the column in the recess, facing it in the desired direction.
- Screw the column to the weighing platform from underneath.

If necessary, lay the scale carefully on its side to tighten up the screws.

- Connect the plugs of the mains supply cable to the underside of the weighing platform as shown in the illustration.
- Plug the plug of the display cable into the socket on the underside of the weighing platform.
- Fix the cables in the cable guide.
- Then place the scale upright again.
- Push the column socket right down until it engages in the weighing platform.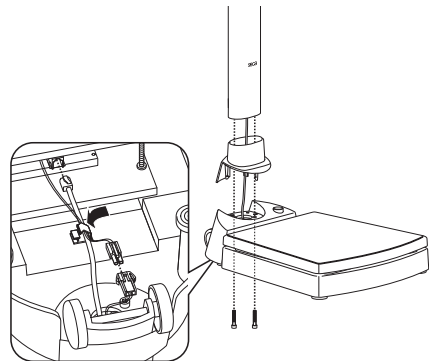

Power supply

The scale is generally powered by batteries.



The LC display has a low power consumption – 16.000 weighing operations can be performed with one set of batteries.

- Push the battery compartment closure towards “Battery” and open the cover.
- Connect the cable to the holder and batteries.
- Place the holder in the battery compartment.
- Close the cover.

Note: the scale is prepared for use with a seca mains unit which is available as an accessory.

- The socket for connecting the mains unit is located at the bottom of the column.

Note: the mains unit can also be used with the batteries in place without the scale or batteries being damaged.

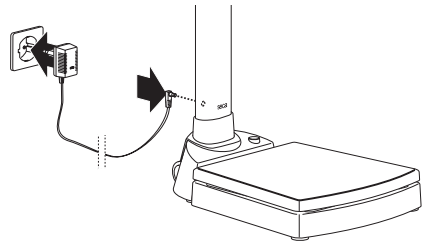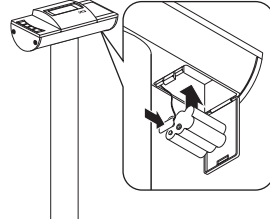4. Setting up and aligning the scale

- Place the scale on a level surface.
- Align the scale by undoing the foot screws. The air bubble in the spirit level must be right in the centre of the circle.
- Ensure that only the feet of the scale are in contact with the floor. The scale may not be in contact at any other point.

Important!

The alignment of the base of the scale must be checked and corrected if necessary every time the location of the scale is changed.

5. Operation

Controls and displays

The controls and display are in the display part.

Switch scale on and off

in normal operation → no function
in function menu → increase value

press briefly → activate/deactivate hold function
hold down → set display to zero, activate/deactivate tare function
in function menu → decrease value

Activating the BMI function, Confirmation

Checking that the contents of the calibration counter are correct

This seca scale is calibrated. Calibration may be performed only by authorised centres. To ensure this, the scale is fitted with a calibration counter which records any change to data relevant to calibration. If you want to check whether the scale is properly calibrated, proceed as follows:

- If necessary, switch off the scale.
- Keep any key depressed and start the scale.

The current contents of the calibration counter are shown on the display for a few seconds.

- Compare the output contents of the calibration counter with the number shown on the calibration counter sticker.

For valid calibration, both numbers must match. If sticker and calibration counter do not match, then re-calibration is required. Contact your service agent or seca Customer Service.

If a supplementary calibration is necessary, then the supplementary calibration mark illustrated opposite is used to record the calibration counter position instead of the seca calibration mark shown above. This mark is secured with an additional seal by the person authorised to perform the supplementary calibration. The supplementary calibration mark is available from seca Customer Service under the number 14-05-01-886.

Both numbers must match.

Correct weighing

- Press the Start key.
SECA, **88888** and **00** appear consecutively in the display. The scale is then automatically set to zero and ready to use.
- Stand on the scale and keep still.
- Read the weighing result off the digital display.
- If a load greater than 250 kg is placed on the scale, **STOP** appears in the display.
- To switch off the scale, press the Start key again.

Automatic switch-off/economy mode

The scale automatically detects whether it is being operated on rechargeable batteries alone or whether it is connected to the mains.

- With rechargeable battery operation: automatic shut-down after 30 seconds.
- With mains operation: shut-down only via keypad.

Taring weight display (TARE)

The Tare function is useful if an extra weight is to be discounted in a weighing operation.

Proceed as follows:

- Press the green Start key.
- First place the extra weight on the scale and activate the Tare function by keeping the **HOLD/TARE** key depressed. The display is reset to zero. **0.0** appears in the display, together with the word "NET".
- Now put the patient on the scale with the extra weight previously weighed. The scale now shows the weight of the patient without the extra load. You can now perform as many additional weighing operations as you like.
- Deactivate the Tare function by keeping the key depressed again; the scale returns to normal weighing mode and the tare value (the value of the extra weight) is erased.
- The Tare value can also be erased by the scale being switched off.

Store weight value (HOLD)

You can also continue displaying the weight determined once the load has been removed. This makes it possible to look after a patient first, before having to note the weight:

- Press the **HOLD/TARE** key whilst the load is on the scale.

“HOLD” appears in the digital display. The value remains saved in the display until the hold function is automatically switched off.

The triangle with the exclamation mark indicates that this frozen display is an additional facility which is not subject to calibration.

- The HOLD function can be switched off by pressing the **HOLD** key again.

00 appears in the display if the scale has no load on it, otherwise the current weight. The “HOLD” display vanishes.

Note:

The HOLD and TARE functions can also be used simultaneously.

Body Mass Index (BMI)

Body Mass Index puts height and weight in relationship to one another, providing more accurate information than ideal Broca weight, for example. A tolerance range is given which is considered ideal for health. The Body Mass Index applies only to adults: these values are not relevant to children and young people.

- Press the Start key.
SELR, 88888 and **00** appear consecutively in the display. The scale is then automatically set to zero and ready for operation.

- Press the **FUNC/SET** key.
On the display, you will see the last height entered.

- You can use the arrow keys to adjust the value in increments according to the graduations displayed.

- Once the correct value is set, press the **FUNC/SET** key again. The BMI function is now activated.

- Step on the scale and keep still.
- Read off the BMI on the digital display and compare it with the categories on page 23.

- Press an arrow key if you would like to see the weight briefly.

- To return to normal weighing mode, press the **FUNC/SET** key.
- To switch off the scale, press the Start key again.

Evaluating Body Mass Index:

Compare the value determined with the categories quoted below which correspond to those used by WHO, 2000.

- **BMI below 18.5:**
Warning, this patient weighs too little. There could be a tendency to anorexia. An increase in weight is recommended to improve well-being and performance. If in doubt, consult a specialist.
- **BMI between 18.5 and 24.9:**
The patient is of a normal weight. He can stay as he is.
- **BMI between 25 and 30 (pre-obese):**
The patient is slightly to moderately overweight. He should cut his weight if he is already ill (e.g. diabetes, hypertension, gout, fat metabolism disorders).
- **BMI over 30:**
Weight reduction is essential. Metabolism, cardiovascular system and bones are all under strain. A consistent diet, plenty of movement and behavioural training are all recommended. If in doubt, consult a specialist.

6. Cleaning

Clean the rubber surface and the housing as required using a domestic cleaning agent or commercially-available disinfectant. Follow the manufacturer's instructions.

Under no circumstances use abrasive or acid cleaners, white spirit, benzene or the like for cleaning. Such substances can damage the high-quality surfaces.

7. What do I do if...

... no weight display comes on?

- Is the scale switched on?
- Check the batteries.
- Is the plug contact for connecting the display to the scale correctly connected?

... does not appear before weighing?

- Press the Start key again – there must not be any load on the scale – and only its feet should be in contact with the floor.

... one segment is illuminated constantly or not at all?

- The relevant segment has a fault. Inform Servicing.

... the display shows  ?

- Battery voltage is running low. Change the batteries in the next few days.

... *bAEE* appears in the display?

- Batteries are empty. Put in new batteries.

... *SEOP* appears in the display?

- Maximum load has been exceeded.

... the display flashes?

- If you have just activated the Hold or Tare function, please wait until the scale has detected a steady weight value. The display will then stop flashing.
- If there has been no activation of a function beforehand, remove the load from the scale and wait until *0.0* is displayed, then weigh again.

... the display *TEMP* appears?

- The ambient temperature of the scale is too high or too low. Place the scale in an ambient temperature between +10 °C and +40 °C. Wait about 15 minutes for the scale to adapt to the ambient temperature and then weigh again.

... the display *E* and a number appear?

- Switch off the scale with the Start key and start the scale again. The scale will then work normally again.
If this is not the case, disconnect the power supply by briefly removing the batteries. If this measure is equally unsuccessful, inform the Service department.

8. Servicing/recalibration


Have the scale recalibrated by authorised persons in accordance with national legal requirements. The year of initial calibration can be found behind the CE symbol on the rating plate above the number of the appointed office, 0108 (Hamburg Calibration Office). Recalibration is essential if one or more security marks are damaged or the calibration counter does not match the number on the applicable calibration counter mark (cf. page 20).

We recommend having your scale serviced by your local service agent before having it recalibrated. seca Customer Service will be pleased to give you further assistance.

Changing the batteries

When  or *bAEE* appear in the display, you should change the batteries. You need 6 type AA 1.5 V batteries.

- Push the battery compartment closure towards “Battery” and open the cover.
- Take out the holder and batteries.
- Remove the used batteries and insert new batteries in the holder.

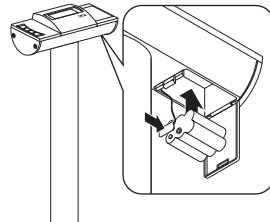Check that the polarity is correct.

- Place the holder and batteries in the battery compartment.
- Close the cover.

9. Technical data

Dimensions scale

width:	360 mm / 14.2 ins
height:	930 mm / 36.6 ins
depth:	520 mm / 20.5 ins

Dimensions weighing platform

width:	360 mm / 14.2 ins
height:	84 mm / 3.3 ins
depth:	360 mm / 14.2 ins

Height of figures

20 mm

Weight

approx. 16 kg

Temperature range

+10 °C to +40 °C

Batteries

6 type AA 1.5 V batteries

Power supply

battery / mains unit (option)

Maximum load

250 kg

Minimum load

2 kg

Graduations

100 g

Calibration

approved for medical use, Class III

Medical product as per Directive 93/42/EEC

Class I

Accuracy at first calibration

to 50 kg:	± 50 g
50 kg to 200 kg:	± 100 g
200 kg to 250 kg:	± 150 g

Functions

Tare, Hold, Body Mass Index

10. Spare parts and accessories

Measuring rods

Material: hard-anodised aluminium, slide with lockable measuring tab

seca 220 telescopic measuring rod

seca model 220-17-14-004:
 Measuring range: 820 - 2000 mm,
 Graduations: 1 mm

seca model 220-18-14-004
 (double graduations):
 Measuring range: 820 - 2000 mm and
 32 1/4 " - 78 3/4 ",
 Graduations: 1 mm and 1/8 "

Power supply

Model dependent mains unit

Plug-in mains unit, Euro: 230V~ / 50Hz / 12V= / 130mA

seca spare part no.: 68-32-10-252

Desk-top mains unit, switchable, Euro: 115V~ / 230V~ / 50-60Hz / 9V= / 350mA

seca spare part no.: 68-32-10-243

Plug-in mains unit, USA: 120V~ / 60Hz / 12V= / 150mA

seca spare part no.: 68-32-10-259

Plug-in mains unit

Use only seca original plug-in mains units with 9 V or controlled 12 V output voltage. Commercially-available mains units may supply a higher voltage than stated and thus lead to damage to the scale.

Recalibration

Supplementary calibration mark seca spare part no.: 14-05-01-886

11. Disposal

Batteries

Do not throw used batteries away in domestic waste. Dispose of batteries at collection points in the vicinity. When buying new batteries, select those low in harmful substances and containing no mercury (Hg), cadmium (Cd) or lead (Pb).

Disposing of the scale

If the scale can no longer be used, the responsible waste disposal association will be glad to advise you on the measures necessary to dispose of it properly.

12. Warranty

A two-year warranty from date of delivery applies to defects attributable to poor materials or workmanship. All moveable parts - batteries, cables, mains units, rechargeable batteries etc. - are excluded. Defects which come under warranty will be made good for the customer at no charge on production of the receipt. No further claims can be entertained. The costs of transport in both directions will be borne by the customer should the equipment be located anywhere other than the customer's premises. In the event of transport damage, claims under warranty can be

honoured only if the complete original packaging was used for any transport and the merchandise secured and attached in that packaging just as it was when originally packed. All the packaging should therefore be retained.

A claim under warranty will not be honoured if the equipment is opened by persons not expressly authorised by seca to do so.

We would ask our customers abroad to contact their local sales agent in the event of a warranty matter.

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