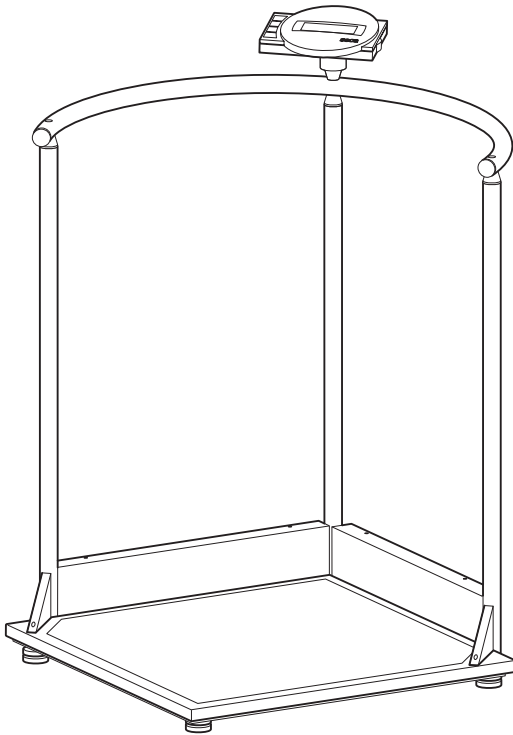
# seca 645

<b>D</b>	<b><i>Bedienungsanleitung und Garantieerklärung</i></b>	<b>3</b>
<b>GB</b>	<b><i>Instruction manual and guarantee</i></b>	<b>23</b>
<b>F</b>	<b><i>Mode d'emploi et garantie</i></b>	<b>43</b>
<b>I</b>	<b><i>Manuale di istruzioni e garanzia</i></b>	<b>63</b>
<b>E</b>	<b><i>Manual de instrucciones y garantía</i></b>	<b>83</b>
<b>DK</b>	<b><i>Betjeningsvejledning og garantibevis</i></b>	<b>103</b>
<b>S</b>	<b><i>Bruksanvisning och garanti</i></b>	<b>123</b>
<b>N</b>	<b><i>Bruksanvisning og garantierklæring</i></b>	<b>143</b>
<b>FIN</b>	<b><i>Käyttöohje ja takuu</i></b>	<b>163</b>
<b>NL</b>	<b><i>Bedieningshandleiding en garantieverklaring</i></b>	<b>183</b>
<b>P</b>	<b><i>Instruções de utilização e declaração de garantia</i></b>	<b>203</b>
<b>GR</b>	<b><i>Οδηγίες χειρισμού και εγγύηση</i></b>	<b>223</b>

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# Full certification

---

seca products you are purchasing not only technology matured over a century, but also quality confirmed by authorities, the legal system and by institutes.

seca products already comply with future statutory provisions. With seca, you are buying the future.

**M**

The products in these operating instructions comply with German legislation governing medical products and therefore with Directive 93/42/EEC of the Council of the European Community which is embodied in national laws throughout Europe.

Scales carrying this symbol are approved for medical purposes in the European Community. seca scales with this symbol meet the stringent quality and technical requirements placed on scales which can be calibrated.

Scales carrying this symbol are calibrated in accordance with Precision Class III of EC Directive 90/384/EEC.

Products carrying this symbol meet the requirements

1. of the Directive for non-automatic scales in medicine, No. 90/384/EEC
2. for the law governing medical products, EC Directive No. 93/42/EEC
3. the Directive on electromagnetic compatibility

seca has been awarded the EN ISO 9001 certificate, which means that our company adheres to strict specifications for consistent high quality in research, production, sales and service. Furthermore, EN 46001 certifies seca's rigorous compliance with the most stringent European quality standards for medical products.

seca helps the environment.

We are concerned to save natural resources. We therefore strive to save packaging material where meaningful. What is left over can conveniently be disposed of locally via the German Dual System (recycling programme).

# 1. Congratulations!

---

In the **seca 645** scale with standing aid, you have purchased a high-precision yet sturdy piece of equipment.

seca has been putting its experience at the service of health for over 150 years now, and as market leader in many countries of the world, is always setting new standards with its innovative developments for weighing and measuring.

In accordance with national regulations, the **seca 645** scale with standing aid is used primarily in hospitals, doctors' surgeries and in-patient nursing facilities. The scales are a Class I medical product, and have been calibrated in accordance with the precision class III.

The **seca 645** has two weighing ranges which can be selected. Weighing range 1 has a reduced maximum capacity but greater display resolution, whilst weighing range 2 allows you to use the maximum load capacity of the scale. Weight is determined within a few seconds.

In addition to the conventional determination of weight, the **seca 645** also has a function for determining body mass index. For this purpose, height is entered on the keypad and the body mass index associated with the weight value is output.

You can use the Pre-Tare function to store the weight of a chair, for example. This weight is automatically deducted in the weighing operation.

The large, secure standing aid also enables persons who are unsteady on their feet to stand safely on the scale.

The scale is of a very sturdy construction and will give you long and loyal service. It is easy to operate and the large display is easy to read.

The **seca 645** can be moved on castors and its low power consumption combined with its rechargeable batteries mean it will provide long, mobile service.

# 2. Safety

---

Before using the new scale, please take a little time to read the following safety instructions.

- Follow the instructions in the instructions for use.
- Keep the operating manual and the declaration of conformity contained therein in a safe place.
- Take measures to ensure that persons who have difficulty walking do not fall over when using the scale.
- The scale must be used with care. The upper surface may be slippery if wet!
- After moving the scale, please lower the scale slowly to the operating height.
- Do not drop the scale or subject it to violent shocks.
- Be sure to use only the seca mains unit supplied. Before plugging the mains unit into a socket, check that the mains voltage indicated on the mains unit matches local mains voltage.
- When using the scale with a mains unit, ensure that the supply cable is routed in such a way as to exclude any type of tripping hazard.
- Have scale serviced and re-calibrated on a regular basis (see „Maintenance“ on page 38).
- Have repairs carried out only by authorised persons.

# 3. Before you start...

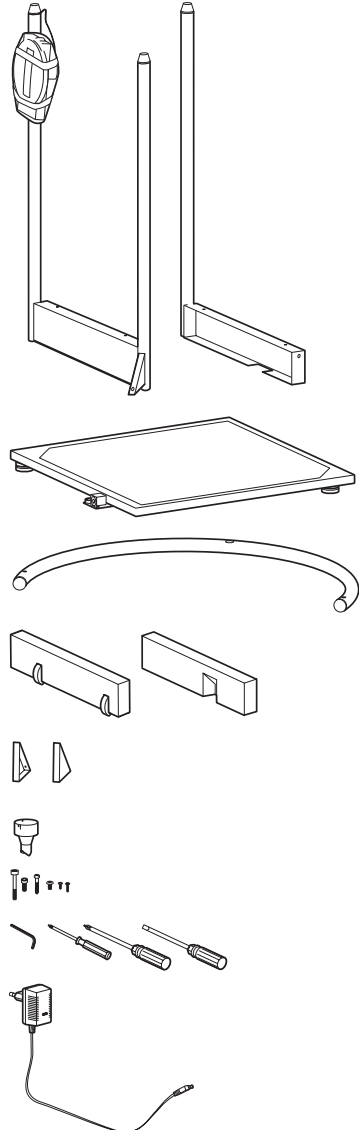
---

## Unpacking

- Remove the packaging, although leave the display head attached to the column. Place the scale platform on a solid, flat surface.

The scope of supply includes:


- Side section with two columns and the display head attached for transport
- Side section with one column
- Scale platform
- Railing
- Cover for the wheel casing
- Cover for the side casing
- 2 triangular covers right/left
- Display mount
- Set of assembly bolts
- Allan key, size 6
- Phillips screwdriver size 1 and size 3
- 1 screwdriver
- 1 seca mains unit
- 1 set of operating instructions



## Assembling the scale

### Step 1: Assemble the first side section

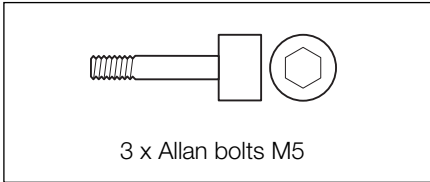You will need the following bolts:

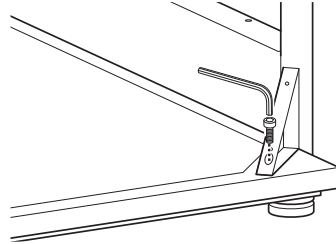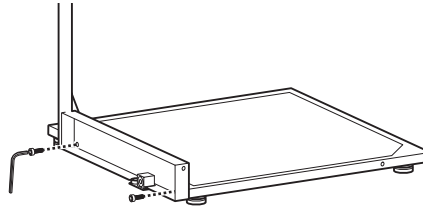
- Position the side section on the right-hand side of the scale platform.
- Bolt the side section to the scale platform using two Allan bolts.
- Use an Allan bolt to attach the triangular reinforcement to the scale platform.


### Step 2: Assemble the side section with the wheel casing

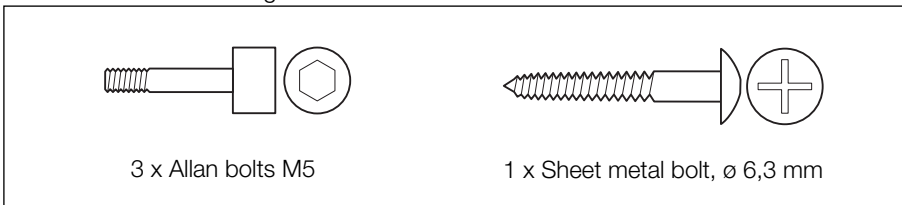You will need the following bolts:


- Position the side section with the wheel casing on the rear side of the scale platform.
- Bolt the side section to the scale platform using two Allan bolts.

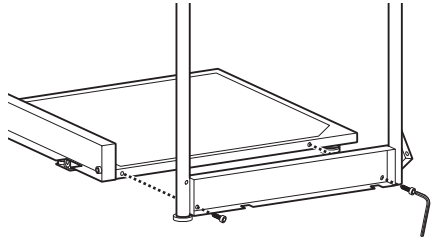- Use an Allan bolt to attach the triangular reinforcement to the scale platform.

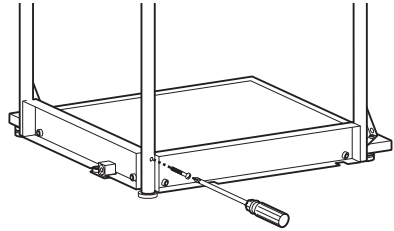
- Use the sheet metal bolt to attach the two side sections to the central column.


### Step 3: Assemble the railing

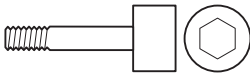You will need the following bolts:


1 x Allan bolts M6

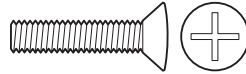
2 x Oval head bolt M6

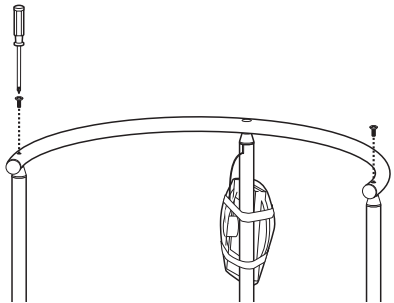- Position the railing on the three columns. The large bore holes must point downwards, and must be inserted into the heads of the columns.
- Use an oval head bolt to attach the railing to the two outer columns.

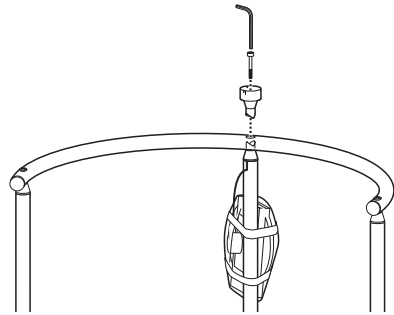
- Place the display mount with the peg-shaped end over the central column on the railing.
- Turn the display mount until the peg clicks into place.
- Screw the display mount tightly into place using the Allan bolt.


#### Step 4: Assemble the display head

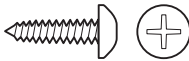You will need the following bolts:

3 x special Allan bolt,  $\varnothing$  2,5 mm

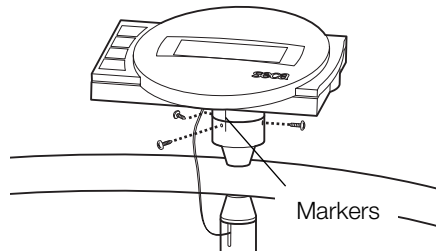- Remove the transport protection from the display head.
- Position the display head over the display mount in such a way that the position markers are aligned.
- Use the three Allan bolts to attach the display head to the display mount.



## Step 5: Assemble the side section covers

You will need the following bolts:



16 x Phillips bolts,  $\varnothing$  3,5 mm

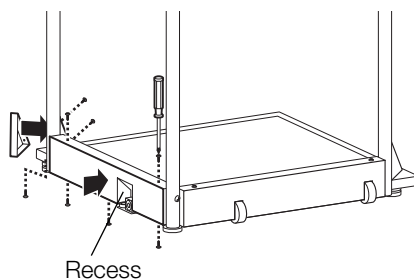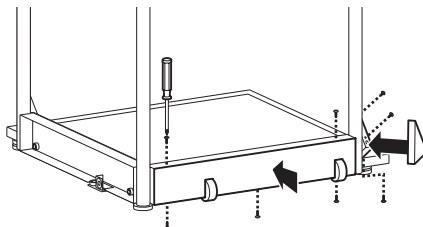- Place the scale on its side, with the rear side section facing upwards.
- Position the cover with the wheels over the side section, and attach this securely using 5 Phillips bolts: 3 x from below, and 2 x from above.
- Position the cover on the reinforcing triangle, and bolt this firmly in place using three bolts.
- Now turn the scale over so that the other side faces upwards.
- Position the cover with the recess pointing inwards over the side section, and attach this securely using 5 Phillips bolts: 3 x from below, and 2 x from above.
- Position the cover on the reinforcing triangle, and bolt this firmly in place using three bolts.

## Step 6: Assemble the electrical connections

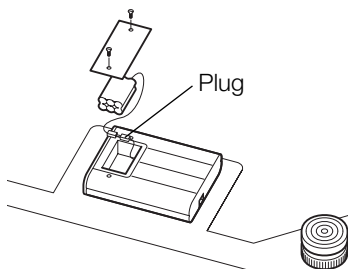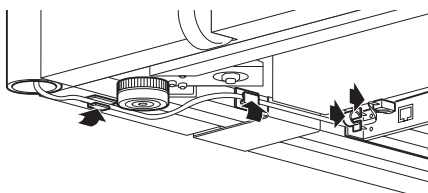- Connect the cable which comes out of the central column to the electronic box socket. In order to prevent measurement errors, lay the cable as shown in the diagram.
- Unscrew the bolts which secure the lid of the battery compartment, and remove.
- Connect the plug with the rechargeable battery block located in the battery compartment.
- Replace the lid of the battery compartment, and bolt this firmly in place.
- Turn the scale to the upright position.


## Power supply

The scale is powered by a mains unit and by rechargeable batteries. The scale works independently of the mains, which is very handy for mobile use.

The LC display and the evaluation electronics have a low power consumption – 2,000 weighing operations can be performed with a fully charged battery. Once battery voltage is no longer adequate, the battery symbol  first appears in the display to indicate that battery power is dropping. If the batteries are not charged, battery voltage will ultimately not be adequate for error-free measuring. In this case, **bAtte** appears in the display and weighing is no longer possible.

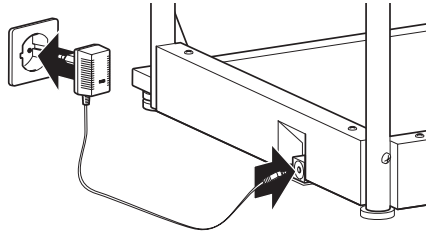
The rechargeable batteries are charged by the mains unit supplied. For this purpose, the mains unit needs to be connected to the mains for at least 24 hours.

## 4. Setting up and aligning the scale

---

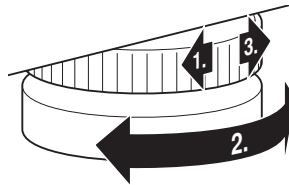- Place the scale on a level surface.
- Undo the locknuts using the four foot screws (1).

- Align the scale by undoing the foot screws. The air bubble in the spirit level must be right in the centre of the circle (2).
- Then re-tighten the locknuts (3).
- Ensure that only the feet of the scale are in contact with the floor. The scale may not be in contact at any other point.

spirit level


### Important!

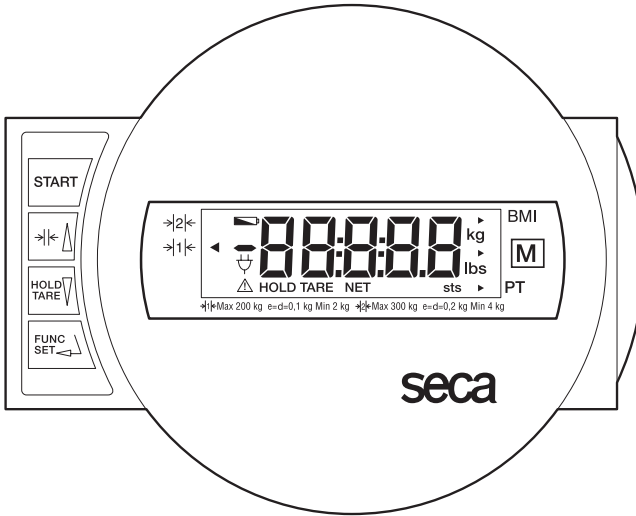**The alignment of the base of the scale must be checked and corrected if necessary every time the location of the scale is changed.**

# 5. Operation

## Controls and displays

The controls and the display are on the display head on the column.

Switch scale on and off

press briefly → switch between weighing range 1 and 2  
in function menu → increase value, scroll

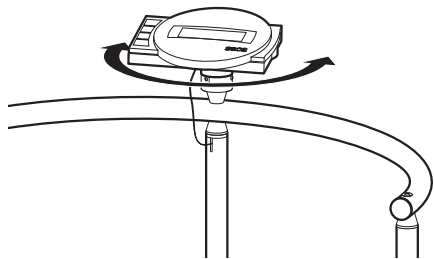
press briefly → activate hold function  
hold down → set display to zero, activate/deactivate tare function  
in function menu → decrease value, scroll

Activate BMI or pre-Tare function, and confirm key for entries

In order to ensure that the display can be read from different positions, the display head can be rotated 180° in both directions.

## Checking that the contents of the calibration counter are correct

This seca scale is calibrated. Calibration may be performed only by authorised centres. To ensure this, the scale is fitted with a calibration counter which records any change to data relevant to calibration. If you want to check whether the scale is properly calibrated, proceed as follows:

- If necessary, switch off the scale.
- Keep any key depressed and start the scale.

The current contents of the calibration counter are shown on the display for a few seconds.

- Compare the output contents of the calibration counter with the number shown on the calibration counter sticker.

For valid calibration, both numbers must match. If sticker and calibration counter do not match, then re-calibration is required. Contact your service agent or seca Customer Service.

**Both numbers must match.**

If a supplementary calibration is necessary, then the supplementary calibration mark illustrated opposite is used to record the calibration counter position instead of the seca calibration mark shown above. This mark is secured with an additional seal by the person authorised to perform the supplementary calibration. The supplementary calibration mark is available from seca Customer Service under the number 14-05-01-886.

## Correct weighing

- Press the Start key.  
**SECA**, **88888** and **000** appear consecutively in the display. The scale is then automatically set to zero and ready to use.
- Switch weighing range if necessary (see page 33).
- Ask the patient to mount the scale. Once on the scale, the patient should stand still.
- Read the weighing result off the digital display.
- If a load greater than 200 kg or 300 kg is placed on the scale, **StOP** appears in the display.
- To switch off the scale, press the Start key again.

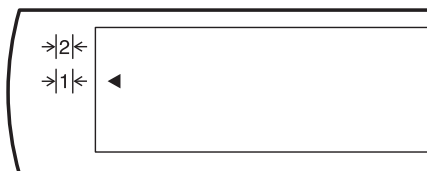
## Switching the weighing range

The **seca 645** has two weighing ranges which can be selected. Weighing range 1 has a reduced maximum capacity but greater display resolution, whilst weighing range 2 allows you to use the maximum load capacity of the scale. Depending on the application in question, you decide whether you would prefer to use the fine display resolution or the high load range.

- Switch on the scale at the green Start key. The scale initially operates in the weighing range last selected.
- To change the weighing range, press the key  $\rightarrow|<$ .
- A triangle in the display indicates the weighing range active in each case. This remains active even when the scale is turned off.

### Note:

It is possible to switch from Range 1 with fine graduations to Range 2 with coarse graduations with a load on the scale unless the scale is switched to a special function (like pre-Tare, for example). For technical reasons, it is only possible to switch from coarse Range 2 to fine Range 1 with no load on the scale. In this case too, no special function may be activated.

## Automatic switch-off/economy mode

The scale automatically detects whether it is being operated on rechargeable batteries alone or whether it is connected to the mains.

- With rechargeable battery operation: automatic shut-down after 5 minutes.
- With mains operation: shut-down only via keypad.

## Taring weight display (TARA)

The Tare function is very useful if an additional weight, e.g. that of a chair, is to be ignored for weighing purposes.

Proceed as follows:

- Press the green Start key.
- First place the additional weight (e.g. the chair) on the scale and activate the Tare function by pressing and holding the **HOLD/TARE** key.

The display is reset to zero. **0.0** appears in the display. The note “NET” appears.

- Now place the patient on the scale. The patient may remain seated.

The scale determines the patient's weight. You can now perform as many weighing operations as you like.

Hold the key down again to deactivate the Tare function; the scale is then back in normal weighing mode and the tare value has been erased.

- The tare value is likewise erased by the scale being switched off.

## Store weight value (HOLD)

You can also continue displaying the weight determined once the load has been removed. This makes it possible to look after a patient first, before having to note the weight.

- Press the **HOLD/TARE** key whilst the load is on the scale.

“HOLD” appears in the digital display. The value is stored until the scale switches off automatically.

The triangle with the exclamation mark indicates that this frozen display is an additional facility which is not subject to calibration.

- The HOLD function can be switched off by pressing the **HOLD** key again.

**0.0** appears in the display if the scale has no load on it, otherwise the current weight. The “HOLD” display vanishes.

### Note:

**The HOLD and TARE functions can also be used simultaneously.**

## Body-Mass-Index (BMI)

Body Mass Index puts height and weight in relationship to one another, providing more accurate information than ideal Broca weight, for example. A tolerance range is given which is considered ideal for health. The Body Mass Index applies only to adults: these values are not relevant to children and young people.

- Press the Start key.  
**SECA**, **88888** and **00** appear consecutively in the display. The scale is then automatically set to zero and ready for operation.

- Press the **FUNC** key.
- On the display, you will see the last height entered.

- Use the arrow keys to select the BMI function **FU 1**. Activate the function by pressing the **FUNC** key once again.  
The patient's recorded height will then appear on the display.

- You can change the value in 1 cm increments using the arrow keys.

- Once the correct value is set, press the **FUNC** key again. The BMI function is now activated.

- Ask the patient to mount the scale. Once on the scale, the patient should stand still.

- Read off the BMI on the digital display and compare it with the categories on page 36.

- To return to normal weighing mode, press the **FUNC** key.

- To switch off the scale, press the Start key again.

## Evaluating Body Mass Index:

Compare the value determined with the categories shown below.

- **BMI below 18:**

Warning, this patient weighs too little. There could be a tendency to anorexia. An increase in weight is recommended to improve well-being and performance. If in doubt, consult a specialist.

- **BMI between 18 and 20:**

Patient is at lower weight limit. If he feels well and is completely healthy, his weight can stay as it is.

- **BMI between 20 and 25:**

The patient is of a normal weight. He can stay as he is.

- **BMI between 25 and 30:**

The patient is slightly to moderately overweight. He should cut his weight if he is already ill (e.g. diabetes, hypertension, gout, fat metabolism disorders).

- **BMI over 30:**

Weight reduction is essential. Metabolism, cardiovascular system and bones are all under strain. A consistent diet, plenty of movement and behavioural training are all recommended. If in doubt, consult a specialist.

## Weighing with pre-Tare

If the pre-Tare function is activated, a stored weight value is deducted from the currently measured weight. This function can be useful, for example, when the patient and the carer mount the scale together, and the weight of the carer is not meant to be shown.

- Press the Start key.

**SECA**, **888888** and **0.00** appear consecutively in the display. After that, the scale is automatically set to zero and ready for operation.

- Press the **FUNC** key.

The last function used appears in the display.

- Use the arrow keys to select the pre-Tare function **FU 3**. Activate the function by pressing the **FUNC** key once again. In the display you will see the tare value from the memory.

- You can alter the tare value in display steps using the arrow keys.
- Once the correct value is set, press the **FUNC** key again.

The pre-Tare function is now activated.

- Mount the scale together with the patient. Read the weighing result off the digital display.

- To return to normal weighing mode, press the **FUNC** key.
- To switch off the scale, press the Start key again.

## Setting damping

Damping is useful for reducing weight determination malfunctions (due to patient movements, for example). The stronger the damping, the more slowly the weight display reacts. You can set damping to suit requirements.

- Press the **FUNC** key and keep it depressed for 2 seconds.

**FIL** and the current setting appear in the display:

- 0 = slight damping  
→ fast determination of weight
- 1 = average damping  
→ normal determination of weight
- 2 = strong damping  
→ slow determination of weight

- Press an arrow key if you want to change the set value.

- Press the **FUNC** key again to store the displayed value. The scale is then back in normal weighing mode.

**The settings are retained even once the scale is switched off.**

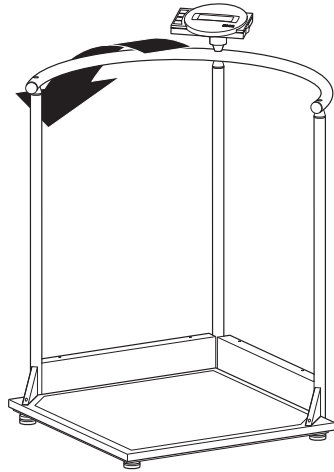
## 6. Moving the scale

---

For mobile deployment in a ward, the scale can be tipped backwards and moved about on the rollers.

### **Important!**

**The alignment of the base of the scale must be checked and corrected if necessary every time the location of the scale is changed.**

## 7. Maintenance

---

Clean the rubber surface and the housing as required using a domestic cleaning agent or commercially-available disinfectant. Follow the manufacturer's instructions.

Under no circumstances use abrasive or acid cleaners, white spirit, benzene or the like for cleaning. Such substances can damage the high-quality surfaces.

## 8. What do I do if...

---

### **... no weight display comes on?**

- Is the scale switched on?
- Check the mains connection and the connection of the battery pack.
- Is the plug contact for connecting the display to the scale correctly connected?
- Switch the scale off for a few seconds. This can be done by opening the plug connected to the rechargeable battery block and unplugging the mains supply from the scale.

### **... does not appear before the weighing operation?**

- Press the Start key again – there must not be any load on the scale – and only its feet should be in contact with the floor.

### **... one segment is illuminated constantly or not at all?**

- The relevant segment has a fault. Inform Servicing.

### **... the display shows ?**

- Battery voltage is dropping. Charge the rechargeable batteries in the next few days.

**... bRLt appears in the display?**

- The rechargeable battery is so severely discharged that weighing is no longer possible. Recharge the battery.

**... 5t0P appears in the display?**

- Maximum load has been exceeded.

**... the display flashes?**

- If you have just activated the Hold or Tare function, please wait until the scale has detected a steady weight value. The display will then stop flashing.
- If there has been no activation of a function beforehand, remove the load from the scale and wait until 0.0 is displayed, then weigh again.

**... the display shows tENP?**

- The ambient temperature of the scale is too high or too low. Place the scale in an ambient temperature between +10 °C and +40 °C. Wait about 15 minutes for the scale to adapt to the ambient temperature and then weigh again.

**... the display E and a number appear?**

- Switch off the scale with the Start key and start the scale again. The scale will then work normally again.  
If this is not the case, interrupt the power supply by briefly removing the rechargeable battery. If this measure is equally unsuccessful, inform the Service department.

**... the weighing range won't switch?**

- Take the load off the scale if necessary.
- Ensure that no special function is active.

## 9. Maintenance

---

Have recalibration performed in accordance with national regulations. Servicing and recalibration may be performed solely by authorised staff. Contact your service agent or seca Customer Service.

# 10. Technical data

---

<b>Dimensions: (width, length, height)</b>	750 x 750 x 1140 mm / 30 x 30 x 45 inch
<b>Size of platform</b>	600 x 600 mm / 23.6 x 23.6 inch
<b>Height of platform</b>	45 mm / 1.77 inch
<b>Weight</b>	approx. 24 kg / 11 lbs
<b>Temperature range</b>	+10 °C to +40 °C
<b>Calibration</b>	approved for medical use, Class III
<b>EN 60 601-1:</b>	
insulated equipment, Protection Class II	
medical electrical equipment type B	
<b>Medical product as per Directive 93/42/EEC</b>	Class I
<b>Colour</b>	white/black
<b>Maximum load</b>	
Weighing range 1	200 kg
Weighing range 2	300 kg
<b>Minimum load</b>	
Weighing range 1	2 kg
Weighing range 2	4 kg
<b>Graduations</b>	
Weighing range 1	100 g
Weighing range 2	200 g
<b>Accuracy at first calibration</b>	
Weighing range 1:	
to 50 kg:	± 50 g
50 kg to 200 kg:	± 100 g
Weighing range 2:	
0 to 100 kg:	± 100 g
100 kg to 300 kg:	± 200 g
<b>Power supply:</b>	model dependent mains unit and built-in rechargeable battery pack

# 11. Spare parts

---

## Rechargeable batteries

seca spare part no.: 68-22-12-721

## Model dependent mains unit

Description: plug-in mains unit  
230 V~/50 Hz/12 V~/130 mA

seca spare part no.: 68-32-10-252

Description: desk-top mains unit, switchable  
115 V~/230 V~/50 Hz/9 V~/350 mA

seca spare part no.: 68-32-10-243

## Supplementary calibration mark

seca spare part no.: 14-05-01-886

# 12. Disposal

---

## Rechargeable batteries

Do not throw old rechargeable batteries into the domestic refuse! Ensure that rechargeable batteries are properly disposed of at local collection points.

## Disposing of the scale

If the scale can no longer be used, the responsible waste disposal association will be glad to advise you on the measures necessary to dispose of it properly.

# 13. Warranty

---

A two-year warranty from date of delivery applies to defects attributable to poor materials or workmanship. All moveable parts - batteries, cables, mains units, rechargeable batteries etc. - are excluded. Defects which come under warranty will be made good for the customer at no charge on production of the receipt. No further claims can be entertained. The costs of transport in both directions will be borne by the customer should the equipment be located anywhere other than the customer's premises. In the event of transport damage, claims under warranty can be

honoured only if the complete original packaging was used for any transport and the scale secured and attached in that packaging just as it was when originally packed. All the packaging should therefore be retained.

A claim under warranty will not be honoured if the equipment is opened by persons not expressly authorised by seca to do so.

We would ask our customers abroad to contact their local sales agent in the event of a warranty matter.

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seca Vogel & Halke GmbH & Co.