
seca 675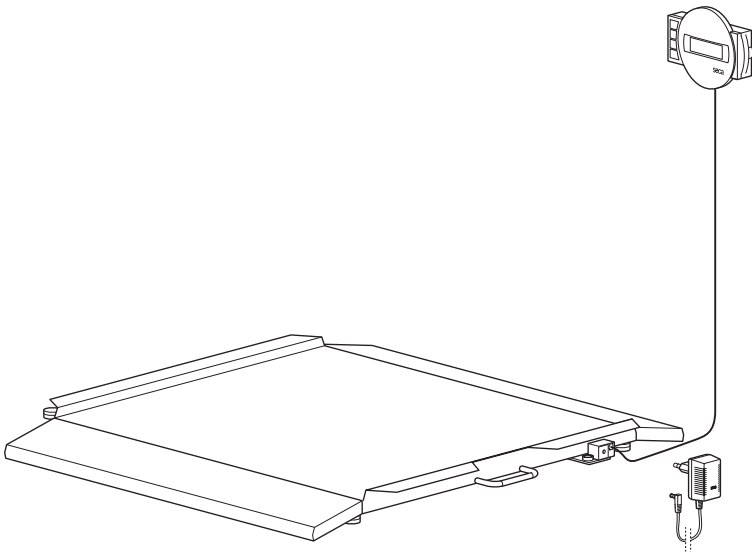

D	<i>Bedienungsanleitung und Garantieerklärung</i>	3
GB	<i>Instruction manual and guarantee</i>	15
F	<i>Mode d'emploi et garantie</i>	27
I	<i>Manuale di istruzioni e garanzia</i>	39
E	<i>Manual de instrucciones y garantía</i>	51
DK	<i>Betjeningsvejledning og garantibevis</i>	63
S	<i>Bruksanvisning och garanti</i>	51
N	<i>Bruksanvisning og garantierklæring</i>	87
FIN	<i>Käyttöohje ja takuu</i>	99
NL	<i>Bedieningshandleiding en garantieverklaring</i>	111
P	<i>Instruções de utilização e declaração de garantia</i>	123
GR	<i>Οδηγίες χειρισμού και εγγύηση</i>	135

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Full certification

In seca products you are purchasing not only technology matured over a century, but also quality confirmed by authorities, the legal system and by institutes. seca products already meet the provisional legal deadlines of tomorrow. With seca, you are buying the future.

M

The products in these operating instructions comply with German legislation governing medical products and therefore with Directive 93/42/EEC of the Council of the European Community which is embodied in national laws throughout Europe.

Scales carrying this symbol are approved for medical purposes in the European Community. seca scales carrying this symbol already meet the quality and technical requirements of the year 2003.

Scales carrying this symbol are calibrated in accordance with Precision Class III of EC Directive 90/384/EEC.

Products carrying this symbol meet the requirements

1. of the Directive for non-automatic scales in medicine, No. 90/384/EEC
2. for the law governing medical products, EC Directive No. 93/42/EEC
3. the Directive on electromagnetic compatibility

seca has been awarded the EN ISO 9001 certificate, which means that our company adheres to strict specifications for consistent high quality in research, production, sales and service. Furthermore, EN 46001 certifies seca's rigorous compliance with the most stringent European quality standards for medical products.

seca helps the environment.

We are concerned to save natural resources. We therefore strive to save packaging material where meaningful. What is left over can conveniently be disposed of locally via the German Dual System (recycling programme).

1. Congratulations!

By purchasing the **seca 675** electronic platform scale you have acquired a highly accurate and sturdy piece of equipment.

seca has been putting its experience at the service of health for over 150 years now, and as market leader in many countries of the world, is always setting new standards with its innovative developments for weighing and measuring.

In accordance with national regulations, the **seca 675** platform scale is used primarily in hospitals, doctors' surgeries and in-patient nursing facilities. The scales are a Class I medical product, and have been calibrated in accordance with the preci-

sion class III.

Weight display can be switched between kilogrammes (kg) and pounds (lbs). Weight is determined within a few seconds.

You can use the pre-Tare function to record the weights of, e.g. three types of chair. This weight is automatically deducted in the weighing operation.

The scale is of a very sturdy construction and will give you long and loyal service. It is easy to operate and the large display is easy to read.

The **seca 675** can be moved on castors.

2. Safety

Before using the new scale, please take a little time to read the following safety instructions.

- Follow the safety instructions in the instruction manual.
- Keep the operating manual and the declaration of conformity contained therein in a safe place.
- Do not drop the scale or subject it to violent shocks.
- Place the scale carefully in its position for use.
- Avoid pulling hard on the connecting cable to the display unit.
- Secure the scale so that it does not fall over when stored in the upright position.
- When moving the scale in its upright position, take care that it does not fall over.
- Be sure to use only the seca mains unit supplied. Before plugging the mains unit into a socket, check that the mains voltage indicated on the mains unit matches local mains voltage.
- When using the scale with a mains unit, ensure that the supply cable is routed in such a way as to exclude any type of tripping hazard.
- The cable between the weighing platform and the case must be laid so that it does not present a tripping hazard.
- Have scale serviced and re-calibrated on a regular basis (see "Servicing/recalibration" on page 25).
- Have repairs carried out exclusively by authorised persons.


3. Before you start...

Unpacking

Remove the packaging and place the scale on a firm, level surface.


The scope of supply includes:

- Weighing platform
- Display head plus connecting cable
- Wall bracket
- 1 angle adapter
- 2 Screws
- 2 Rawlplugs
- 1 seca mains unit
- Set of operating instructions plus declaration of conformity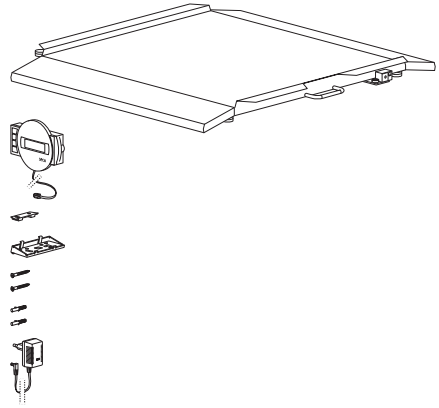


Fitting the display

- Carefully put down the scale.
- Connect the connecting cable to the electronics box.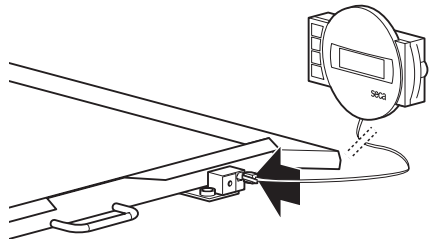


Desk-top stand

- Position the desk-top stand on the display casing from underneath. The **broad** side must be located at the top of the display. The desk-top stand locks in position.
- Place the display casing on a desk-top.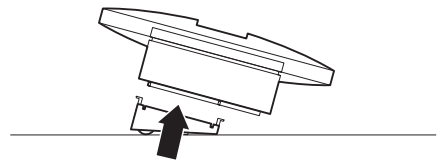

Wall bracket


- Place the desk-top stand on the display casing from underneath. The **narrow** side must be located at the top of the display. The desk-top stand locks in position.
- Fit the wall bracket to the wall at the desired height.
- Push the display casing onto the wall bracket.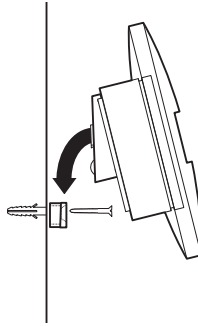

Power supply


The scale is powered by a mains unit.

You must check before connecting the equipment whether the AC voltage given on the mains unit matches the local mains voltage given on your electricity meter.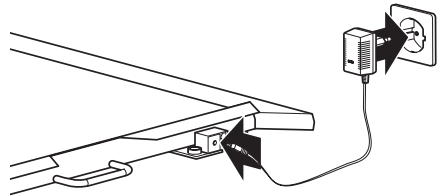

4. Setting up and aligning the scale

- Place the scale on a level surface.
- Undo the locknuts using the four foot screws (1).
- Align the scale by undoing the foot screws. The air bubble in the spirit level must be right in the centre of the circle (2).
- Then re-tighten the locknuts (3).
- Ensure that only the feet of the scale are in contact with the floor. The scale may not be in contact at any other point.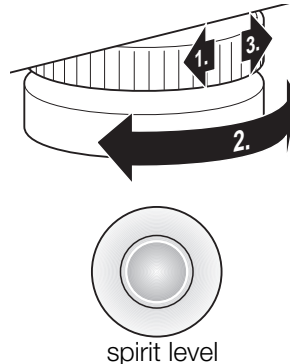


Important!

The alignment of the base of the scale must be checked and corrected if necessary every time the location of the scale is changed.

5. Operation

Controls and displays

The controls and display are in the display part.

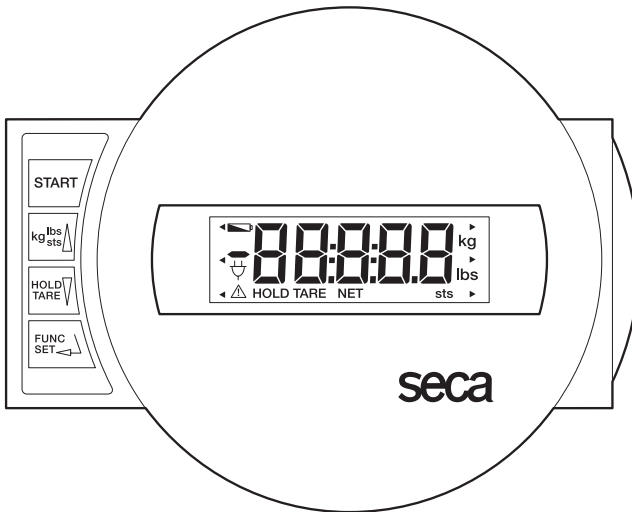Switch scale on and off

press briefly → switch between kg and lbs
in function menu → increase value, scroll

press briefly → activate hold function
hold down → set display to zero, activate/deactivate tare function
in function menu → decrease value, scroll

activate/deactivate pre-tare function, Confirmation

Checking that the contents of the calibration counter are correct

This seca scale is calibrated. Calibration may be performed only by authorised centres. To ensure this, the scale is fitted with a calibration counter which records any change to data relevant to calibration. If you want to check whether the scale is properly calibrated, proceed as follows:

- If necessary, switch off the scale.
- Keep any key depressed and start the scale.

The current contents of the calibration counter are shown on the display for a few seconds.

- Compare the output contents of the calibration counter with the number shown on the calibration counter sticker.

For valid calibration, both numbers must match. If sticker and calibration counter do not match, then re-calibration is required. Contact your service agent or seca Customer Service.

If a supplementary calibration is necessary, then the supplementary calibration mark illustrated opposite is used to record the calibration counter position instead of the seca calibration mark shown above. This mark is secured with an additional seal by the person authorised to perform the supplementary calibration. The supplementary calibration mark is available from seca Customer Service under the number 14-05-01-886.

Both numbers must match.

Correct weighing

- Press the Start key.
SECA, **8.8.8.8.8** and **0.0** appear consecutively in the display. The scale is then automatically set to zero and ready to use.
- If necessary, switch the weight display (see page 21).
- Stand on the scale and keep still.
- Read the weighing result off the digital display.
- If a load greater than 300 kg or 660 lbs is placed on the scale, **STOP** appears in the display.
- To switch off the scale, press the Start key again.

Switching the weight display

The weight display can be switched between kilogrammes (kg) and pounds (lbs).

- Switch on the scale using the green Start key. Weight is initially displayed in the last setting selected.
- To switch the weight display between kilogrammes and pounds, press the **kg/lbs/sts** key.
- The **kg** or **lbs** display lights up.

Switching off the scale

- To switch off the scale, press the **Start** key again.

Taring weight display (TARE)

The Tare function is very useful if an additional weight, e.g. that of a wheelchair, is to be ignored for weighing purposes.

Proceed as follows:

- Press the green Start key.
- First place the additional weight (e.g. the wheelchair) on the scale and activate the Tare function by pressing and holding the **HOLD/TARE** key.

The display is reset to zero, **00** flashes in the display until weight is detected to be stable. Only then is the Tare function fully activated. The note "NET" appears. You can now perform as many weighing operations as you like.

- Now move the patient and the wheelchair previously weighed onto the scale. The scale determines the weight of the additional load. The tare function can be re-activated at any time with an altered weight or without a weight.
- Put the new weight on the scale and depress and hold the **HOLD/TARE** key.
- The Tare weight is erased when the scale is turned off.

Store weight value (HOLD)

You can also continue displaying the weight determined once the load has been removed. This makes it possible to look after a patient first, before having to note the weight:

- Press the **HOLD/TARE** key whilst the load is on the scale.

"HOLD" appears in the digital display. The value is stored until the scale switches off automatically.

The triangle with the exclamation mark indicates that this frozen display is an additional facility which is not subject to calibration.

- The HOLD function can be switched off by pressing the **HOLD** key again.

0.0 appears in the display if the scale has no load on it, otherwise the current weight. The "HOLD" display vanishes.

Note:

The HOLD and TARE functions can also be used simultaneously.

Weighing with pre-Tare

If the pre-Tare function is activated, a stored weight value is deducted from the currently measured weight. This function is useful, for example, if the weight of a wheelchair is not to be displayed.

Three different tare weights (wheelchair weights) can be stored..

- Press the Start key.
SECA, **88888** and **0.0** appear consecutively in the display. The scale is then automatically set to zero and ready for operation.

- Press the **FUNC** key.
The last function used appears in the display.

An arrow on the left-hand side of the display indicates which memory is set.

- Use the arrow keys to select the desired memory for storing the additional weight or where a weight may already be stored.
- Press the **FUNC** key again. In the display you will see the tare value from the memory.
- You can use the arrow keys to adjust the value in increments according to the graduations displayed.

- Once the correct value is set, press the **FUNC** key again. The pre-Tare function is now activated.
- Move onto the scale in the wheelchair associated with that memory. Read the weighing result off the digital display.
- Press an arrow key if you would like to see the weight briefly.
- To return to normal weighing mode, press the **FUNC** key.
- To switch off the scale, press the Start key again.


Tip: note weight and the memory in which the weight is stored on your wheelchairs.

6. Storing the scale

The scale can be stored upright to save space. Secure the scale so that it does not fall over when stored in the upright position.

- Remove the mains unit plug from the mains socket.
- Remove the mains unit plug from the connecting socket on the scale.
- Press the locking mechanism on the Western plug and remove the plug from the socket on the electronics box.
- Carefully pick up the scale by the handle and move it using the castors on the other side of the scale.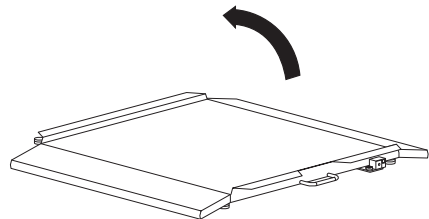

7. Cleaning

Clean the rubber surface and the housing as required using a domestic cleaning agent or commercially-available disinfectant. Follow the manufacturer's instructions.

Under no circumstances use abrasive or acid cleaners, white spirit, benzene or the like for cleaning. Such substances can damage the high-quality surfaces.

8. What do I do if...

... no weight display comes on?

- Is the scale switched on?
- Is the plug contact of the mains unit properly connected to the scale?
- Is the plug contact for connecting the display to the scale correctly connected?

... **0.0** does not appear before weighing?

- Press the Start key again – there must not be any load on the scale – and only its feet should be in contact with the floor.

... one segment is illuminated constantly or not at all?

- The relevant segment has a fault. Inform Servicing.

... **StOP** appears in the display?

- Maximum load has been exceeded.

... the display flashes?

- If you have just activated the Hold or Tare function, please wait until the scale has detected a steady weight value. The display will then stop flashing.
- If there has been no activation of a function beforehand, remove the load from the scale and wait until **0.0** is displayed, then weigh again.

... the display **tEMP** appears?

- The ambient temperature of the scale is too high or too low. Place the scale in an ambient temperature between +10 °C and +40 °C. Wait about 15 minutes for the scale to adapt to the ambient temperature and then weigh again.

... the display **E** and a number appear?

- Switch off the scale with the Start key and start the scale again. The scale will then work normally again.

If this is not the case, disconnect the power supply by briefly removing the batteries. If this measure is equally unsuccessful, inform the Service department.

9. Servicing/recalibration

Have the scale recalibrated by authorised persons in accordance with national legal requirements. The year of initial calibration can be found behind the CE symbol on the rating plate above the number of the appointed office, 0108 (Hamburg Calibration Office). Recalibration is essential if one or more security marks are damaged or the calibration counter does not match the number on the applicable calibration counter mark (cf. page 20).

We recommend having your scale serviced by your local service agent before having it recalibrated. seca Customer Service will be pleased to give you further assistance.

10. Technical data

Dimensions scale

width: 890 mm / 35 ins
height: 65^{+10} mm / $2.6^{+0.4}$ ins
depth: 965 mm / 38 ins

Dimensions weighing platform

width: 800 mm / 31.5 ins
height: 46^{+10} mm / $1.8^{+0.4}$ ins
depth: 965 mm / 38 ins

Weight

approx. 23.5 kg

Temperature range

+10 °C to +40 °C

Calibration

approved for medical use, Class III

Medical product as per Directive 93/42/EWG

Class I

Colour

silver grey/black

Maximum load

300 kg / 660 lbs

Minimum load

4 kg / 8.8 lbs

Graduations

200 g / 0.5 lbs

Accuracy at first calibration

0 to 100 kg: ± 100 g

100 kg to 300 kg: ± 200 g

Power supply

mains unit

11. Disposal

Disposing of the scale

If the scale can no longer be used, the responsible waste disposal association will be glad to advise you on the measures necessary to dispose of it properly.

12. Warranty

A two-year warranty from date of delivery applies to defects attributable to poor materials or workmanship. All moveable parts - batteries, cables, mains units, rechargeable batteries etc. - are excluded. Defects which come under warranty will be made good for the customer at no charge on production of the receipt. No further claims can be entertained. The costs of transport in both directions will be borne by the customer should the equipment be located anywhere other than the customer's premises. In the event of transport damage, claims under warranty can be

honoured only if the complete original packaging was used for any transport and the scale secured and attached in that packaging just as it was when originally packed. All the packaging should therefore be retained.

A claim under warranty will not be honoured if the equipment is opened by persons not expressly authorised by seca to do so.

We would ask our customers abroad to contact their local sales agent in the event of a warranty matter.

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seca Vogel & Halke GmbH & Co.