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User Instructions

UK

Fully read the user instructions carefully before using the Shower and Toilet Stool Atlantic for the first time. Keep the user instructions in a safe place so that they are available for later use.

Intended Purpose

The shower and toilet stool is suitable for domestic use by patients who, for example, due to paralysis, loss of limbs, frailty, cardiovascular illness or through injury require a stable support when showering or when using the toilet and are not able to visit the bathroom / WC independently.

Indications

The shower and toilet stool can be used in the event of restricted mobility of the legs and / or knees as well as the upper limbs, loss of strength, balance disorders and slight visual impairment.

Instructions / Assembly: Have this carried out by your specialist dealer / care assistant

1. Remove all the individual components from the cardboard box and check the contents and the packaging for damage and completeness.

2. Place the frame on the rubber feet on the floor and secure the upholstered toilet seat with the aid of the fixing brackets.


3. Secure the four wheels with the clips – ensure that these are all set to the same height! From the back, push the toilet pan with the lid into the holder and place the cover on to it.


4. All four wheels swivel and have a parking brake with directional locking. To brake, press the red brake lever down.

5. To release the brakes press the black locking mechanism.

6. Stick the tubes with the footrests in the openings provided for this on the left and the right. Ensure that the black bracket runs on the tube. Turn the footrests to the front until the pin engages in the bracket. The rests for the feet point inwards. Fold the footrests down.

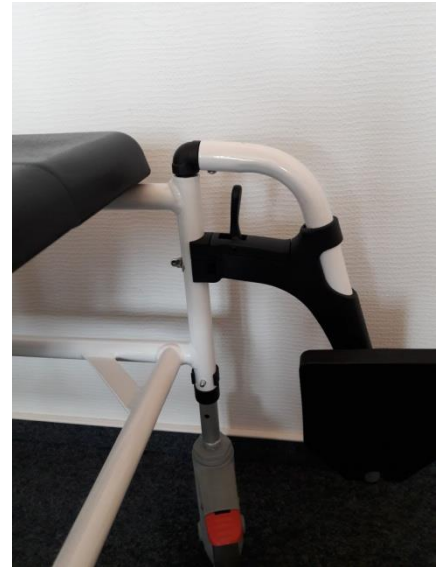
7. To adjust the height of the footrests, loosen the wing nuts, remove the screws and set the tubes to the desired position. Then replace the screws and firmly tighten the wing nuts.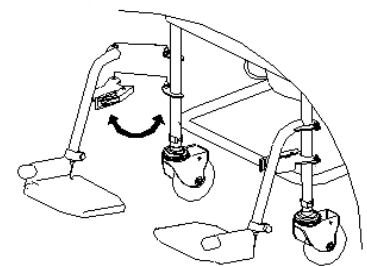

The footrests can be pivoted outwards or taken off. To do so, release the brackets on both sides through the lever and pull the tubes with the footrests from the frame.

8. Stick the lower tubes of the back rest in the back openings of the frame and fix them on with the two bolts. Fold the arm rests down.

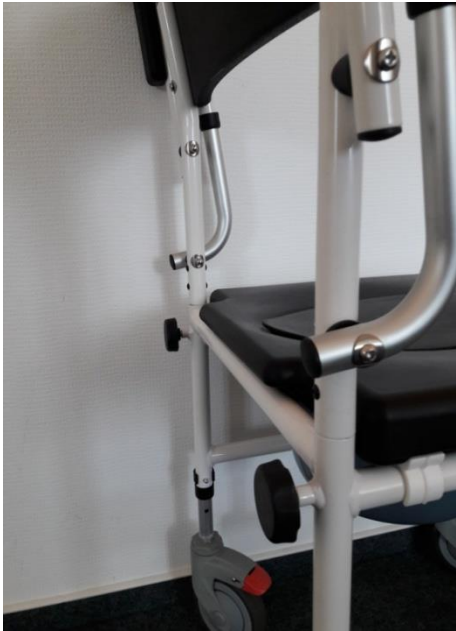9. The arm rests can be folded up to facilitate transport as well as sitting down on and getting up from the rollator. By releasing the bolts on the back of the frame, it is possible to pull the part with the back rest and the arm rests from the frame.

Safety Instructions

1. Ensure that the brakes have been applied before a person sits on the Shower and Toilet Stool Atlantic.
2. Before **every use**, check that the backrest and the footrests are secure.
3. Ensure that the wheels and the footrests have been set to the same height!
4. Before use, check that the toilet pan is seated firmly in its mount and if needed that the cover closes with the edge of the toilet seat.
5. During use ensure that the armrests are folded down. **Risk of being trapped!**

6. Ensure that the footrests are first folded down when the patient is sitting on the Shower and Toilet Stool Atlantic. Otherwise there is the risk of **tripping and injury!**
7. The maximum permissible load on the shower and toilet stool is 136 kg!
8. Do not use a defective shower and toilet stool and contact your specialist dealer if you notice damage to the material!
9. Do not stand on the footrests and do not lean one-sided on the armrests! **Risk of tipping over! Risk of falling!**
10. Do not stand on the shower and toilet stool and do not allow unattended children in its vicinity! **Risk of injury!**

Instructions for use

The shower and toilet chair can be used for sitting under the shower, for sitting when needed and as a means of transport, for example from the bed to the shower.

Without bucket and after removing the covering, the shower and toilet chair can also be positioned directly above a toilet bowl. As the seat height is of 48.5 – 56 cm, the toilet bowl should not be higher than 45 cm.

Cleaning

Clean the shower and toilet stool regularly with a mild soap solution and clean water. Use a soft cloth for drying. Do not use any abrasive or corrosive cleaning agents. The removable seat can be washed off under running water and can then be disinfected. The toilet pan should be treated in exactly the same way after every use. Use a commercially available disinfectant to disinfect the shower and toilet stool. Do not use high-pressure or steam cleaners for cleaning!

If they are dirty, clean the wheels using a soft brush or a cloth and a mild soap solution and afterwards dry them using a cloth.

Repeated use

The Atlantic shower and toilet chair is designed for daily use. It can be used by several people. Before the shower and toilet chair is used by another person or is transferred to the ownership of another person, it should be thoroughly cleaned and disinfected to prevent the transmission of germs (contamination). For this, the procedure described in the point *Cleaning* should be followed and a standard household disinfectant should be used.

Storage

Good ventilation must be ensured during use in the bathroom in order to avoid the building up of deposits. The shower and toilet stool must be dried after use. Before storing the shower and toilet stool it must be completely dry. Subsequently store it in a dry location protected against light in the polybag and cardboard box supplied.

Contraindications

The use of the shower and toilet stool is not suitable in the event of joint contractures, severe occurrences of paralysis, severe balance or visual disorders or severe pains in the trunk or in the upper

and lower limbs.

With significant loss of strength when sitting, the use of the shower and toilet stool must be refrained from as safe seating cannot be guaranteed.

Maintenance

The Atlantic shower and toilet chair is maintenance-free. The frame and connections consist of powder-coated aluminium and are not susceptible to corrosion. The seating surface with cover, the arm rests and the back rest consist of water-repelling PU.

Functional Faults:

If you notice the weakening of the brakes, stiffness of the wheels or a defect in the material, please contact your specialist dealer!

Do not use a defective or damaged Shower and Toilet Stool Atlantic!

Scope of Supply

Shower and toilet stool frame, 4 wheels including insertion tubes and clamps, 2 footrests including insertion tubes and fixing screws and nuts, backrest including insertion tubes and 2 locking screws, seat with lid, toilet pan with lid, user instructions, polybag, cardboard box.

Technical Data / Dimensions and Weight

Material: frame - aluminium, powder-coated, backrest, armrests and seating surface / toilet seat - PU, footrests – thermoplastic elastomer

Weight: 11,8 kg

Height: 94 -101,5 cm

Seat height: 48.5 – 56 cm

Distance from the seating surface to the upper edge of the arm rests: 21cm

Backrest height: 18 cm

Width of the backrest: 44 cm

Seat width: 43 cm

Seat depth: 39 cm

Seat angle: 4°

Back angle: approx. 5°

Overall width: 54,5 cm

Overall depth: 90 cm

Footrests: approx. 15 x 14 cm

Wheels: diameter – 12.5 cm, width approx. 3 cm

Maximum load-bearing capacity: 136 kg

Disposal

A worn-out shower and toilet stool is a source of valuable materials which can be reintroduced into the industrial cycle. Therefore do not dispose of your shower and toilet stool in the household waste, instead have it disposed of professionally by an officially approved waste disposal centre. Your local waste collection department will be pleased to provide you with the relevant information.

Guarantee

The guarantee period is 24 months and relates to all product defects which can be verifiably shown to be attributable to manufacturing or material defects.

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