

JETPULL® 2

Vielen Dank, dass Sie sich für diesen Marken-Venenstauer entschieden haben.

Der Stauer hat einen speziellen 2-Schritt Sicherheitsverschluss.

1. Lösen der Spannung des Bandes durch das Herunterdrücken der geriffelten Taste.
2. Zusammendrücken der beiden seitlich gegenüberliegenden Druckknöpfe.

Der Verschluss öffnet sich automatisch.

Bitte beachten Sie die Anweisungen auf der Rückseite.

„Jetpull 2“ ist komplett waschbar bis 90° oder mit geeigneten Desinfektionslösungen zu behandeln.

Intern. Patents · Made in Germany

JETPULL® 2

Thank you for deciding on the reusable tourniquet.

This device has a two-step special security opening:

1. Release by pushing down the ribbed key and then
2. Simply squeeze the lateral push buttons.

The closure casing ejects automatically.

Jetpull 2 can be machine washed up to 90° C or treated with appropriate disinfectants.

Intern. Patents · Made in Germany

JETPULL® 2


Instructions



open



close

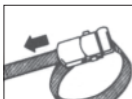

tighten



slow
release



Intern. Patents · Made in Germany

JETPULL® 2


Gebrauchsanleitung



öffnen



anlegen

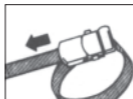

festziehen



langsam
lösen



Intern. Patents · Made in Germany
Holtsch Medizinprodukte GmbH, In den Faltern 13,
65232 Taunusstein, Germany