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**EN-** Please read these safety instructions carefully before use, and keep them for future question. If somebody else should use this product, ask him to read carefully these instructions.

### 1. Safety indications

- ♥ ALWAYS consult with your physician or supplier to determine proper adjustment and use of the device.
- ♥ Don't store outside and don't let under the sun (Beware: don't let the product in a car in case of high outside temperature !!).
- ♥ Don't use in extreme temperature (above 38°C or 100°F or below 0°C or 32°F).
- ♥ Check carefully all parts of this item.
- ♥ Use only as a bed/chair raiser and in proper way.
- ♥ ALWAYS observe the weight limit on the labelling of your product. Before using, check stability.
- ♥ No modification must be made on this device. Use only original parts to replace the broken or defect one and let this be done by the technical staff.
- ♥ In case you didn't use your product for a long time, let it checked by the technical staff, before using.

### 2. Installation

Only specialists can adjust the product and teach you how to use it. A wrong use or adjustment could lead to overstress of the device or of the patient. The device must be adjusted individually by the technical staff to suit your needs. Ask the technical staff how to use this product and make sure that you understood clearly their explanation.

In case of doubt, don't hesitate to ask again.

#### Indication

Furniture leg extenders are used for people that struggle to transfer in and out of low furniture and/or want to expand storage space under bed.

They are suitable for use with beds, chairs and other furniture on castors or legs. A recessed top section can safely accommodate the furniture leg.

#### Use

Carefully remove all packaging. Avoid using any knives or other sharp instruments as this may damage the product.

Check the product for any visible damage or cracks. If you see any damage or suspect a fault, do not use your product, but contact your supplier for support.

Carrefully lift the furniture leg on to the raiser. Repeat the same operation for each leg of the furniture. Make sure that each leg is centred on the raiser.

### 3. Cleaning

The product has to be cleaned regularly with a mild cleaning agent and a soft cloth.

### 4. Materials

The product is made up of sturdy plastic (PP).

### 5. Disposal

Dispose of this product in accordance with your regional waste disposal regulations.

### 6. Technical specifications

This product has a 2 years warranty, from the date of purchase, against any manufacturing defect.

**HR-** MOLIMOPrije uporabe pročitajte ove sigurnosne upute pažljivo, i zadržati ih za buduće pitanje. Ako netko drugi treba koristiti ovaj proizvod, zamolite ga da pročita ove upute.

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