- 1 Disposable
- 2 Lid
- 3 Sock
  - Coil
  - Disc
- 4 Drum
- 5 Drum holder
- 6 Bag holder
- 7 Pail
- 8 Adaptors
- 9 Baseplate
- 10 Brake pedal bracket \*
- 11 Brake pedal \*
- 12 2x wheels (rear wheels) \*
- 13 2 x wheels (front wheels) \*
- 14 16x screws

\* if ordered

### **important points of attention:**

1. **Always use the original consumer items or the warranty will be void (disposables, hygiene bags and storage bags)**
2. **Place the drum in the horizontal (rest) position when replacing the bag.**

#### **Prepare for use - Open the drum and deposit the disposable (1)**

- 1 Press and hold the black button and move the lid upwards using the bottom flap.
  - 2 Then press the bottom flap to release the bag holder and place the waste bag through the opening of the bag holder (see the icons on the bag holder).
- ATTENTION!** The top rim of the waste bag should be between the bag holder and the rim of the housing.
- 3 Close the lid. **CLICK!**

#### **Use**

- 4 Grab the handle and open the drum.
- 5 Place the incontinence material (diaper) in the opening. **ATTENTION!** Always use hygiene bags
- 6 Close the drum. **ATTENTION!** This is also the rest position.

#### **Replace disposable**

- 7 Use both hands and place two fingers in the notches underneath the black rim and pull the disposable upwards.
- 8 Press the black sides inwards to make sure that the disposable fits into the opening.
- 9 Press the black rim into the top of the drum. **CLICK!**

#### **Replace bag**

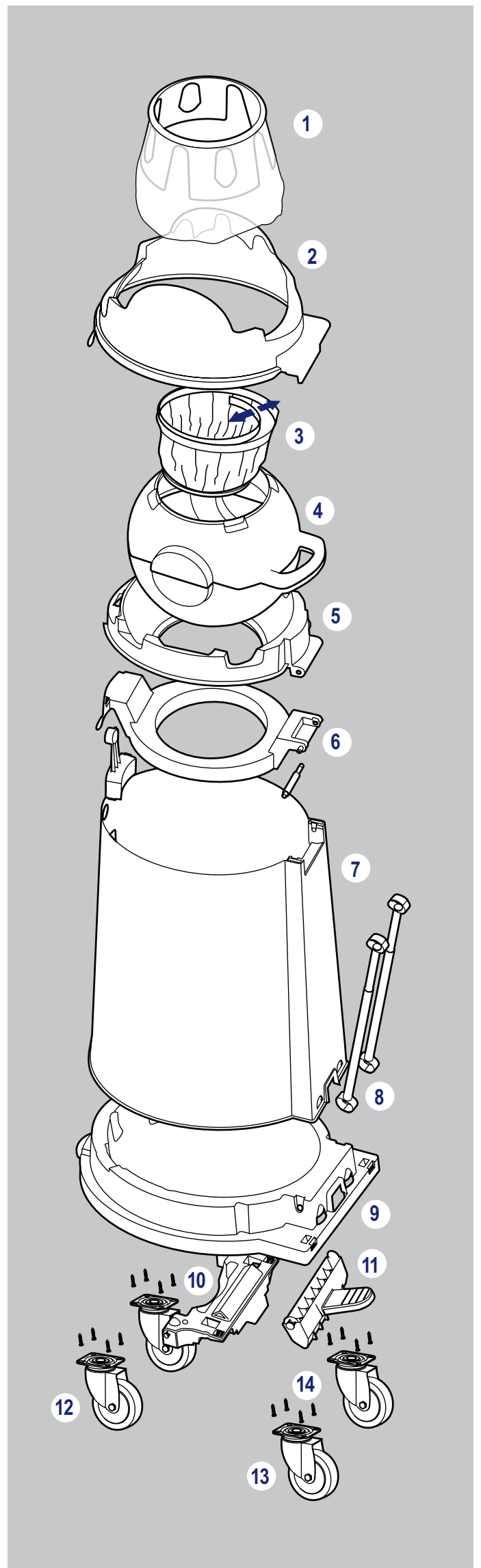
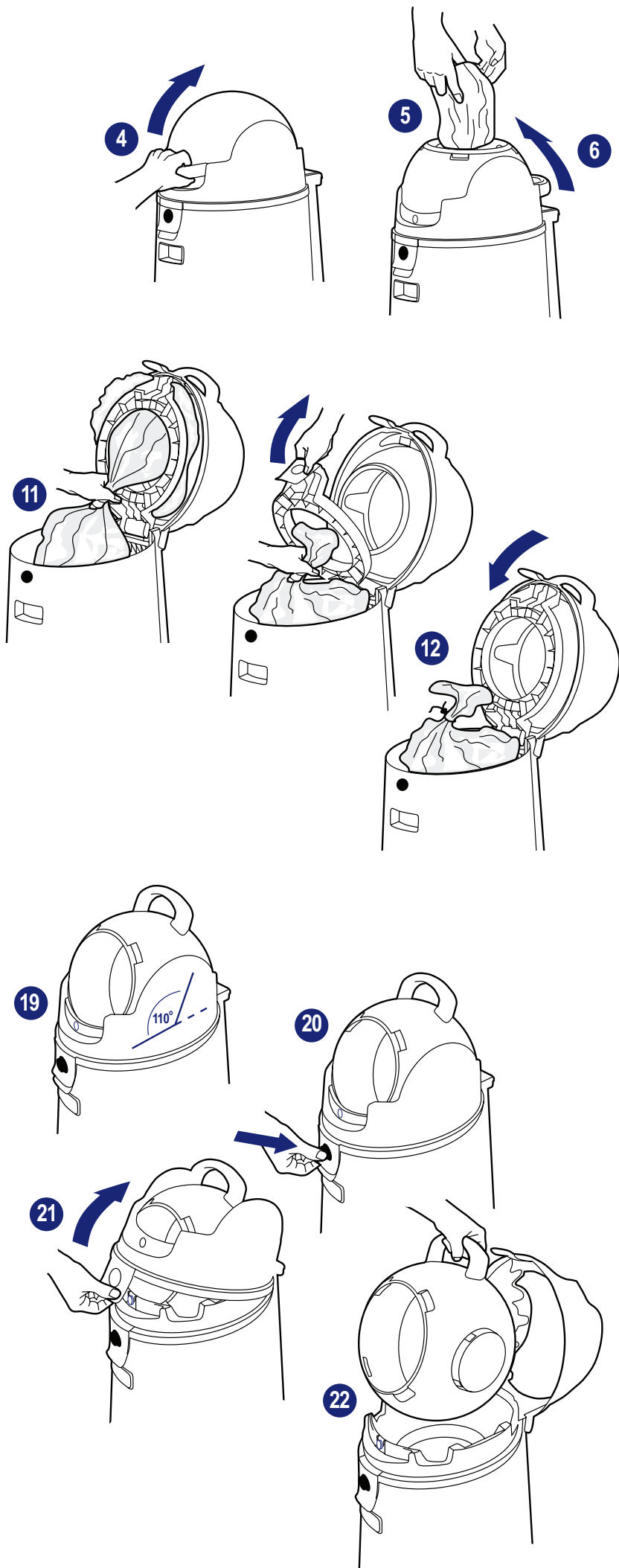
- 10 Press and hold the black button and move the bottom flap upwards. **ATTENTION!** The waste bag must move upwards as well. Always use original bags!.
- 11 For the odourless replacing of the waste bag, hold the waste bag closed with one hand and use the other to unlock the bag holder in order to release the waste bag.
- 12 Tie the waste bag closed. Leave the waste bag in the housing with the bottom flap.
- 13 Depress the pedal at the front, grab the handles at the front and back, and angle the top backwards and upwards to lift the housing up over the full waste bag
- 14 Remove the waste bag and replace the container on the base plate. For the placement of a new waste bag, see drawings 2 and 3. If necessary, replace the disposable as shown in drawings 7, 8 and 9.

#### **Cleaning the sock**

- 15 First remove the disposable. Pull the sock sideways and then upwards through the opening.
- 16 Remove the coil from the sock.
- 17 Loosen the sock from the disc. The sock can now be cleaned or replaced.
- 18 Replace the coil in the sock. Replace the disc in the sock. Compress the coil and replace the sock halfway into the drum in the recess created for this purpose.

#### **Cleaning the drum**

- 19 Place the drum in the indicated angle/position (see the drawing 110°). Use the same angle when replacing it.
- 20 Press and hold the black button.
- 21 Move the top flap upwards so that the hood moves over the drum.
- 22 Remove the drum and clean both the drum and drum holder. Apply a small amount of silicon gel on the white rim after cleaning. **ATTENTION!** Do NOT clean the white ring with cleaning agents containing chlorine.  
For drum replacement see drawing 19.

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