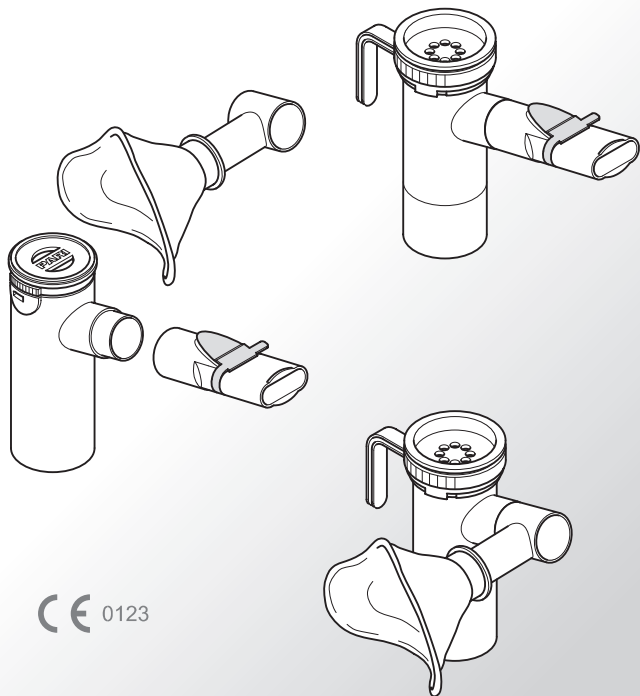
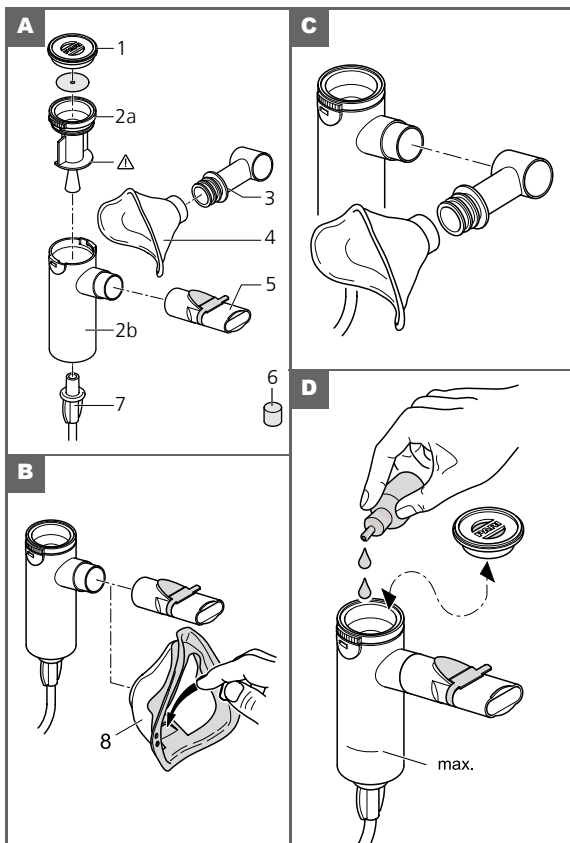
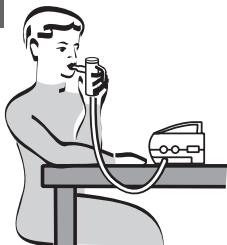
PARI LC Familie

CE 0123

E**H****F****I****G**

DE	Gebrauchsanweisung	1
EN	Instructions for use	25
DA	Brugsanvisning	47
EL	Οδηγίες χρήσεως	69
ES	Instrucciones de uso	93
FI	Käyttöohje	117
FR	Notice d'utilisation	139
IT	Istruzioni per l'uso	163
NL	Gebruiksaanwijzing	187
NO	Bruksanvisning	211
PT	Manual de Instruções	233
SV	Bruksanvisning	257
TR	Kullanma talimatı	279

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Stand der Information: November 2008

EN Instructions for use

Read these instructions carefully and completely before using for the first time. Keep these instructions in a safe place.

Personal safety cannot be guaranteed if these instructions are not strictly followed.

Table of contents

1	Components	26
2	Intended use	27
3	Information	28
4	Preparing to inhale	29
4.1	Assembling the nebuliser.....	29
4.2	Filling with medication.....	30
5	Inhalation	31
5.1	Important information.....	31
5.2	Inhalation with BABY bend and BABY mask.....	31
5.3	Inhalation with mouthpiece, child or adult mask.....	32
6	Hygienic re-use	33
6.1	General.....	33
6.2	Preparation.....	34
6.3	At home.....	34
6.4	Hospital and medical practice.....	36
7	Miscellaneous	38
7.1	Material resistance.....	38
7.2	Storage and transportation conditions.....	38
7.3	Disposal.....	38
8	Technical data	39
8.1	General.....	39
8.2	Aerosol characteristics in accordance with EN 13544-1 appendix CC.....	39
9	Replacement sets, replacement parts and accessories	42

1 Components

Figure A:

Please refer to the following table to ensure that all illustrated components are included in the corresponding nebuliser kit package. If not, please notify the dealer from whom you obtained this PARI nebuliser immediately.

Nebuliser kits	(1) Inspiratory valve	(2a) Nebuliser upper section	(2b) Nebuliser lower section	(3) BABY bend	(4) BABY mask	(5) Mouth-piece		(6) Filter insert for PARI BOY compressors	(7) Connection tubing
						with expiratory valve	without expiratory valve		
PARI LC		x	x				x		x ^{a)}
PARI LC PLUS PARI LC Junior	x	x	x			x			x ^{b)}
PARI Junior Package I	x	x	x	x	x	x			
PARI BABY	x	x	x	x	x				x
PARI LC STAR	x	x	x			x			
PARI Year Packs	x	x	x			x		x	x

a) only for PARI LC Kit 022G6100

b) only for PARI LC PLUS Kits 022G8100, 022G8104

2 Intended use

Nebulisers of the PARI LC family are medication nebulisers; they can be combined with the PARI compressor, or connected to a hospital's central compressed air supply using the PARI CENTRAL and to PARI accessories as necessary to provide inhalation treatment for the lower respiratory tract. They are only suitable for temporary¹⁾ oral inhalation of liquid medicine prescribed or recommended by a doctor. The nebulisers of the PARI LC family are capable of nebulising all commercially available medications that are currently approved for inhalation therapy.

The nebulisers of the PARI LC family can be operated with **all** PARI compressors that are available on the market (for example the PARI BOY S, PARI BOY SX, PARI BOY N, PARI MASTER etc.) or with a clinic's central compressed air supply (see Chapter 8.1 "General"). **Please read and follow the instructions for use with your PARI compressor and/or PARI accessory.** PARI GmbH cannot be held liable for damage that is caused by using PARI nebulisers with other compressors.

The manufacturer will be able to tell you whether the PARI LC nebuliser can be used with new PARI compressors that are introduced in the future. This information will also appear in the instructions for use of the compressor.

1) Each application lasts less than 1 hour, in this case max. 20 minutes.

The PARI LC family of nebulisers guarantee effective inhalation therapy for all ages, from infant to adult. With the exception of the PARI LC and BABY nebulisers (the PARI BABY nebuliser has only one inspiratory valve), all PARI nebulisers are equipped with a valve system. During inhalation the nebuliser's valve system controls the flow of medication and substantially reduces medication losses.

PARI LC PLUS, LC PLUS Turbo and LC PLUS Junior nebuliser

The PARI LC PLUS and PARI LC PLUS Turbo nebulisers are suitable for use in inhalation therapy by adults. The design of the PARI LC PLUS Junior has been specially modified to suit the needs of children. The aerosol produced by these nebulisers gives the optimum particle size for the effective administration of medication to the central region of the lung.

PARI LC STAR nebuliser

The PARI LC STAR nebuliser is a mist nebuliser for targeted administration of medication to the deep respiratory tract (peripheral area) of the lung. It is suitable for use by adults and children (aged 3-4 years and older) who can already inhale through the mouthpiece.

PARI BABY nebuliser

The PARI BABY nebuliser enables efficient inhalation therapy with medication aerosols for the anatomically smaller respiratory tracts of infants. It is equipped with a soft PARI BABY mask made from latex-free silicone and a PARI BABY bend. The PARI BABY nebuliser is available in 4 models, each of which is equipped with a different size mask (see Chapter 9, "Replacement sets, replacement parts and accessories") corresponding to the age of the child. The PARI BABY bend can be rotated to any angle, thus enabling inhalation while the baby is sitting up or lying down.

If nebulisers of the PARI LC family are rented to an end-user as part of a PARI inhalation device, by a pharmacist for example, this is done on the responsibility of the pharmacy involved and is not the responsibility of PARI GmbH. We strongly advise users to follow the guidelines for preparing PARI inhalation devices for re-use on a change in patients. These guidelines are available upon request from PARI GmbH, or on the Internet at www.pari.de, "For Pharmacists and Dealer" tab.

- Only use medication prescribed or recommended by your doctor.
- You must conscientiously follow the general hygiene instructions for using your nebuliser, e.g., thorough washing of hands or disinfection if required.
- LC nebulisers are only suitable for patients who are able to breathe spontaneously.
- In patients who have undergone tracheostomy, the PARI TRACHEO SET has to be used.

4 Preparing to inhale

- Before the device is used for the first time, before it is re-used, and after any long break in use, please follow the hygiene regulations.
- Ensure that the nebuliser was cleaned, disinfected, and, if appropriate, sterilised after the last treatment (see Chapter 6 "Hygienic re-use").

4.1 Assembling the nebuliser

Figure A:

- Insert the nebuliser upper section (2a) in the nebuliser lower section (2b) and screw it tight.
- Attach the connection tubing (7) from the compressor or compressed air supply to the nebuliser.

4.1.1 Using with a mouthpiece

Figure B:

- Hold the nebuliser upright.
- Attach the mouthpiece (5) to the nebuliser.

4.1.2 Using with a child or adult mask

Inhalation with the mouthpiece is the most effective form of inhalation, since it results in minimum medication loss on the way to the lung. If it is not possible to use a mouthpiece, or if the patient prefers to inhale through a mask, PARI can provide the right accessories.

For babies, inhalation with the PARI BABY mask (of the appropriate size) together with the BABY bend is recommended (see Chapter 4.1.3).

Figure B:

- Hold the nebuliser upright.
- Attach the child or adult mask (8) to the nebuliser.

Safety instruction for PARI Child and Adult masks soft:

It is essential to ensure that the expiratory valve plate on your mask is pressed out before each use (see Figure B).

4.1.3 Use with BABY bend and BABY mask

Figure C:

- Hold the nebuliser upright.
- Place the BABY mask (4) on the BABY bend (3). Use your thumb to press the BABY mask from the inside over the first lip of the BABY bend.
- Attach the BABY mask with the BABY bend to the nebuliser.

Caution: PARI BABY masks must only be used with the PARI

BABY bend, because the bend includes the apertures for exhaling.

4.2 Filling with medication

Figure D:

- If necessary, remove the inspiratory valve (1) from the nebuliser.
- Pour the quantity of medication prescribed by your doctor into the nebuliser funnel.

Ensure that the medication is filled no higher than the upper scale marking (max. level 8ml). If you overfill the nebuliser and the medication leaks out of the bottom, please empty and clean (see Chapter 6 "Hygienic re-use"). Then fill the nebuliser with the medication again.

- Attach the inspiratory valve to the nebuliser.

If you are using a PARI Child or PARI Adult mask (014G1501 or 012E1721):

- Use the nebuliser without the inspiratory valve.

5 Inhalation

5.1 Important information

- Before the patient starts inhaling, ensure that all parts are firmly connected to each other.
- Ensure that the air outlets are connected firmly to the air connection of the PARI compressor, to the compressed air supply via the PARI CENTRAL, and also to the nebuliser.

Medication nebulisation can be adversely affected by an incorrectly assembled nebuliser.

Babies, infants, children, and anyone who requires assistance must be supervised constantly by an adult during inhalation therapy. Otherwise, safe and effective treatment cannot be guaranteed.

Caution: Contains small parts which can be swallowed!

5.2 Inhalation with BABY bend and BABY mask

- Swivel the bend so that it corresponds with the position of the baby or small child. Always hold the nebuliser upright.
- Switch the compressor on or insert the connection tubing in the compressed air outlet in the hospital wall provided for this purpose.
- Press the mask gently but firmly over the mouth and nose (see Figure E).
- For best inhalation, your child/baby should breathe in and out slowly and deeply.
- Conduct the treatment for the period prescribed by the doctor. Ensure that the mask forms a good seal.
- Please continue reading in Chapter 6 "Hygienic re-use".

5.3 Inhalation with mouthpiece, child or adult mask

- Sit in an upright position and relax (see Figure F).
- Switch the compressor on or insert the connection tubing in the compressed air outlet in the hospital wall provided for this purpose.
- Take the mouthpiece between your teeth and surround it with your lips (see Figure G) or hold the mask firmly over the mouth and nose.
- Breathe in through the mouthpiece or mask as slowly and deeply as possible (see Figure H).
- Breathe out gently through the mouthpiece or mask. The exhaled air should pass through the expiratory valve in the mouthpiece or the valve plate in the mask (expiratory valve will open, see Figure I).
- Continue breathing in and out as described until a change in the sound of the nebuliser signals that the medicine has been used up.
- Check whether a clearly visible aerosol is still flowing from the nebuliser.
- Stop treatment as soon as the aerosol emerges irregularly.

6 Hygienic re-use

6.1 General

In order to prevent a risk to health, e.g., an infection occurring through a contaminated nebuliser, it is imperative that the following hygiene regulations are followed.

Nebulisers of the PARI LC family are designed for multiple use. Please be aware that different requirements regarding hygienic re-use are in effect for different fields of application:

- At home (single patient use):
The nebuliser must be cleaned and disinfected after each use.
- Hospital / medical practice (possible change in patient):
The nebuliser must be cleaned, disinfected and sterilised after each use.
- Additional requirements concerning the necessary hygienic preparations (hand care, handling of medications or inhalation solutions) in high-risk groups (e.g., patients with cystic fibrosis) can be obtained from the respective self-help groups.
- Proof of suitability of the nebulisers for effective cleaning, disinfection and sterilisation has been obtained through an independent testing laboratory using the recommended procedures stated in Section 6.4.1 "Cleaning/disinfection" (test documentation available for inspection on request). Using the named alternatives remains the responsibility of the user.
- Please ensure adequate drying after each cleaning, disinfection and/or sterilisation. Condensation or residual wetness can present an increased risk through the growth of bacteria.
- Please check the parts of your nebuliser regularly and replace any defective (broken, misshapen, discoloured) parts.

The PARI Child and Adult masks soft cannot be sterilised.

Please also read and follow the hygiene instructions in the instructions for use of the PARI compressor.

6.2 Preparation

Directly after each treatment, all parts of the PARI LC nebuliser must be cleaned of medication residue and contamination. To do this, dismantle the nebuliser into its individual parts (as described in the following).

Figure A:

- Detach the mouthpiece (5) and the BABY mask (4) and BABY bend (3) from the nebuliser together with the inspiratory valve (1).
- Twist the nebuliser upper section (2a) anti-clockwise and pull it out of the nebuliser lower section (2b).

For hygienic re-use at home please continue reading the following section.

For hygienic re-use in the hospital or medical practice, please read Chapter 6.4 "Hospital and medical practice".

6.3 At home

6.3.1 Cleaning

- Remove the tubing from the nebuliser and dismantle the nebuliser (see Chapter 6.2 "Preparation").

Refer to Section 6.3.5 for care of the connection tubing.

Recommended method: Use warm tap water

- Clean all parts of the nebuliser thoroughly for 5 minutes with warm tap water (approx. 40°C) and a little dishwashing liquid (dosage according to the manufacturer's recommendations).
- Then rinse all parts thoroughly under warm running water (approx. 40°C without dishwashing liquid).
- Drain off well.

You can remove excess water more quickly by shaking the parts.

Possible alternative: Wash in the dishwasher

- Clean the nebuliser in your dishwasher (not together with dirty crockery).
- Distribute the nebuliser parts in the cutlery basket.
- Select a programme of at least 50°C.

6.3.2 Disinfection

Disinfect the dismantled nebuliser after cleaning.

Recommended method: Use the NUK Vaporisator 6 (part no. 500G2000) or the Petra disinfectant DI 6.00 (part no. 041G6000)

Be sure to follow the instructions for use of the NUK Vaporisator 6 and the DI 6.00 disinfectant. The treatment time is about 15 minutes.

- Do not use Vaporisators unless they are clean and working perfectly.
- Water residue in the bowl is a sign that the equipment is malfunctioning.
- For information about the quantity of water needed, please read the instructions for use of the Vaporisator you are using.
- Make sure that the Vaporisator is functioning safely.

Possible alternative: In boiling water

- Place the individual parts of your nebuliser in boiling water for at least 15 minutes.
- Use a clean saucepan. If your local tap water is hard, please use low-lime or distilled water.

Ensure that the water level in the pot remains high enough to prevent the nebuliser parts from coming into contact with the hot base of the pot.

6.3.3 Drying

- Place the nebuliser parts on a dry, clean and absorbent surface and let them dry completely (at least 4 hours).

6.3.4 Storage

- Between uses, and in particular during long breaks in treatment, wrap the nebuliser in a clean, lint-free cloth (e.g., a tea-towel) and keep it in a dry, dust-free environment.

6.3.5 Care of the connection tubing

- After each treatment, pull the tubing from the nebuliser and switch on your PARI compressor.
- Leave the compressor running until any condensation in the tubing has been removed by the air-flow.
- Replace the connection tubing (item no.041B4591, see Chapter 9) if it becomes contaminated.

Please continue reading Chapter 7 "Miscellaneous".

6.4 Hospital and medical practice

- When using the nebuliser for several patients, the nebuliser must be cleaned, disinfected and sterilised before each change in patient.
- Please ensure that only adequately validated device-specific and product-specific procedures are used for cleaning, disinfection and sterilisation, and that the validated parameters are observed in each cycle.
- The cleaning and disinfection procedure must be recognised as effective (e.g., in the list of tested and recognised disinfectants and procedures issued by the Robert Koch Institute/DGHM) and already validated in all cases.
- If another procedure is used, its effectiveness must be confirmed during validation. CE-approved chemical disinfectants may also be used, provided they satisfy the requirements for material resistance (see Chapter 7.1 "Material resistance").
- Please follow the instructions in Chapter 7.1 "Material resistance".
- In addition, please follow the hygiene regulations of the hospital or medical practice.

6.4.1 Cleaning/disinfection

Clean and disinfect immediately after use. Ideally, a mechanical procedure (instrument disinfectant) should be used.

- Prepare the cleaning process and disinfection as described in Chapter 6.2 "Preparation".

Any suitable cleaner/disinfectant can be used.

Recommended procedure: Thermal disinfection

- Place the nebuliser components in the instrument disinfectant.
- Select the 93°C programme (treatment time 10 minutes).

The effectiveness of this procedure has been confirmed using a G7836 CD disinfectant manufactured by Miele with neodisher® medizym cleaning agent manufactured by Chemische Fabrik Dr. Weigert, Hamburg.

or

Recommended procedure: Manual cleaning/disinfection

The effectiveness of this procedure has been confirmed using the Korsolex®-Endo-Cleaner cleaning agent in combination with the Korsolex® basic disinfectant manufactured by BODE CHEMIE HAMBURG.

- The components must be placed in a 0.5% solution of Korsolex®-Endo-Cleaner and approx. 50°C warm water, and cleaned thoroughly for 5 minutes.

- The components are then placed in a 4% disinfectant solution of Korsolex® basic for 15 minutes.
- Then thoroughly rinse the nebuliser components under hygienically tested water and leave them to dry completely on a clean, dry and absorbent surface (for at least 4 hours).

Possible alternative: Chemical-thermal disinfection

- Place the nebuliser components in the instrument disinfectant.
- Add a suitable cleaner/disinfectant and select the 60°C programme.

Storage

- Between uses, wrap the nebuliser in a clean, lint-free cloth and keep it in a dry, dust-free place protected from contamination.

6.4.2 Sterilisation

After cleaning/disinfection, pack the dismantled nebuliser in sterilisation packaging (disposable sterilisation packaging, e.g., foil/paper sterilisation bags). The sterilisation packaging must conform to DIN EN ISO 11607-1 and be suitable for steam sterilisation. Then sterilise the nebuliser using the following sterilisation procedure:

Recommended procedure: Steam sterilisation

Sterilisation temperature: 121°C (to be maintained for at least 20 min.) or
132°C/134°C (to be maintained for at least 3 min.)
max. 137°C.

This procedure has been validated according to DIN EN ISO 17665 as well as DGKH guidelines for the validation and routine monitoring of sterilisation processes using damp heat for medical devices.

Storage

Keep the sterilised nebuliser in a dry, dust-free place protected from contamination.

6.4.3 Care of the connection tubing

- In the case of visible condensation in the tubing, pull it off the nebuliser after treatment and switch on your PARI compressor.
- Leave the compressor running until any condensation in the tubing has been removed by the air-flow.
- Replace the connection tubing (item no.041G4550, see Chapter 9) when changing patients or if it becomes contaminated).

7.1 Material resistance

Do not expose the nebuliser to temperatures above 137°C. In general, like all plastic parts, nebulisers are susceptible to wear. Nebulisers can be sterilised up to 300 times, but then they must be replaced. Even if the nebuliser has been sterilised fewer than 300 times, it must still be replaced after one year's use. Note the following when selecting cleaners/disinfectants:

- In all cases, the aldehyde group of cleaning and disinfecting agents is suitable for cleaning and disinfecting nebulisers in the PARI LC family.
- The use of other cleaners or disinfectants has not been tested with regard to the material resistance of this nebuliser.

7.2 Storage and transportation conditions

Do not store in damp rooms (e.g., bathrooms) or transport together with damp objects.

Protect from continuous direct sunlight during storage and transportation.

In general, the nebuliser should be stored as described in Chapter 6 "Hygienic re-use", "Storage" between applications. See Chapter 6.3.4 ("At home") or Chapter 6.4 ("Hospital and medical practice").

7.3 Disposal

The PARI nebuliser may be disposed of with domestic waste unless the disposal regulations in the respective member countries provide otherwise (e.g., disposal by local authorities or dealers).

Materials recycling helps to reduce the consumption of raw materials and protect the environment.

8 Technical data

8.1 General

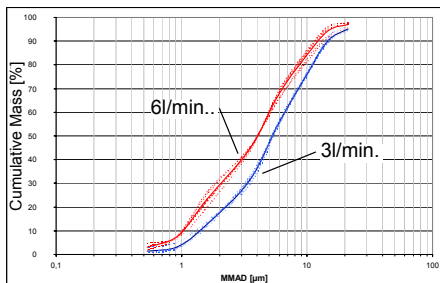
Recommended nebuliser gases:	Air, oxygen
Minimum flow:	3.0l/min.
Corresponding operating pressure:	0.5bar/50kPa
Maximum flow:	6.0l/min.
Corresponding operating pressure:	2.0bar/200kPa
Minimum fill:	2ml
Maximum fill:	8ml

8.2 Aerosol characteristics in accordance with EN 13544-1 appendix CC

8.2.1 Determination of particle size

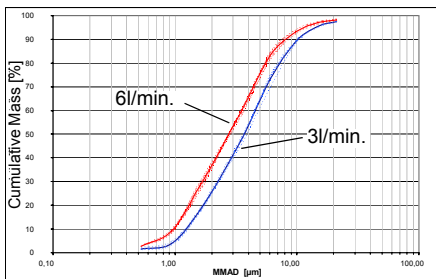
Fill the nebuliser with 3ml of a 2.5% solution of sodium fluoride and then attach the nebuliser to a Marple cascade impactor and run for at least 3 minutes.

Cumulative mass distribution for the PARI LC/ LC PLUS/ LC PLUS Turbo/ LC PLUS Junior nebulisers at the minimum flow (3l/min.) and maximum flow (6l/min.):

(Averages from each of 2 separate measurement series with 3 nebuliser/ compressed air combinations)

Cumulative mass distribution for the PARI LC STAR/BABY nebuliser at the minimum flow (3l/min.) and the maximum flow (6l/min.):

(Averages from each of 2 separate measurement series with 3 nebuliser/ compressed air combinations)

8.2.2 Determination of the aerosol output and aerosol output rate

Aerosol output (aerosol quantity output by the nebuliser system for a specified fill volume)

Fill the nebuliser with 2ml of a 1% sodium fluoride solution. Attach the nebuliser to the PARI Compass Breath Simulator until the aerosol becomes irregular, then one more minute.

Aerosol output in [ml] for minimum flow (3l/min.) and maximum flow (6l/min.):

	PARI LC	PARI LC PLUS/ LC PLUS TURBO/ LC PLUS JUNIOR	PARI LC STAR PARI BABY
Min. flow (3l/min.)	0.14	0.38	0.47
Max. flow (6l/min.)	0.32	0.42	0.60

(Averages from each of 2 separate measurement series with 3 nebuliser/ compressed air combinations)

Aerosol output rate (volume of aerosol that is output by the nebuliser system per unit of time)

Fill the nebuliser with 2ml of a 1% sodium fluoride solution. Attach the nebuliser to the PARI Compass Breath Simulator for at least 1 minute.

Aerosol output rate in [ml/min.] for minimum flow (3l/min.) and maximum flow (6l/min.):

	PARI LC	PARI LC PLUS/ LC PLUS TURBO/ LC PLUS JUNIOR	PARI LC STAR PARI BABY
Min. flow (3l/min.)	0.06	0.10	0.07
Max. flow (6l/min.)	0.17	0.16	0.15

(Averages from each of 2 separate measurement series with 3 nebuliser/compressed air combinations)

9 Replacement sets, replacement parts and accessories

For illustrations of replacement parts and accessories, refer to the fold-out pages in the brochure cover.

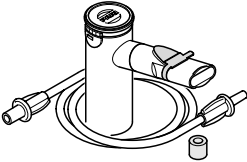
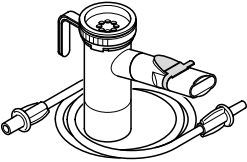
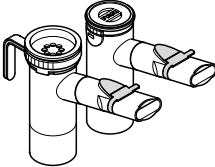
Figure no.	Description	Part no.
Replacement sets:		
	PARI JuniorBOY Year Pack	022G7021
	PARI LC PLUS Year Pack	022G8010
Replacement parts and accessories:		
	PARI LC PLUS with connection tubing (1.2 m)	022G8104
	PARI LC PLUS with connection tubing (2.1 m) to the hospital compressed air supply	022G8100
	PARI LC PLUS	022G8000
	PARI LC PLUS Junior	022G7001
	PARI LC STAR	022G8700

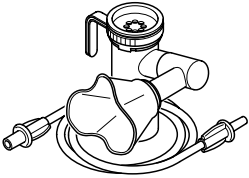
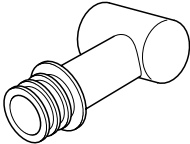
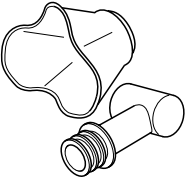
Figure no.	Description	Part no.
	PARI BABY size 0 (for premature babies)	022G9400
	PARI BABY size 1 (for age up to 1 year)	022G9100
	PARI BABY size 2 (for age up to 3 years)	022G9200
	PARI BABY size 3 (approx. 4 years and up)	022G9300
	BABY bend	041E0701
	BABY mask size 0 with BABY bend (for premature babies)	041G0700
	BABY mask size 1 with BABY bend (for age up to 1 year)	041G0701
	BABY mask size 2 with BABY bend (for age 1-3 years)	041G0702
	BABY mask size 3 with BABY bend (for age about 4 years and above)	041G0703
	Inspiratory valve for PARI LC PLUS	041B0510

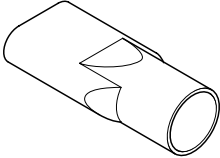
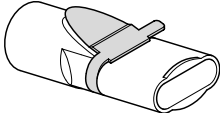
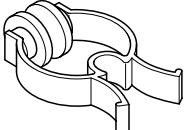
Figure no.	Description	Part no.
	Mouthpiece (without expiratory valve)	012E1720
	Mouthpiece (with expiratory valve)	022E3050
	Connection tubing for PARI compressors (1.2 m) (e.g., PARI BOY family, PARI MASTER etc.)	041B4591
	Connection tubing (2.1m; for use only with the hospital com- pressed air system)	041G4550
	LC interrupter	022G1000
	Nose clip	041E3500

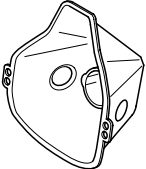
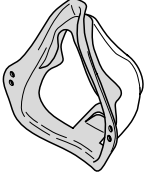
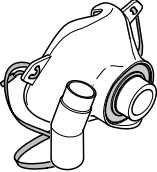
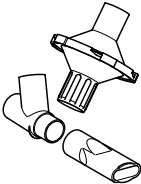
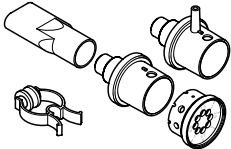
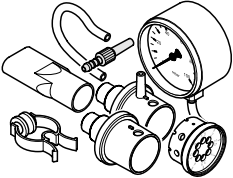
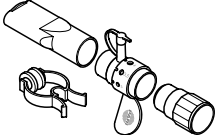
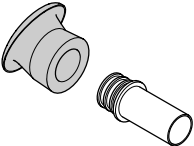
Figure no.	Description	Part no.
	Adult mask	012E1721
	Child mask	014G1501
	Adult mask soft	041G0740
	Child mask soft "Spiggy"	041G0741
	PARI SMARTMASK (inc. 45 degree bend)	041G0730
	PARI Filter/Valve Set	041G0500
	PARI PEP System I	018G6101

Figure no.	Description	Part no.
	PARI PEP System II	018G6201
	PARI PEP S System	018G4000
	PARI TRACHEO SET	025G0007

We reserve the right to make technical changes.

For all product information please contact our Service Center:
Tel.: +49-(0)8151-279 220

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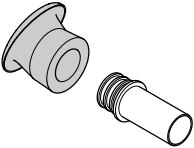
Şekil no.	Tanım	Art. No.
A line drawing of a PARI SMARTMASK, which is a protective mask with a circular lens and a strap.	PARI SMARTMASK (45 derecelik dirsek parçası dahil)	041G0730
A line drawing of a PARI Filtre/Supap Seti, showing a filter and a valve component.	PARI Filtre/Supap Seti	041G0500
A line drawing of the PARI PEP-System I, showing various components including a filter, a valve, and a pressure gauge.	PARI PEP-System I	018G6101
A line drawing of the PARI PEP-System II, showing various components including a filter, a valve, and a pressure gauge.	PARI PEP-System II	018G6201
A line drawing of the PARI PEP S-System, showing various components including a filter, a valve, and a pressure gauge.	PARI PEP S-System	018G4000

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