Haarwaschbecken RFM®

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DE

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Instructions for use

UK

Read the complete instructions for use carefully before using the inflatable RFM hair washing basin for the first time. Keep the instructions available for later use.

Intended purpose

The RFM hair washing basin is suitable for household use, for patients who are not in a position to leave the bed to wash their hair in the bathtub or the shower, due to paralysis, loss of members, weakness, cardiovascular illness or an injury, for example.

Indication

The RFM hair washing basin is to be used in case of full confinement to the bed and immobility of the person requiring care if other washing possibilities that mobilise the patient can no longer be used.

Instructions

1. Remove the RFM hair washing basin from its packaging and check whether any damage or change to the material or hose connection is apparent.
2. Unfold the RFM hair washing basin and pump it up with the hand pump provided. Close the opening.
3. Place the RFM hair washing basin on the patient's bed on a towel and make sure it is straight. Close the opening in the base of the basin, where the outlet hose is.
4. If there is no washing basin in reach of the hose, put a bucket next to the bed for the used water.
5. Lay the patient's head in the RFM hair washing basin. The neck should lay in the opening.
6. Wash the patient's hair and rinse it out. You can use the RFM shower set available for the hair washing basin for this. The used water can be drained through the opening in the base of the hair washing basin and the hose into a bucket or a wash basin. Close the opening again and continue to rinse the hair if needed.

7. To finish, hold the patient's head out of the RFM hair washing basin and drain all of the water.

Safety indications

1. Make sure the hair washing basin is straight so that the patient's head is securely supported.
2. Check before EVERY use that the hair washing basin is airtight and not leaking air!
3. Make sure that the openings of the RFM hair washing basin are tightly sealed.
4. Never let the patient lay with their head in the RFM hair washing basin without supervision.
5. Clean and dry the RFM hair washing basin after every use and do not let children play with it, at risk of injury!
6. Make sure that no sharp objects damage the RFM hair washing basin.
7. Do not use any defective RFM hair washing basin!

Cleaning

Rinse the RFM hair washing basin with clear water and dry it with a towel. In case of ingrained dirt, you can use a mild, standard household cleaning product. Make sure to thoroughly clean all the product surfaces. Lastly, cleaning product residue can be removed with a damp cloth. Do not use any abrasive or caustic cleaning products.

Disinfecting

The product can be wipe-disinfected with standard household disinfectant products. Observe the concentration and acting time indicated by the disinfectant manufacturer. All surfaces are to be wiped with a clean cloth with disinfectant. Treat the product evenly with disinfectant. Do not rinse the disinfectant off and let the product air dry.

Repeated use

The RFM hair washing basin is designed for daily or regular use. It can be used by several people. Before the RFM hair washing basin is used by another person or is transferred to the ownership of another person, it should be thoroughly cleaned and disinfected to prevent the transmission of germs (contamination). For this, the procedure described in the point Cleaning should be followed and a standard household disinfectant should be used.

Storage

The RFM hair washing basin must be dried after use. For storage, the hair washing basin must be completely dry. Then store it in its packaging (plastic bag) or in similar conditions in a dry and dark place.

Contraindication

Using the RFM hair washing basin is not recommended in case of major sight and balance disturbances, dizziness, head or neck aches and severe loss of strength leading to the patient not being able to lift their head.

Maintenance

The RFM hair washing basin is maintenance-free. If you notice a defect or a change in the material, contact your retailer immediately! Do not use any defective RFM hair washing basin! In case of cleaning and storage according to the instructions, the RFM hair washing basin has a duration of at least 2 years.

Scope of supply

RFM hair washing basin with drainage hose
Hand pump
Instructions for use
Plastic bag

Technical data/dimensions

Weight: 570 g
Material: PVC
Height: 15 cm
Width: 66 cm
Depth: 60 cm
Volume capacity: approx. 5 l
Hose material: PVC
Hose length: approx. 90 cm
Hose diameter: approx. 1 cm

Disposal

A worn-out RFM hair washing basin represents a valuable source of material that can be reused in the industrial cycle. Therefore, do not dispose of it in the household waste, instead allow a government-approved disposal point dispose of it properly. Your local waste-disposal scheme will gladly give you more information.

Guarantee

The guarantee has a duration of 24 months and pertains to product defects that are demonstrably manufacturing or material faults.

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