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Toilet Seat Raiser RFM with side-mounted clamps raises the seat of your toilet by approx. 5 cm. The Toilet Seat Raiser RFM® makes sitting down and standing up as well as using the toilet easier. It helps elderly people, people with handicaps or restricted movement following surgical procedures in their daily use of the toilet. A slight incline of the surface promotes the optimum sitting posture and reduces the bending of the hips and knee joints. The Toilet Seat Raiser RFM® has a hygiene cutout at the front as well as the back and thus provides additional comfort. Thanks to a uniform smooth surface, the product is easy to clean and, thanks to the high-quality workmanship, presents no problem areas on which dirt and residues can build up. The seat is made of polypropylene and is, therefore, extremely robust and durable. The toilet seat raiser can be fitted without problems to practically all conventional toilet bowls and is available with or without a lid. For cleaning, the Toilet Seat Raiser RFM® can be completely immersed in water. It is autoclavable for 1 minute at 85°C, total weight without lid: 0.81 kg, total weight with lid: 1.20 kg, maximum load-bearing capacity 200 kg. Two side-mounted clamps ensure your Toilet Seat Raiser RFM® is held securely in place and so guarantee safe sitting down and getting up. The clamps are made out of nylon and are very robust and durable. The ergonomic shape makes the safe and simple securing or loosening of the seat possible even for users with arthritis or other diseases of the joints in the area of the hands.

Fitting Instructions

- Fit the L-shaped clamps as follows
- Place the seat raiser upside down on a flat surface and place the clamps in the recess provided for them. Carefully slide them along the recess and turn the screws clockwise until the threads engage.
- Adjust the clamps so that a space between the clamps and the toilet rim remains.
- Pick up the seat raiser and place it directly on the rim with the clamps facing backwards. The turned down front lip must touch the front toilet rim on the outside of the bowl.
- Turn the screws on the clamps clockwise until you feel a noticeable resistance and the seat raiser can no longer be moved on the bowl.
- **Attention!** The screws have been specially developed for easy turning / light tightening. Therefore do not continue turning when you notice resistance that indicates the screws are already tight.
- Check that the Toilet Seat Raiser RFM does not move before using it.

Cleaning / Disinfection

- The toilet seat raiser should be cleaned and disinfected daily. Before giving it to another person, thorough cleaning and disinfection are absolutely necessary. All parts can be cleaned with warm water and neutral detergents.

Alternatively the seat raiser can be immersed in water and / or be sterilized in an autoclave for 1 minute at 85°C, **without the clamps**. Do not use any abrasive or caustic cleaning agents or high pressure or steam cleaners!

Warning

- Do not exceed the maximum weight: 200 kg.
- Always check that the Toilet Seat Raiser RFM does not move before using it.
- Do not use the seat raiser as a handhold when sitting down or getting up.

Attention: Do not over-tighten the screws!

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Rehaforum Medical GmbH