## Rollator Actimo® Light

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### Manual

UK

Read carefully the complete instructions for use before installing, adjusting or using the Rollator Actimo Light. Keep this manual in a safe place for future reference.

#### **Intended purpose and instructions for use**



The Actimo Light rollator, which is intended for reuse, is suitable for use in the house and outdoors. The rollator has swivelling front wheels, is pushed and serves to support people with impaired mobility following illness or operations when they walk. The rollator has a sitting function, a bag and storage for transporting utensils and commodities. The rollator can be folded up to save space by folding up the seat and pushing the rollator together. The seating surface allows the user to take breaks. The handle height is adjustable.

#### **Indication**



The rollator can be used for minor balance disturbances and impaired mobility due to loss of strength or reduced ease of movement. The Actimo Light rollator is suitable for maintaining corporal activity. The padded back rest allows safe seating.

#### **1. Installation/Instructions: Your specialist dealer or nursing staff will be happy to assist you during assembly**

- a. Remove all parts from the carton, check for completeness and damage; place this handy.
- b. Press down the seatbelt to unfold the rollator; In in this case, the upper crossbar must audibly engage.
- c. The rollator is now secured and unfolded. Insert the frame part with the front wheels into the rollator frame and adjust the height with the silver push buttons.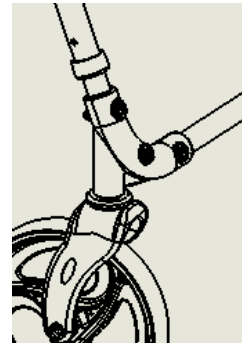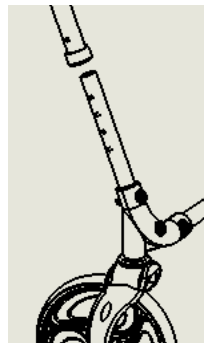

- d. Insert the two rear wheels into the openings of the rear frame part while pressing the silver push buttons. Adjust the desired height by inserting the silver buttons into the opening provided for this purpose. Make sure that the brake cables are not pinched!
- e. Move the rollator and insert the brakes from the top into the rack up to the desired height. Fix this with the wing screws.
- f. To attach the net, place the loops open around the rollator underneath the seat and close them with the pushbuttons.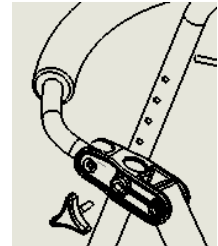
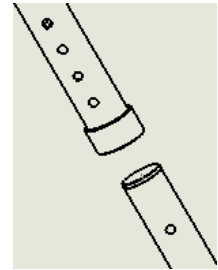


**2. Safety instructions** ⚠

- a. Use the rollator only when it is in good condition and make sure that the seat is locked into the crossbar.
- b. The ends of the handles must point parallel to the ground. Check the tightness of the wing screws before using the rollator!
- c. Test the brakes before **each** use and always use the right and left brakes at the same time for braking!
- d. Take care of the wear of the tires! Have damaged parts replaced by your dealer!
- e. Use the rollator on a level surface - never on stairs or high edges!!
- f. Always hold on to both halves to maintain the balance.
- g. Do not overload the rollator. (Maximum luggage load: 5 kg)
- h. **Always** sit down with brakes on the seat board!
- i. The rollator is not a wheelchair but only a walking aid. Do not let yourself slip when you sit on the seat board!
- j. Maximum user weight: 136 kg!
- k. Before transfer the rollator Actimo Light to another person, check it for damage!

**3. For braking, the rollator has a controllable braking system and a parking brake**

- a. To brake when using the rollator, pull both brake handles upwards simultaneously. The braking power intensity can be adjusted according to the speed and the characteristics of the ground surface.
- b. To lock the brake, press the brake handles down with the handlebars until they feel audible and audible.
- c. To unlock the brake, pull both handles evenly upward until they come out of the lock.

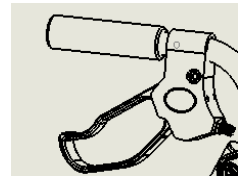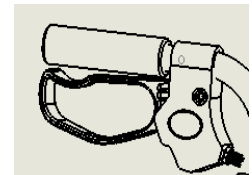**4. Folding and unfolding the rollator**

- a. Before you fold the rollator, empty the net.
- b. Stand behind the rollator and make sure the brakes are tightened. Fold the seat up and slide the rollator together.
- c. To unfold, stand behind the rollator and release the backrest. Press the seat downwards making sure that the seat is clicked into place. You can release the brakes again and the rollator is ready for use.

## 5. Cleaning

- a. Clean and disinfect the rollator at regular intervals, for example, daily. Before you transfer the walker to another person, thorough cleaning and disinfection are mandatory.
- b. Use a mild soapy solution without abrasive additives. Disinfection can be carried out using commercially available agents as per Manufacturer's instructions, for example, Descosept AF. Do not use abrasive or corrosive cleaning agents or high pressure and steam cleaners!
- c. Ensure that no cleaning liquid runs into the frame tubes to avoid corrosion!

## 6. Reuse

The rollator is intended for reuse. Several people can use the rollator. Before the rollator is used by another person or is transferred to the ownership of another person, it should be thoroughly cleaned and disinfected to prevent the transmission of germs (contamination). For this, the procedure described in the point "Cleaning" should be followed and a standard household disinfectant should be used.

The rollator is maintenance-free. If you notice a change in the material, the wheels or the braking behaviour, contact your retailer immediately!

## 7. Maintenance

The rollator Actimo Light is maintenance-free.

Functional error: 

If you notice a decrease in the braking force, a sluggishness of the wheels, a rough running of the wheels or other disturbances, such as rust on the linkage, consult your dealer. Do not use a damaged rollator!

## 8. Storage/Transport

Protect the rollator from moisture and store it in a safe place to protect the linkage and the wheels from damage! For safe transport, the rollator must be folded; for this, pull the seat board upwards at the handle, after removing the adapter.

## 9. Contraindication

The use of the rollator is unsuitable for perceptual disturbances, severe equilibrium disturbances, loss of limbs on both arms, joint injuries or articular contractions on both arms and reduced or insufficient vision.

## 10. Scope of supply

- Rollator Actimo Light
- Shopping bag/Net
- Instructions for use

## 11. Technical data/dimensions and weight:

Width (frame): 61 cm

Weight: 7.2 kg, size when folded: approx. height 98 to 118 cm x width (frame) 61 cm x depth 38 cm

Material: aluminium (anodised), seat cushion polyurethane/polypropylene, back rest EVA, wheels polypropylene/PVC

Seat height (incl. cushion): approx. 53 – 61 cm

Seat width: 36 cm

Height (adjustable handles): approx. 85- 99 cm

Width between the handle centre lines: 50 cm

Handle width: 30 mm

Turning radius: 94 cm

Wheel size: 8"

Depth: approx. 80 cm

Max. load capacity: 136 kg, max. load/bags: 5 kg

Recommended body size: 160 – 190 cm

## 12. Warranty

The warranty period is 24 months and refers to all product deficiencies which are demonstrably due to manufacturing or material defects.

Daimlerstr. 12a  
25337 Elmshorn  
Tel.: +49 (0)4121 473312  
Fax: +49 (0)4121 473323  
[www.rehaforum.com](http://www.rehaforum.com)

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