Gebrauchsanweisung	3	DE
Instructions for use	9	EN
Instructions d'utilisation	15	FR
Istruzioni per l'uso	21	IT

D^{care}® **amalia**

LED Lese- und Pflegeleuchte
LED care and reading light
Lampe de lecture et de soins à LED
Luce a LED per lettura e per visita

SYMBOLE / SYMBOLS / SYMBOLES / SIMBOLI

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EN Welcome to Derungs medical Lighting

Thank you for purchasing a Derungs Light product.

High product quality standards as well as a customer-friendly service are the basis for the globally expanding of success of Derungs Light.

If you would like to use our service, please contact our experts:

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









Or contact our argents in your country.

Your Derungs Light-Team

CONTENT

1	VARIANTS	10
2	FOR YOUR SAFETY.....	11
3	ASSEMBLY	11
4	CONNECTION	12
5	OPERATION	12
6	TROUBLESHOOTING	13
7	REPLACING THE LIGHT SOURCE.....	13
8	CLEANING	13
9	REPAIR	13
10	DISASSEMBLY	13
11	DISPOSAL	14
12	TECHNICAL DATA.....	14

1 VARIANTS

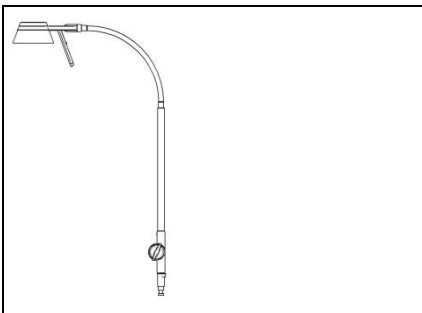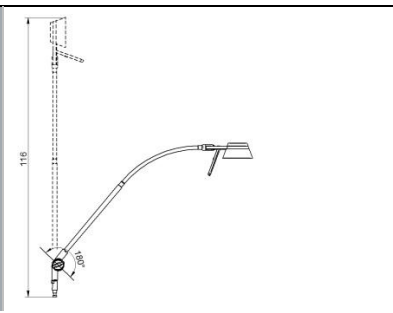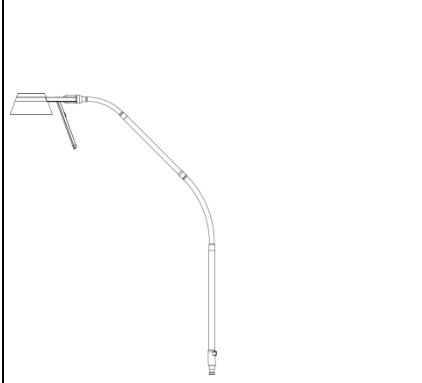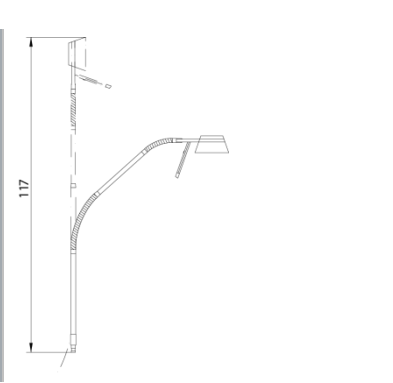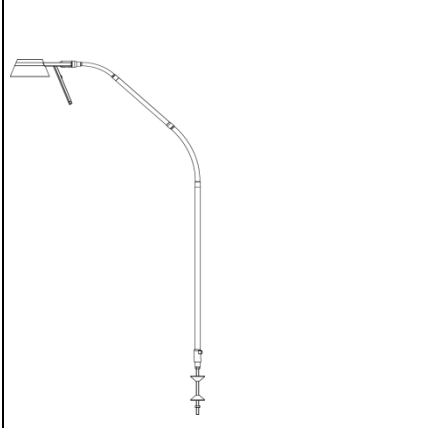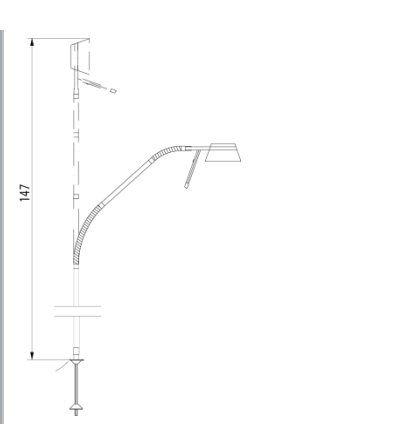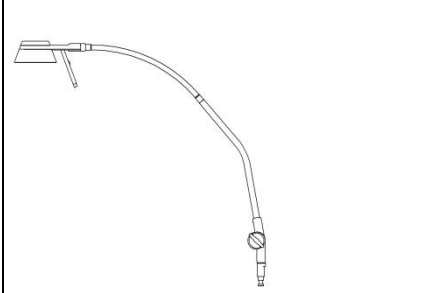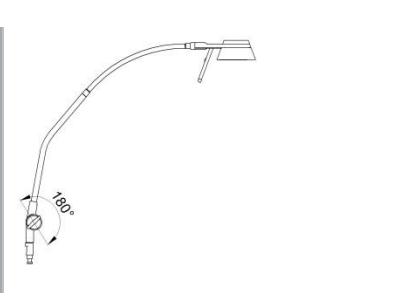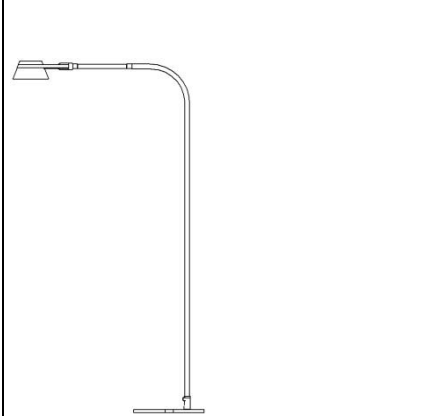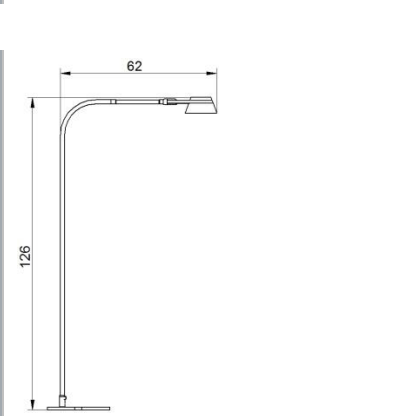	<p>amalia 9 P S4 D15.250.000 Version with ankle and gooseneck arm</p> <ul style="list-style-type: none"> • for rail fasteners • for wall mounting • for mounting with table clamp • for universal mounting 	
	<p>amalia 9 P S7 D15.251.000 Version with two gooseneck arms</p> <ul style="list-style-type: none"> • for rail fasteners • for wall mounting • for mounting with table clamp • for universal mounting 	
	<p>amalia 9 B S8 D15.252.000 Version bed connection</p> <ul style="list-style-type: none"> • with integrated bed adapter 	
	<p>amalia 9P S4 clinic D15.360.000</p> <ul style="list-style-type: none"> • for rail fasteners • for wall mounting • for mounting with table clamp • for universal mounting 	
	<p>amalia 9F S3 D15.368.000 – white aluminum D15.372.000 – pure white</p> <ul style="list-style-type: none"> • floor stand luminaire 	

2 FOR YOUR SAFETY

The light is developed in accordance with the latest technical standards, made of superior materials, produced with great care and tested.

However personal injury or property damage may occur, when using the light.

- ▶ Read all included instructions and informations.
- ▶ Please take notice of the alerts in the documents and directly at the device.
- ▶ The device must be used in technically perfect condition, and only by safety-conscious persons who are fully aware of the risks.
- ▶ Keep all the instructions available at the device.

Intended use

The light is designed for the indoor use to support the patient when reading as well as the medical staff in their care work.

The light is designed for use with hospital and nursing beds. The light is not designed for use as treatment and examination light or operating light!

Explosion prevention

The use of the light in explosive and hazardous atmospheres could result in an explosion and in death or serious injury.

- ▶ Do not use in explosive and hazardous atmospheres.

Danger of death from electric shock

Electric current in the light could result in death or serious injury.

- ▶ Check: The mains voltage with the nominal voltage and the frequency must match the data on the name plate.
- ▶ Before starting work disconnect the light from mains.
- ▶ All maintenance and repairs must be done by qualified personnel trained and authorised by the manufacturer.
- ▶ Disconnect damaged cable from the mains. Repairs to the cable must be done immediately by qualified personnel trained and authorised by the manufacturer.

Danger of unsuitable spare parts

The use of unsuitable spare parts could result in an injury and property damage.

- ▶ Use of spare parts to be released by the manufacturer only.
- ▶ The manufacturer or a service technician authorized by the manufacturer or a similarly qualified person may only replace the light source of this luminaire.

Alert levels in this document

DANGER

Warnings against hazards that may result in **serious injuries or death** in case of non-observance.

WARNING


Warnings against hazards that may result in **injuries** in case of non-observance.

CAUTION

Warnings against hazards that may result in **material damage** in case of non-observance.

3 ASSEMBLY


Connect the gooseneck arm with the light head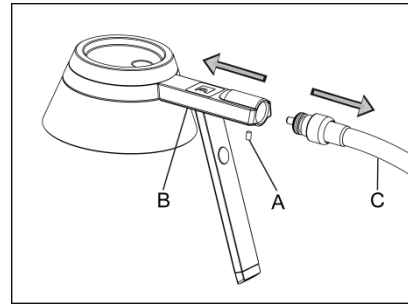


- ▶ Loosen the grub screw **A**.
- ▶ Hold the gooseneck arm **C** and connect the light head **B**.
- ▶ Tighten the grub screw **A**.

Assembly of the integrated bed adapter

Assembly only for type 9 B S8 (Version with bed adapter).

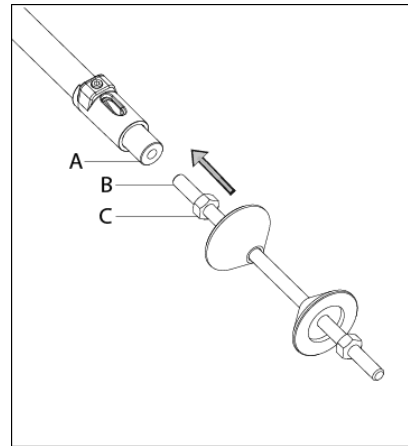
- ▶ Fit the thread rod **B** into the gooseneck arm **A** until the limit stop and lock the nut **C**.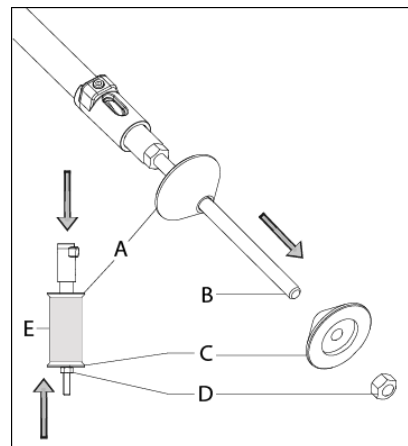

- ▶ Disassemble the cone **C** and the nut **D** of the thread rod **B**.
- ▶ Insert the light with the thread rod **B** and the cone **A** into the cylinder **E** bedside mounted.
- ▶ Assemble the cone **C** and the nut **D** and adjust the light.
- ▶ Lock the nut **D**.

Assembly of floor stand version (amalia 9 F S3):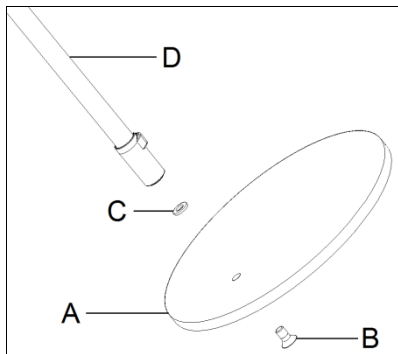

- ▶ Hold the base plate **A** with the cork ring and the countersunk hole downwards.
- ▶ Insert the M8 countersunk screw **B** into the base plate.
- ▶ Put the lock washer **C** onto the screw.
- ▶ Put on the stand pipe **D** and fasten the M8 countersunk screw **B** (20Nm).

4 CONNECTION


VORSICHT

Property damage caused by wrong supply voltage.

Destruction or damage of the light.

The mains voltage with the nominal voltage and the frequency must match the data on the name plate.

Connect the light to the mains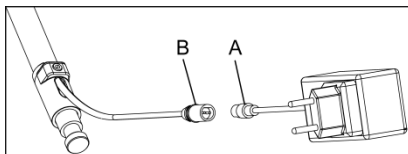


- ▶ Connect the double-pole plug **A** with the socket **B** of the power unit.
- ▶ Connect the power supply with the socket

5 OPERATION


Arm versions:

Light operation reading light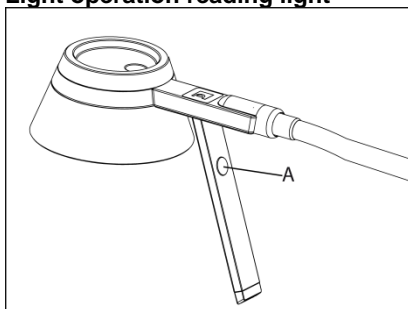

- ▶ Touch the sensor **A** and switch the LED reading lights on and off.


Light operation night light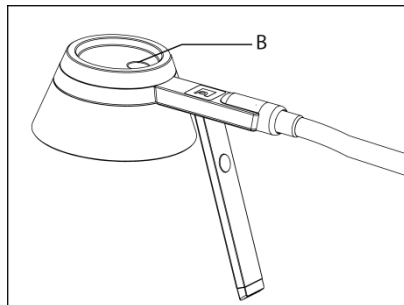

- ▶ Touch the sensor **A** and switch the LED night lights on and off.


If the night lights are switched on, the reading light switched off automatically.

Turnover system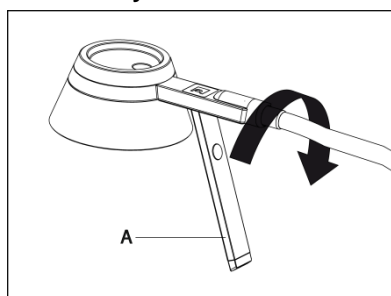

- ▶ With the handle **A** turn the light head into the desired position. The light head can be turn through 360°

Floor stand version (amalia 9 F S3):

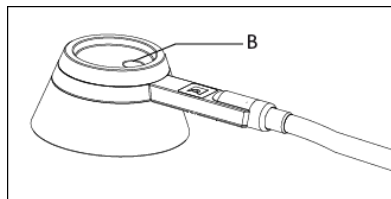- ▶ Touch the sensor **B** and switch on the light (LED reading light and LED night light).

6 TROUBLESHOOTING

Problem	Possible cause	Fault clearance
The light does not glow.	The mains plug is not connected to the power outlet.	▶ Connect the mains plug.
	Defective LED module.	▶ Contact our experts.
The light does not react to inputs.	Software error.	▶ Disconnect the main plug for a few seconds. ▶ Contact our experts, if the light is still not react.
<p>If you would like to use our services please contact us, our experienced experts are on hand for you at any time:</p> <p>Service-Hotline: +41 71 388 11 66</p> <p>Service-E-Mail: mailbox@derungslicht.com</p> <p>Or contact our argents in your country.</p>		

7 REPLACING THE LIGHT SOURCE

Notice: The light is maintenance-free. A light source replace is not necessary. If a light source should fail, contact our experts.

8 CLEANING

DANGER

Danger of death from electric shock

- ▶ Disconnect light from mains.
- ▶ Do not clean wet.

CAUTION

Property damage caused by wrong cleaning.

Damage of the light.

- ▶ Check the compatibility of the cleaning agents with the surface.
- ▶ For cleaning do not use alcohol, solvents, and chlorine- or abrasive detergents.
- ▶ Damage of the shade by concentrated disinfectant.
- ▶ Notice the mixing ratios of the manufacturer.

Scratches by wrong cleaning cloth

CAUTION

Cleaning of the outer light elements in accordance with regulations of the following diluted products:

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Sagrotan®- disinfectant fast cleaner

9 REPAIR

CAUTION

Property damage caused by inappropriate repair.

Damage or destruction of the light.

- ▶ Repairs must be done by qualified personnel trained and authorised by the manufacturer.

Use of spare parts to be released by the manufacturer only.

10 DISASSEMBLY

DANGER

Danger of death from electric shock

- ▶ Disconnect light from mains.


CAUTION

Danger of injury by falling light head.

Personal injury and property damage.


Hold the light head.

Disassembly of light head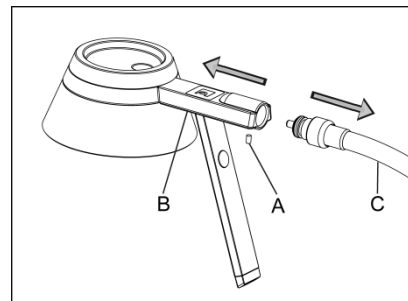

- ▶ Loosen the grub screw **A**.
- ▶ Hold the gooseneck arm **C** and disconnect the light head **B**.

Disassembly of the shade

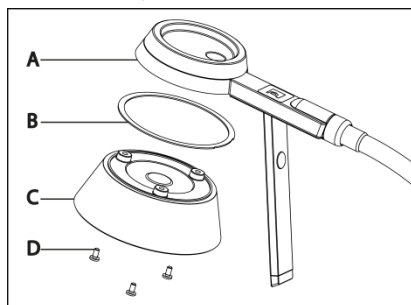- ▶ Unscrew the three fixing screws **D** of the light head **A** and remove the shade **C** with the locking ring **B**.

11 DISPOSAL

The light meets the requirements of the European directive 2002/96/EC.

Do not dispose of the light with the normal refuse. Dispose of the light at a state-certified recycling center. This helps you avoid possible negative consequences for health and the environment

12 TECHNICAL DATA

Note: The specifications on the name plate of the light apply. The name plate is on the back side of the supply unit.

Electrical values:

Rated voltage	 100 - 240 V
Frequency range	50 / 60 Hz
Power consumption	8W
Assembly	LED module
Operating device	Power supply

Classification:

Protection classification	II
Protection class	IP 21
Operating mode	Continuous operation
Standards	EN 60598-1; EN 60598-2-25
Blue light hazard according to EN/IEC 62471	RG 1 (low risk)

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Waldmann Illuminotecnica S.r.l.
Derungs Licht AG