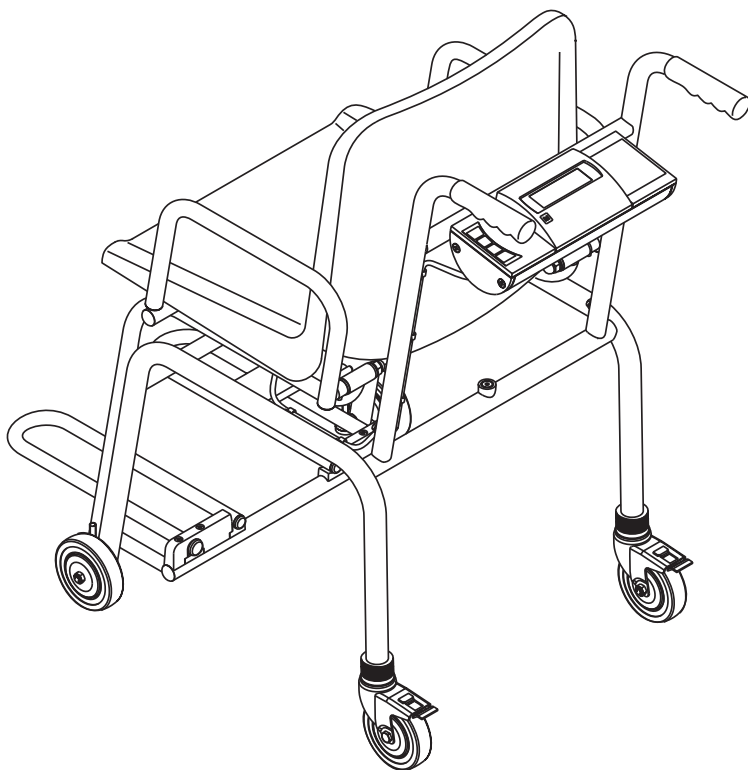
**seca 959 /  
seca 958**

<b>D</b>	<b><i>Bedienungsanleitung und Garantieerklärung</i></b>	<b>3</b>
<b>GB</b>	<b><i>Instruction manual and guarantee</i></b>	<b>17</b>
<b>F</b>	<b><i>Mode d'emploi et garantie</i></b>	<b>31</b>
<b>I</b>	<b><i>Manuale di istruzioni e garanzia</i></b>	<b>45</b>
<b>E</b>	<b><i>Manual de instrucciones y garantía</i></b>	<b>59</b>
<b>DK</b>	<b><i>Betjeningsvejledning og garantibevis</i></b>	<b>73</b>
<b>S</b>	<b><i>Bruksanvisning och garanti</i></b>	<b>87</b>
<b>N</b>	<b><i>Bruksanvisning og garantierklæring</i></b>	<b>101</b>
<b>FIN</b>	<b><i>Käyttöohje ja takuu</i></b>	<b>115</b>
<b>NL</b>	<b><i>Bedieningshandleiding en garantieverklaring</i></b>	<b>129</b>
<b>P</b>	<b><i>Instruções de utilização e declaração de garantia</i></b>	<b>143</b>
<b>GR</b>	<b><i>Οδηγίες χειρισμού και εγγύηση</i></b>	<b>157</b>

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---

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# Full certification

---

In seca products you are purchasing not only technology matured over a century, but also quality confirmed by authorities, the legal system and by institutes. seca products already meet the quality and technology requirements of tomorrow. With seca, you are buying the future.

**M**

The products in these operating instructions comply with German legislation governing medical products and therefore with Directive 93/42/EEC of the Council of the European Community which is embodied in national laws throughout Europe.

Scales carrying this symbol are approved for medical purposes in the European Community. seca scales with this symbol meet the stringent quality and technical requirements placed on scales which can be calibrated.

Scales carrying this symbol are calibrated in accordance with Precision Class III of EC Directive 90/384/EEC.

Products carrying this symbol meet the requirements

1. of the Directive for non-automatic scales in medicine, No. 90/384/EEC
2. for the law governing medical products, EC Directive No. 93/42/EEC
3. the Directive on electromagnetic compatibility

seca has been awarded the EN ISO 9001 certificate, which means that our company adheres to strict specifications for consistent high quality in research, production, sales and service. Furthermore, EN 46001 certifies seca's rigorous compliance with the most stringent European quality standards for medical products.

seca helps the environment.

We are concerned to save natural resources. We therefore strive to save packaging material where meaningful. What is left over can conveniently be disposed of locally via the German Dual System (recycling programme).

# 1. Congratulations!

---

By purchasing the **seca 958/959** electronic chair scale, you have acquired a highly-accurate and sturdy piece of equipment.

seca has been putting its experience at the service of health for over 150 years now, and as market leader in many countries of the world, is always setting new standards with its innovative developments for weighing and measuring.

In accordance with national regulations, the **seca 958/959** wheelchair scale with standing aid is used primarily in hospitals, doctors' surgeries and in-patient nursing facilities. The scale is a class I medical product and is approved to CE class III (seca 959)/III (seca 958).

Weight is determined within a few seconds.

The **seca 958/959** can be moved on castors and its low power consumption combined with its rechargeable batteries mean it will provide long, mobile service.

**seca 958/959** chair scales are made for people who are not especially steady on their feet. To give you a safe feeling while weighing, the chair scale has a solid tubular frame and a firmly bolted seat. The free-running casters make the scale very easy to move. The wheel-locking lever, together with the swiveling armrests and foot supports, makes it easier to sit in safety.

The scale is of a very sturdy construction and will give you long and loyal service. It is easy to operate and the large display is easy to read.

## 2. Safety

---

Before using the new scale, please take a little time to read the following safety instructions.

- Follow the safety instructions in the instruction manual.
- Keep the operating manual and the declaration of conformity contained therein in a safe place.
- Have scale serviced and re-calibrated on a regular basis (see "Servicing/recalibration" on page 28).
- Be sure to use only the seca mains unit supplied. Before plugging the mains unit into a socket, check that the mains voltage indicated on the mains unit matches local mains voltage.
- Ensure that the cables are not crushed and do not come into contact with hot objects.
- Use only the type of battery stated (see "Spare parts and accessories" on page 29).
- Prevent the scale from rolling away accidentally. Use the wheel brake when the patient is sitting down or getting up and assist people who are unsteady on their feet.
- When using the scale with a mains unit, ensure that the supply cable is routed in such a way as to exclude any type of tripping hazard.

# 3. Before you start...

---

## Unpacking

The scope of supply includes:

- Chair scale comes complete or partly assembled
- 1 seca mains unit
- 1 set of operating instructions

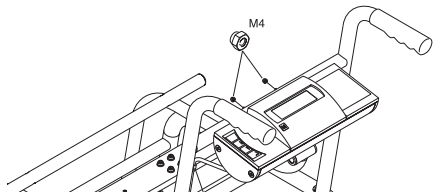
## Assembling

This chair scale is available either fully or partly assembled. To assemble the partly-assembled model, please proceed as follows:

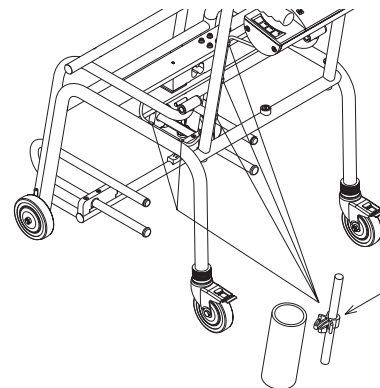
- Locate the handles on the journals provided and bolt in place using the two M6 x 16 bolts, following the diagrams.

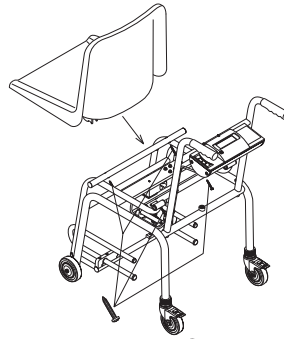
- Thread the mounting bolts for the display unit through the bores provided in the handles and screw the display tightly in place using two M4 nuts.

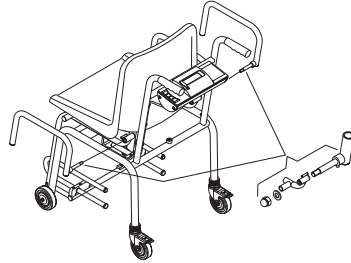
- Attach the weighing cell cable using the enclosed plastic cable clip by pushing this into the bores on the handles.

- Fit the seat to the frame using the four self-tapping plastic screws. Check that the seat is screwed tightly to the frame with no play at all.

- Push the hinge bolts for the armrests through the hinge bushes on the seat frame. Put on the washer and cap nut (M10) and screw them tight.

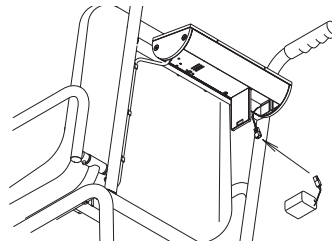
## Power supply

The scale is powered by a mains unit and by rechargeable batteries.

Connecting the rechargeable battery pack:

- Open the battery compartment in the base of the display housing.
- Connect the plug contacts to the rechargeable battery pack located in the battery compartment.
- Close the lid of the battery compartment again.

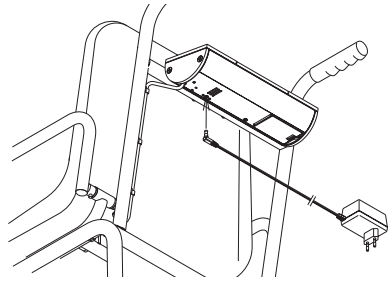
The LC display and the evaluation electronics have a low power consumption – 1,000 weighing operations can be performed with a fully charged battery.

Once battery voltage is no longer adequate, the battery symbol  first appears in the display to indicate that battery power is dropping.

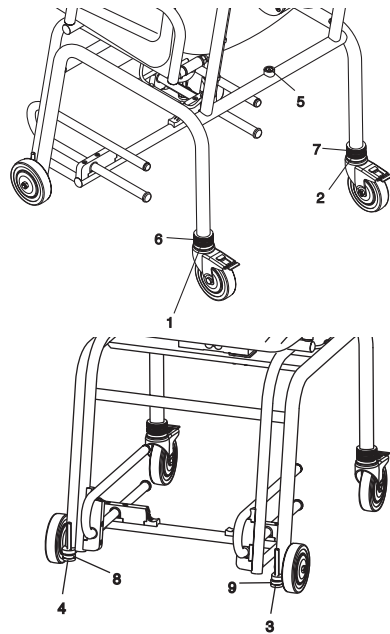
If the batteries are not charged, battery voltage will ultimately not be adequate for error-free measuring. In this case, **bAtte** appears in the display and weighing is no longer possible.

The rechargeable batteries are charged by the mains unit supplied. For this purpose, the mains unit needs to be connected to the mains for at least 24 hours.

## 4. Setting up and aligning the scale

- Place the scale on a level surface.
- Align the scale using the two bottom knurled nuts (1 and 2) and the two adjustable feet (3 and 4) so that the display in the spirit level (5) is central. This setting can be secured using the top lock-nuts (6, 7, 8 and 9).

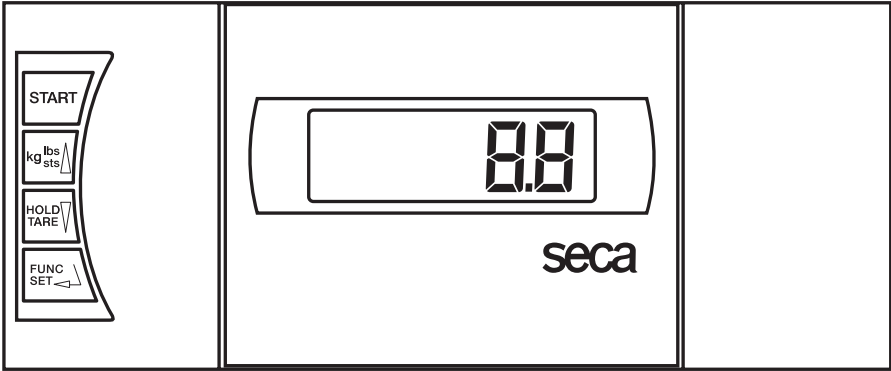
spirit level

# 5. Operation

## Controls and displays

Controls on the display

Switch scale on and off

in normal operation → no function  
in function menu → increase value

press briefly → activate hold function  
hold down → set display to zero, activate/deactivate tare function  
in function menu → decrease value

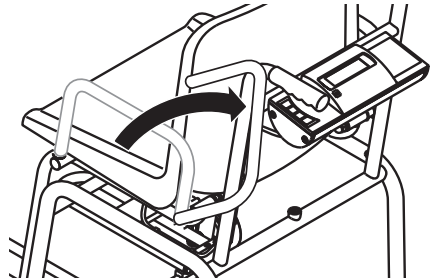
Activating the BMI function, Confirmation

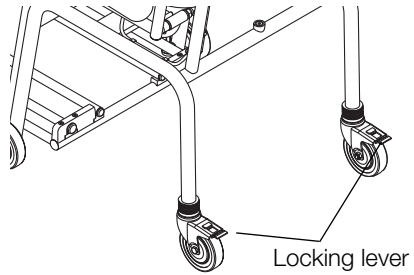
## Other controls

The armrests on the chair scale can be swivelled backwards independently. This is useful because it makes it easier for the person being weighed to sit down.

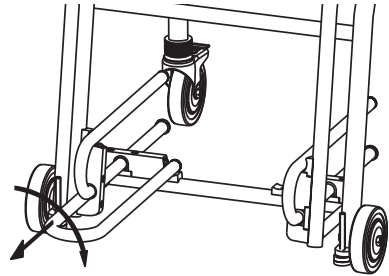
- Ensure that you do not trap any part of the body or pieces of clothing when folding the rests down.

- The rear casters can be locked if you push down the two locking levers on the casters. This device should be used when the patient is sitting down on the scale or getting up from it.

- Once the person who is to be weighed is seated, the foot-rests are pulled out and tilted downwards. The patient can now place his or her feet on the foot-rests.

## Checking that the contents of the calibration counter are correct

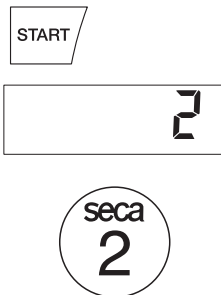
This seca scale is calibrated. Calibration may be performed only by authorised centres. To ensure this, the scale is fitted with a calibration counter which records any change to data relevant to calibration. If you want to check whether the scale is properly calibrated, proceed as follows:

- If necessary, switch off the scale.
- Keep any key depressed and start the scale.

The current contents of the calibration counter are shown on the display for a few seconds.

- Compare the output contents of the calibration counter with the number shown on the calibration counter sticker.

For valid calibration, both numbers must match. If sticker and calibration counter do not match, then re-calibration is required. Contact your service agent or seca Customer Service.

**Both numbers must match.**

If a supplementary calibration is necessary, then the supplementary calibration mark illustrated opposite is used to record the calibration counter position instead of the seca calibration mark shown above. This mark is secured with an additional seal by the person authorised to perform the supplementary calibration. The supplementary calibration mark is available from seca Customer Service under the number 14-05-01-886.

## Correct weighing

- Press the Start key.  
**SECA, 88888** and **00** appear consecutively in the display. The scale is then automatically set to zero and ready to use.
- Get the patient to sit on the scale.
- The patient's weight can only be determined once he or she is no longer in contact with the floor. Be sure, therefore, that the patient's feet are on the foot rest throughout the weighing operation.
- Read the weighing result off the digital display.
- If a load greater than 200 kg is placed on the scale, **STOP** appears in the display.
- To switch off the scale, press the Start key again.

## Automatic switch-off/economy mode

The scale switches off automatically after 5 minutes.

## Taring weight display (TARE)

The Tare function is very useful if an additional weight (a blanket, for example) is to be ignored for weighing purposes.

Proceed as follows:

- Press the green Start key.

- First place the additional weight (a blanket, for example) on the scale and activate the Tare function by pressing and holding the **HOLD/TARE** key.

The display is reset to zero. **0.0** appears in the display. The note “NET” appears.

- Now put the patient and the additional weight on the scale.

The scales calculate the patient’s weight without the additional weight. You can now perform as many weighing operations as you like.

- Hold the key down again to deactivate the Tare function; the scale is then back in normal weighing mode and the tare value has been erased.
- The tare value is likewise erased by the scale being switched off.

## Store weight value (HOLD)

You can also continue displaying the weight determined once the load has been removed. This makes it possible to look after a patient first, before having to note the weight:

- Press the **HOLD/TARE** key whilst the load is on the scale.

“HOLD” appears in the digital display. The value is stored until the scale switches off automatically.

The triangle with the exclamation mark indicates that this frozen display is an additional facility which is not subject to calibration.

- The HOLD function can be switched off by pressing the **HOLD** key again.

**0.0** appears in the display if the scale has no load on it, otherwise the current weight. The “HOLD” display vanishes.

### Note:

**The HOLD and TARE functions can also be used simultaneously.**

## Body Mass Index (BMI)

Body Mass Index puts height and weight in relationship to one another, providing more accurate information than ideal Broca weight, for example. A tolerance range is given which is considered ideal for health. The Body Mass Index applies only to adults: these values are not relevant to children and young people.

- Press the Start key.  
**SECA**, **88888** and **00** appear consecutively in the display. The scale is then automatically set to zero and ready for operation.

- Press the **FUNC** key.  
On the display, you will see the last height entered.

- You can use the arrow keys to adjust the value in increments according to the graduations displayed.

- Once the correct value is set, press the **FUNC** key again. The BMI function is now activated.

- Get the patient to sit on the scale.
- Read off the BMI on the digital display and compare it with the categories on page 26.

- Press an arrow key if you would like to see the weight briefly.

- To return to normal weighing mode, press the **FUNC** key.

- To switch off the scale, press the Start key again.

## Evaluating Body Mass Index:

Compare the value determined with the categories shown below.

- **BMI below 18:**  
Warning, this patient weighs too little. There could be a tendency to anorexia. An increase in weight is recommended to improve well-being and performance. If in doubt, consult a specialist.
- **BMI between 18 and 20:**  
Patient is at lower weight limit. If he feels well and is completely healthy, his weight can stay as it is.
- **BMI between 20 and 25:**  
The patient is of a normal weight. He can stay as he is.
- **BMI between 25 and 30:**  
The patient is slightly to moderately overweight. He should cut his weight if he is already ill (e.g. diabetes, hypertension, gout, fat metabolism disorders).

- **BMI over 30:**

Weight reduction is essential. Metabolism, cardiovascular system and bones are all under strain. A consistent diet, plenty of movement and behavioural training are all recommended. If in doubt, consult a specialist.

## 6. Cleaning

---

Clean the seat and the housing as required using a domestic cleaning agent or commercially-available disinfectant. Follow the manufacturer's instructions.

Under no circumstances use abrasive or acid cleaners, white spirit, benzene or the like for cleaning. Such substances can damage the high-quality surfaces.

## 7. What do I do if...

---

### ... no weight display comes on?

- Is the scale switched on?
- Check the mains connection and the connection of the battery pack.
- Switch the scale off for a few seconds. This can be done by opening the plug connected to the rechargeable battery block and unplugging the mains supply from the scale.

### ... does not appear before weighing?

- Switch off the scale with the Start key and start the scale again – there must not be any load on the scale – and only its feet should be in contact with the floor.

### ... one segment is illuminated constantly or not at all?

- The relevant segment has a fault. Inform Servicing.

### ... the display shows ?

- Battery voltage is dropping. Charge the rechargeable batteries in the next few days.

### ... appears in the display?

- The rechargeable battery is so severely discharged that weighing is no longer possible. Recharge the battery.

### ... appears in the display?

- Maximum load has been exceeded.

### ... the display flashes?

- If you have just activated the Hold or Tare function, please wait until the scale has detected a steady weight value. The display will then stop flashing.
- If there has been no activation of a function beforehand, remove the load from the scale and wait until  is displayed, then weigh again.

### ... the display appears?

- The ambient temperature of the scale is too high or too low. Place the scale in an ambient temperature between +10 °C and +40 °C. Wait about 15 minutes for the scale to adapt to the ambient temperature and then weigh again.

### ... the display E and a number appear?

- Switch off the scale with the Start key and start the scale again. The scale will then work normally again.  
If this is not the case, disconnect the power supply by briefly removing the batteries. If this measure is equally unsuccessful, inform the Service department.

## 8. Servicing/recalibration

---

Have the scale recalibrated by authorised persons in accordance with national legal requirements. The year of initial calibration can be found behind the CE symbol on the rating plate above the number of the appointed office, 0108 (Hamburg Calibration Office). Recalibration is essential if one or more security marks are damaged or the calibration counter does not match the number on the applicable calibration counter mark (cf. page 23).

We recommend having your scale serviced by your local service agent before having it recalibrated. seca Customer Service will be pleased to give you further assistance.

## 9. Technical data

---

<b>Maximum load</b>	200 kg
<b>Graduations</b>	
<b>seca 959:</b>	
0 up to 200 kg:	100 g
<b>seca 958:</b>	
0 up to 100 kg:	100 g
100 up to 200 kg:	200 g
<b>Dimensions</b>	
height:	920 mm
width:	565 mm
depth:	810 mm
<b>Weight</b>	approx. 24.5 kg
<b>Temperature range</b>	+10 °C to +40 °C
<b>Power supply</b>	rechargeable batteries/mains unit
<b>Power consumption</b>	approx. 25 mA
<b>Minimum load</b>	2 kg
<b>Calibration</b>	
<b>seca 959:</b>	approved for medical use, Class III
<b>seca 958:</b>	approved for medical use, Class III
<b>Medical product as per Directive 93/42/EEC</b>	
<b>EN 60 601-1:</b>	Class I
Insulated equipment, Protection Class II:	
medical electrical equipment type B:	

## Accuracy at first calibration

### seca 959:

to 50 kg:	± 50 g
50 kg to 200 kg:	± 100 g

### seca 958:

to 5 kg:	± 50 g
5 kg to 20 kg:	± 100 g
20 kg to 100 kg:	± 150 g
100 kg to 200 kg:	± 300 g

### Functions

Tare, Hold, BMI

# 10. Spare parts and accessories

---

## Power supply

### Model dependent mains unit

Description: plug-in mains unit, Euro: 12 V / 130 mA

seca spare part no.: 68-32-10-252

Description: plug-in mains unit, USA: 12 V / 150 mA

seca spare part no.: 68-32-10-259

Description: desk-top mains unit, switchable, Euro

seca spare part no.: 68- 32-10-243

### Plug-in mains unit

Use only seca original plug-in mains units with 9 V or controlled 12 V output voltage. Commercially-available mains units may supply a higher voltage than stated and thus lead to damage to the scale.

### Rechargeable batteries

seca spare part no.: 68-21-12-721

## Recalibration

### Supplementary calibration mark

seca spare part no.: 14-05-01-886

# 11. Disposal

---

## Rechargeable batteries

Do not throw old rechargeable batteries into the domestic refuse! Ensure that rechargeable batteries are properly disposed of at local collection points.

## Disposing of the scale

If the scale can no longer be used, the responsible waste disposal association will be glad to advise you on the measures necessary to dispose of it properly.

# 12. Warranty

---

A two-year warranty from date of delivery applies to defects attributable to poor materials or workmanship. All moveable parts - batteries, cables, mains units, rechargeable batteries etc. - are excluded. Defects which come under warranty will be made good for the customer at no charge on production of the receipt. No further claims can be entertained. The costs of transport in both directions will be borne by the customer should the equipment be located anywhere other than the customer's premises. In the event of transport damage, claims under warranty can be

honoured only if the complete original packaging was used for any transport and the scale secured and attached in that packaging just as it was when originally packed. All the packaging should therefore be retained.

A claim under warranty will not be honoured if the equipment is opened by persons not expressly authorised by seca to do so.

We would ask our customers abroad to contact their local sales agent in the event of a warranty matter.

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seca Vogel & Halke GmbH & Co.