Duschemel RFM

DEUTSCH

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ENGLISH

1. Intended use:

The RFM shower stool for use in bathroom areas offers users with restricted mobility a secure seating surface, which can be adjusted to their body size, and which has a maximum load of up to 113 kilograms. Seating surface: 33 cm diameter, seating height: 40.6 cm to 50.8 cm.

2. Warning notice:

Please read these instructions for use carefully before using the shower stool. Keep the instructions safely, so that they are available for later use. Request a copy of the instructions from your retailer if they get lost.


3. Assembly instructions

1. Place the round seating surface upside down on the floor
2. Put the upper (1) and lower (2) frames together and join them with the screw (a), spacer ring (b) and screw cap (c).
3. Place the combined frame on the seating surface in such a way, that the holes for the screws sit over the holes for the seating surface, and then fasten the frame to the seating surface with the screws (d) and spacer rings (b).
4. Slide the leg extensions (3) into the frame and ensure that the legs lock into place.

⚠ Please note: Make sure that all the screws are tightly fastened and that all four legs are locked into place!

List of parts:

No.	Description	Specification	Quantity
1	Upper frame	Ø25.4*1.4*955	1
2	Lower frame	Ø25.4*1.4*955	1
3	Leg extension	Ø28.6*1.4*190	4
a	Screw	M6*37	1
b	Spacer ring	Ø19*1,0	5
c	Screw cap	M6	1
d	Screw	2/3*1/4	4

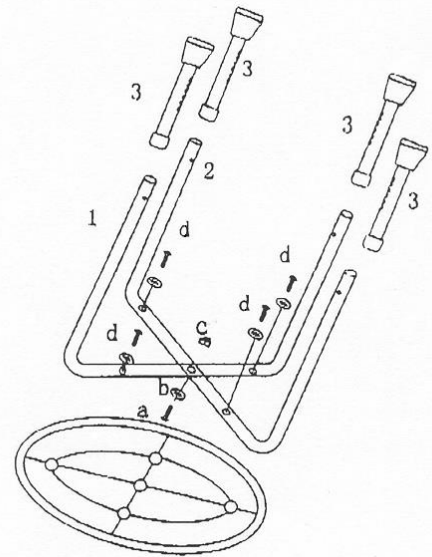4. Safety information:

Before using the shower stool, make sure that all 4 legs are locked at the same height. If the legs are not the same height, the safety of the seat can not be guaranteed!

The shower stool is protected against corrosion. Should you however notice any defects, it is necessary for repairs to be carried out urgently. Never sit on the shower stool, if the legs are loose or at a different height, or if the seating surface is defective. Never load the shower stool above its permitted maximum loading of 113 kilograms.

5. Cleaning information:

The shower stool can be cleaned with the usual commercially available cleaning agents that are used in wet rooms.

6. Information about repairs:

If you have any complaints, please contact your sales outlet or the manufacturer of the shower stool.

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