Algoplaque Algoplaque

—Border/Sacrum

FR PANSEMENTS HYDROCOLLOIDES STERILES

Algoplaque

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EN STERILE HYDROCOLLOID DRESSINGS

Algoplaque

Algoplaque and **Algoplaque Border/Sacrum** are sterile adhesive dressings consisting of carboxymethylcellulose particles in an elastomer network and applied on a polyurethane semi-permeable backing.

Algoplaque

On contact with wound exudate **Algoplaque** and **Algoplaque Border/Sacrum** form a soft moist gel which creates favourable conditions for the healing process. Their excellent adhesion around the wound means that they may be left in place for several days (3-4 days) without any discomfort for the patient.

Because they are flexible, **Algoplaque** and **Algoplaque Border/Sacrum** are particularly intended to wounds located in awkward areas, and as they are water proof and bacteria proof, baths and showers can be taken.

As the protective backing of **Algoplaque** and **Algoplaque Border/Sacrum** is transparent the appearance and formation of the gel can be continually monitored.

Algoplaque

INDICATIONS

Algoplaque is indicated for the local treatment of pressure ulcers and leg ulcers.

Algoplaque Border is indicated for the local treatment of difficult-to-dress areas and/or areas that are prone to friction such as the sacrum or heel.

Algoplaque Sacrum is indicated for the local treatment of pressure ulcers, especially in the sacrum area.

Algoplaque

Algoplaque

INSTRUCTIONS FOR USE

1. Preparation of the wound:

- Clean the wound following the local protocol.

- Dry carefully around the wound.

2. Application of the dressing:

- Remove the protective films.

- Apply the dressing without stretching to the wound taking care not to touch the adhesive area with your fingers.

- Apply the dressing gently to the wound pressing firmly around the edges.

- To apply **Algoplaque Sacrum**, smooth the dressing into place ensuring the narrow part of the dressing is pointing towards the buttocks.

3. Changing the dressing:

The frequency of dressing change is decided by the physician depending on the condition of the wound.

First take off the entire adhesive border than withdraw all the dressing.

- On a clean wound that is free from infection, and where all necrotic tissue has been removed, Algoplaque and Algoplaque Border/Sacrum can be left in place for several days (3-4 days).

After the dressing has been removed the ulcer can be covered by a soluble gel which can be washed away easily.

- For a wound at the end of de-sloughing phase where sloughy tissue remains Algoplaque and Algoplaque Border/Sacrum need to be changed more frequently.

- In a heavily exuding wound, if leakage occurs around the edge of the dressing it should be changed immediately.

Algoplaque

WARNINGS AND PRECAUTIONS FOR USE

- The use of **Algoplaque** for leg ulcers does not replace the need for compression treatment (stockings or bandages) where prescribed.

- The use of **Algoplaque** and **Algoplaque Border/Sacrum** on pressure ulcers does not replace the need for normal nursing care.

- In the case of a deep wound, the use of **Algoplaque** paste reduces the frequency of dressing change of **Algoplaque** provided that the paste does not fill the wound completely.

- **Algoplaque** and **Algoplaque Border/Sacrum** must not be used in a hyperbaric chamber.

- Single use sterile individual packaging: re-using a single use dressing may lead to risks of infection.

- (*) Check that the sterile protector is intact before use. Do not use if package is damaged.

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