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# POSIMED®

## POSIMED®

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## POSIMED®

Positioning pillow

Please read the operating instructions carefully before using the system and heed the **warnings!**

<b>Product name</b>	<b>POSIMED®</b>
<b>scope of delivery</b>	Storage cushion with PU cover, operating instructions
<b>Indications / areas of application</b>	<ul style="list-style-type: none"><li>• Pressure ulcer prophylaxis</li><li>• Pressure ulcer therapy</li><li>• Pain therapy</li></ul>
<b>Contraindications</b>	Not known
<b>Permitted patient weight min. - Max.</b>	Suitable for all patient weights.
<b>material</b>	<ul style="list-style-type: none"><li>• Filling: polypropylene microbeads</li><li>• Cover: PU-coated PES carrier material</li></ul>
<b>Dimensions</b>	See label on the product
<b>Product weight</b>	0.2 - 1 kg
<b>Aid number (as defined in §139 SGB V)</b>	
<b>Medical device</b>	Class 1, MDR 2017/745 
<b>Technical safety / flame retardancy</b>	Flame retardancy test according to DIN EN 597 I + II, BS 7175 sec 3 crib 5, BS 7177
<b>Testing for toxicological harmlessness / biocompatibility</b>	DIN EN ISO 10993-5, DIN EN ISO 10993-1, DIN EN ISO 10993-10
<b>Warnings</b>	Before the first application, the system must be prepared for the first use (see "Preparing the system for use").
<b>guarantee</b>	5 years on the functional properties (pro-rata guarantee)
<b>maintenance</b>	No
<b>Cleaning instructions (pillowcase)</b>	The cover can be disinfected by wiping with commercially available disinfectants made from isopropanol (e.g. Bacillo) or with other suitable DGHM-listed products (e.g. ECOLAB cleaning cloth Incidin OxyWipe S XL). The cover can be reprocessed by autoclaving according to the guidelines of the RKI. 
<b>Cleaning instructions (core)</b>	The entire pillow can be reprocessed by autoclaving according to the guidelines of the RKI.
<b>Prepare the system for use</b>	<ol style="list-style-type: none"><li>1. Remove the pillow from the box.</li><li>2. The system is now ready for use.</li></ol>

RKI = Robert Koch Institute

DHGM = German Society for Hygiene and Microbiology

MPG = Medical Devices Act

Autoclaving = fractional vacuum process at 105 degrees C and 55 minutes

 **Manufacturer in the sense of the MPG and MDR 2017/745**

Funke Medical AG • Ährenfeld 10 • 46348 Raesfeld • Tel. : + 49-2865-266 93 50 • Fax: + 49-2865-8133

info@funke-medical.de • www.funke-medical.de

Download: 02.07.2020 12:30

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Funke Medical AG • Ährenfeld 10 • 46348 Raesfeld • Tel.: +49-2865-266 93 50 • Fax: +49-2865-8133