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

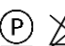
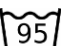
User manual/ instruction for use

Antidecubitusmattress / Decubitus therapy system with evacuation function

Please read the instruction and the notes on safety carefully before using the system to ensure safe operation

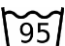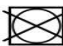Product name	<i>MemoCare EVAQ</i>			
Scope of delivery	Mattress with foam core and PU-TEX Cover with zipper, instruction for use			
Indication	Decubitus therapy up to stage 3 EPUAP) middle risk			
Contraindication	none			
Range of application	Decubitus therapy up to stage 3 middle risk			
Patient weight min.-max.	0 kg – 200 kg			
Material	PUR- foam Cover PU-coating			
Sizes	width: 80-100 cm length: 190-220 cm			
Weight of product (90/200 cm)	11 kg			
Medical device class 1 according 93/42 EWG and medical product law (MPG)				
Technical safety/ flame retardancy	DIN EN 597 I + II // BS 7175 sec 3 crib 5			
Biocompatibility	DIN EN ISO 10993-5 ; DIN EN ISO 10993-1			
Notes on safety	Suitable for beds with adjustable lying surface side bars of the bed must be at least 22 cm above surface			
Guaranty	3 years			

Cleaning/ maintenance/ further use

Homecare	Clinical use
While using	
Cover disinfection by wiping	Cover disinfection by wiping
    	
After changing patient	
Cover disinfection by wiping	Cover disinfection by wiping
Cover disinfection by wiping	autoclaving
    	
No further maintenance	

Instructions for storage and use

1. After receipt, the mattress must be taken out of the cardboard box immediately
2. Put the mattress onto the bed.
3. Please make sure, that the inprint "This side up" is on the the up side.
4. Belts should only be used in emergency cases. Please watch the instructionvideo on www.funke-medical.de !

Manufacturer according to the MPG (German Devices Act)

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Funke Medical AG