Stand: 12/2014

As per: 12/2014

Mise à jour: 12/2014

EU Authorized Representative
Drive Medical LTD
Ainley's Industrial Estate
Elland, West Yorkshire,
United Kingdom HX59JP

Drive Medical GmbH & Co. KG
Leutkircher Str. 44
D-88316 Isny/Allgäu

Gebrauchsanweisung
Operating manual
Mode d'emploi
Gegruiksaanwijzig

Solido

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Suction Cup Grab Bar

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DE / EN / FR / NL

www.drivemedical.de / www.drivemedical.com / www.drivemedical.co.uk

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Indication

For persons who are not able to use bath tub or shower for daily personal hygiene without support of medical devices due to reduced physical functions e.g. instability in standing and/or reduced function of muscles of the lower extremities

Product Contents

Suction grab bar with 2 suction cups. Suction grab bar is ready for use.

Assembling

Assembling requires no tools. Check the vacuum suction discs and the surface it is to be attached to. Clean the surface thoroughly. Press the rubber discs firmly to the surface. The tilting vacuum levers must be loose at this point and the whole of each rubber disc must lie on the surface. (1)

Ensure that the rubber discs are completely on the tile area or other surfaces so that the discs do not cover any grouting lines or other joints. The discs should be positioned at least 1-1.5 cm from all joints. (see 1) First press down one tilting lever and then the second. The resistance of the negative pressure generated must be clearly felt. The greater the resistance felt on the vacuum lever, the better the suction power of the grab bar. If you use the bar regularly, you will get a feel for the vacuum strength and the suction power generated. Before each use, test the firm hold of the suction device. If the adhesion seems insufficient, release the rail and reattach again.

If the required suction power hasn't been achieved, the rail should not be used !

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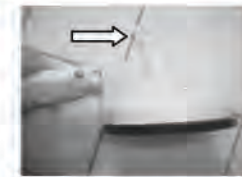
Falsch - wrong

Removal instructions: Release bath latches. Slide fingertip under rubber seal at each end to release suction.

Suction cup exchange

For the exchange of the suction head press the interlock at the side and draw off the suction head in direction of the arrow. Push the new suction head until it lock clearly.

Safety Instruction

ATTENTION:

This product is designed to provide support. It is not designed to hold a person's entire body weight. Before using the grab bar, clean and dry the area where the grab bar is to be attached, and read the instructions carefully.

Be sure to test the stability of the grab bar before every use.

Always make sure that the grab bar is securely attached before each use and always reattach if loose. **Do not use the suction grab bar if it is damaged!**

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Cleaning / Maintenance

Always keep the rubber discs clear of oil and dust. Clean the rubber discs with standard household detergents (no acetic cleaner). Never expose the rubber discs to direct sunlight for longer periods otherwise they will quickly lose their elasticity. Check the rubber discs for possible deterioration. If the suction cups show damage, replace immediately.

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Art.-Nr. – Art. No.	660 100 100	660 100 200	660 100 300
Max. Belastung – Max. load:	70 kg	70 kg	70 kg
Ø Sauger – Ø suction cups:	120 mm	120 mm	120 mm
Gesamtlänge – Length over all:	325 mm	580 mm	880 mm
Länge Grifffläche – Space for hand:	100 mm	320 mm	620 mm
Abstand zur Wand – Distance wall:	35 mm	35 mm	35 mm
Farbe – Color:	weiß – white	weiß – white	weiß – white
Gewicht – Weight:	0,6 kg	1,0 kg	1,3 kg
Material Griff – Material grip:	Kunststoff PP -High-quality plastic PP		
Material Saugteller – Material suction cups:	NBR		
Material Innere Mechanik – Material inner parts:	VA (stainless steel)		

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Warranty

We issue a warranty of 24 months from date of purchase.

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